SenSura[®] Mio 2-piece Flex

Pictorial instructions for use

SenSura[®] Mio

 **Coloplast**


Abbreviated instructions for use. Please refer to product labeling for complete product instructions for use, contraindications, warnings, precautions and adverse events.

Prepare — 1


Measure the stoma.

Prepare — 2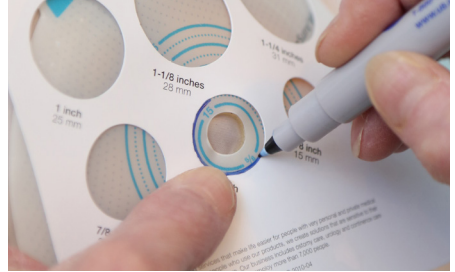

Trace measurement onto back of the barrier.

Prepare — 3

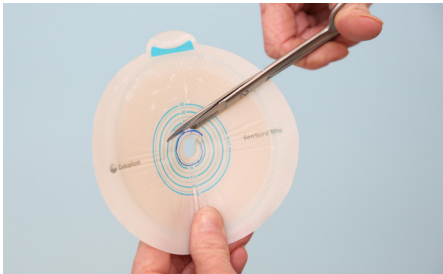
Cut opening in the barrier.

Remove protective backing by pulling the turquoise release tab away from the barrier.

Apply — 1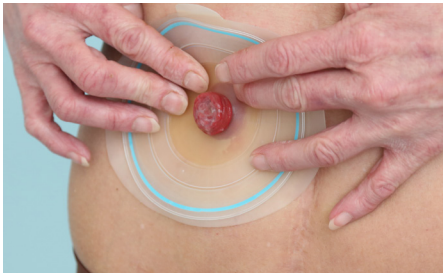

Center barrier around stoma.
Secure to skin by applying gentle pressure.


Apply — 2

Remove the protective paper from the adhesive ring on the pouch.

For product support, call the Coloplast Consumer Care Team at 1-877-858-2656.

Apply — 3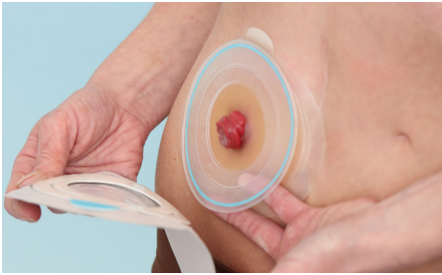

Place pouch by aligning the bottom edge of the adhesive ring with the turquoise line at the bottom of the floating baseplate.

Apply — 4

Secure pouch by applying gentle pressure around the baseplate. When pouch is $\frac{1}{3}$ to $\frac{1}{2}$ full, proceed to either empty or remove.

For drainable pouches only

Empty — 1

Open the outlet by lifting both tabs off the Velcro® plate. Unfold outlet.

Empty — 2

Fold back lower plate to avoid soiling when emptying. Attach Velcro® dots to hold plate in place.

Empty — 3


Empty pouch by pinching outer edges of the outlet open.

Empty — 4

Clean the outlet.

Close — 1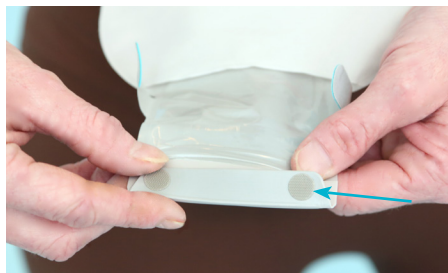


Release the Velcro® dots. Using the guide, fold the lower plate over the upper plate.

Close — 2

Fold upward. Seal the outlet by attaching the Velcro® tabs onto the Velcro® plate.

Remove - 1

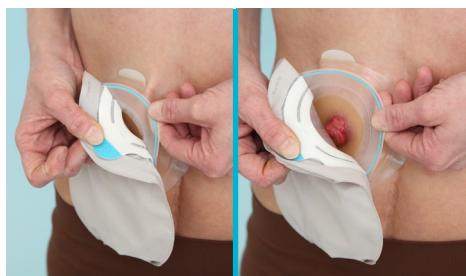Remove pouch by using the turquoise tab located on the adhesive ring and gently pull down and away.

Remove - 2

Use the white tab to gently pull the barrier away from the skin using the "push-pull" technique.


Features

Hide-away

Hide-away: Tuck sealed outlet up and under the protective pocket.

For extra discretion: Adhere the closed outlet on a drainable pouch to the Velcro® dot near inspection window.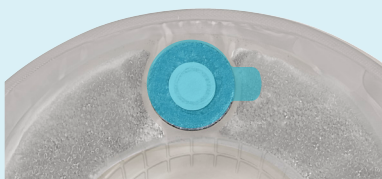

Apply teal sticker to exposed part of the filter, on the back of the pouch, before showering, swimming or submerging in water. Remove sticker when done. Stickers come in the box of pouches.

The Coloplast logo is a registered trademark of Coloplast A/S.
© 2017 Coloplast Corp. All rights reserved.
Velcro® is a registered trademark of Velcro Industries B.V.

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El logotipo de Coloplast es una marca comercial registrada de Coloplast A/S.
© 2017 Coloplast Corp. Todos los derechos reservados.
Velcro® es una marca comercial registrada de Velcro Industries B.V.