

ColActive[®] PLUS Ag

Collagen Matrix Dressing with Silver

✓ FACILITATES NATURAL WOUND HEALING


Collagen and EDTA in ColActive[®] Plus moderate MMP's elevated activity.

✓ OPTIMIZES FLUID MANAGEMENT

Alginate and CMC in ColActive[®] Plus Ag allows the dressing to absorb up to 40 times its weight in wound drainage, while still maintaining an optimal moisture balance for wound closure.

✓ HIGH PATIENT COMPLIANCE AND LOW PAIN

Dressing gels with moisture on contact providing maximum patient comfort by creating a soothing, cooling effect. Bioresorbable eliminates the need for in-growth removal.

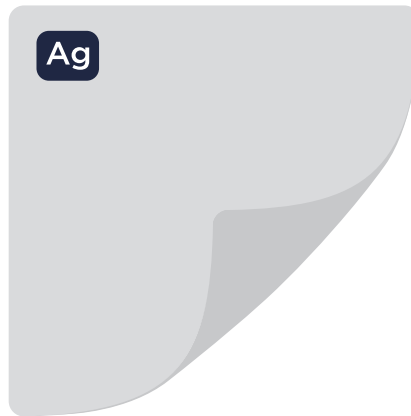

✓ ANTIMICROBIAL SILVER

ColActive[®] Plus Ag contains ionic silver for broad spectrum antimicrobial activity.

✓ CLINICAL TRACK RECORD

Over seven years of clinical success in treating chronic stalled wounds.

HOW DOES IT WORK?

ColActive[®] Plus is an advanced wound dressing made from collagen, sodium alginate, carboxymethylcellulose (CMC), and ethylenediaminetetraacetic acid (EDTA). ColActive[®] Plus is a topically applied wound dressing that will transform into a soft gel sheet when in contact with wound exudates. The dressing maintains a moist wound environment that aids in the formation of granulation tissue and epithelialization. The EDTA in the dressing removes zinc to inhibit the activity of matrix metalloproteinases (MMP), thus creating a suitable environment for wound healing. ColActive[®] Plus dressings may be trimmed and layered for the management of deep wounds.

WHEN TO USE:

ColActive[®] Plus is indicated for management of full and partial thickness wounds including: pressure ulcers, diabetic ulcers, ulcers caused by mixed vascular aetiologies, venous ulcers, donor and graft sites, abrasions, traumatic wounds healing by secondary intention, dehiscent surgical wounds, first and secondary degree burns.

WHEN NOT TO USE:

ColActive[®] Plus should not be used on patients with a known allergy or sensitivity to animal derived collagen, any other ingredient or on third degree burns.




ColActive[®] PLUS Ag
Collagen Matrix Dressing with Silver

1-877-711-6055 | colactive@covalon.com


covalon
TECHNOLOGIES LTD.

HOW TO APPLY:


1. Debride and irrigate the wound bed with an appropriate wound cleanser or distilled water before application.
2. Cut the dressing to fit the exact wound size. For heavily exudating wounds, apply the dressing directly to the wound bed. For dry wounds with very little exudate, moisten the wound bed with distilled water to begin the gelling process.
3. Cover the ColActive[®] Plus with an appropriate secondary dressing.
4. Change dressings daily or as good nursing practice dictates.

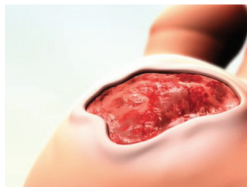

1.

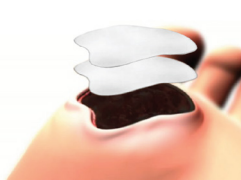

2.

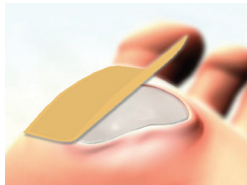

3.



4.


CARE AND REMOVAL:

1. Remove the secondary dressing with care. Gently remove ColActive[®] Plus ensuring that the dressing removal does not damage any newly formed tissues.
2. Use an appropriate wound cleanser or distilled water to cleanse the wound site prior to applying a new ColActive[®] Plus dressing as per application instructions.

NOTE: As with any medical device, always refer to the full instructions for use prior to application.

AVAILABILITY:

Reference Number		Size (in/cm)	Dressings per Carton	Cartons per Case	HCPCS
ColActive[®] Plus ColActive[®] Plus Ag					
TWBC1016	TWBC1033	2in x 2in (5cm x 5cm)	10	10	A6021
TWBC1018	TWBC1034	4in x 4in (10cm x 10cm)	10	10	A6022
TWBC1035	TWBC1032	7in x 7in (18cm x 18cm)	10	10	A6023

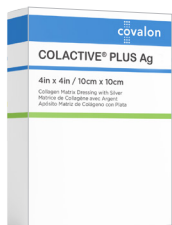

ColActive[®] PLUS Ag
Collagen Matrix Dressing with Silver

1-877-711-6055 | colactive@covalon.com

covalon
TECHNOLOGIES LTD.