

Clearo

by Breas

Clearo rework instructions to set region and language

1

Turn on
Clearo
device



3

Click on
User
Preferences



2

Click on
Menu



4


Click on
Select
Language



Clearo rework instructions to set region and language

5

Choose the American flag

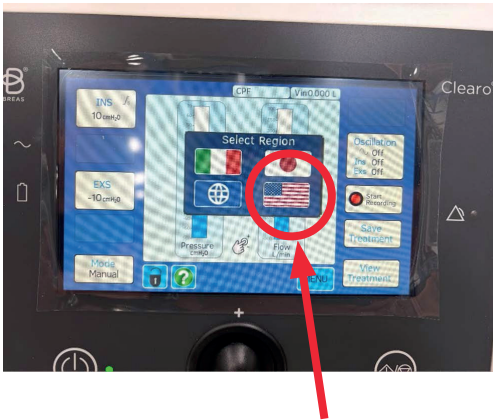


7

Click anywhere on the device to exit. The Language is now set.
Turn off the device.

6

Select language

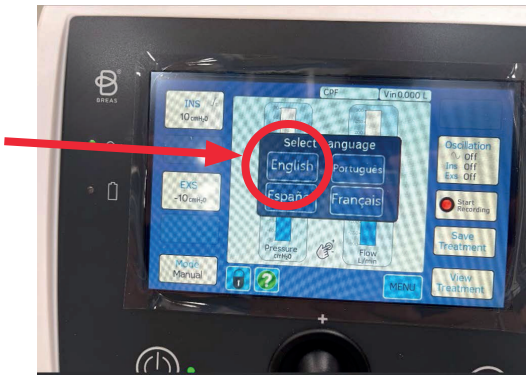

8

Contact Breas' Technical Service at orderus@breas.com with any questions.

Breas, the Breas logo, and Be More are trademarks of Breas Medical AB. Clearo is a trademark of Breas Medical Ltd. © 2026 Breas Medical - All rights reserved. Breas Medical reserves the right to make changes in specifications and features shown herein, or discontinue the product described at any time without notice or obligation. Contact your Breas representative for the most current information
BOS-MAR-MAR-008324 Rev 0



Breas Medical, Inc.
16 Esquire Rd.
N. Billerica, MA 01862
+1 855 436 8724



For more information visit breas.us