

Product Code	Description
26048	1mL Luer Slip
26049	1mL Low Dead Space (LDS) Luer Lock
26050	1mL Luer Lock
26200	2-3mL Luer Lock
26201	2-3mL Luer Slip
26230	5-6mL Luer Lock
26231	5-6mL Luer Slip
26265	10-12mL Luer Lock
26266	10-12mL Luer Slip
26267	10mL Curved Tip Syringe (Non-Sterile)
26280	20-25mL Luer Lock
26281	20-25mL Luer Slip
26290	30-35mL Luer Lock
26291	30-35mL Luer Slip
26292	30-35mL Catheter Tip
26300	50-60mL Luer Lock
26301	50-60mL Luer Slip Eccentric
26304	50-60mL Catheter Tip
26305	50-60mL Luer Slip Eccentric
OSTDS	1mL Delivery Syringe


Instructions For Use (IFU)

Table of Contents

1	Device Description
2	Intended Use
3	Indications for Use/Target Users
4	Benefits of Use
5	Warnings / Precautions
6	Contraindications
7	Sterility
8	Preparation for Use
9	Operating Instructions
10	Storage

1 Device Description

- 1.1 The disposable syringe (without needle) is a single-use, hollow cylindrical tube with a tip that can receive or expel fluid.
- 1.2 The tip of the syringe provides:
 - 1.2.1 The point of attachment of a needle;
 - 1.2.2 connection to another device to administer and/or withdraw (aspirate) fluids/drugs;
 - 1.2.3 it may also be used for fluid/drug preparation;
 - 1.2.4 connection to a non-invasive device;
 - 1.2.5 for irrigation.
- 1.3 The volume of solution inside a syringe is indicated by graduation lines on the barrel. Graduation lines may be in milliliters or fractions

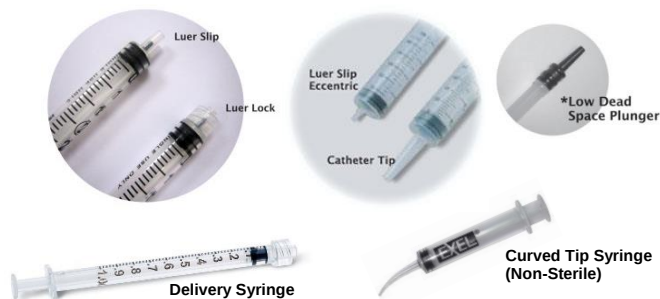

of a milliliter, depending on the capacity of the syringe.

- 1.4 Force applied on the plunger and piston moves the piston down the barrel reducing the volume of the barrel below the seal expelling fluid. tubular

2 Intended Use

- 2.1 The disposable syringe is intended to be used (depending on the syringe tip)
 - 2.1.1 in conjunction with hypodermic needles, and infusion/transfusion sets to inject fluids into, or withdraw fluids from, the body.
 - 2.1.2 connection to another device to administer and/or withdraw (aspirate) fluids/drugs;
 - 2.1.3 it may also be used for fluid/drug preparation;
 - 2.1.4 in connection to a non-invasive device;
 - 2.1.5 for irrigation.

3 Indications for Use/Target Users

- 3.1 Disposable Syringes are equipped with a Luer Slip or Luer Lock connector that can be used in conjunction with hypodermic needles and infusion/transfusion sets.
- 3.2 The device is designed for use by a qualified physician or clinician.
- 3.3 There is no particular intended patient population for the use of disposable syringes.

4 Benefits of Use

- 4.1 Safe, easy to use.
- 4.2 Rapid delivery of fluids.
- 4.3 Prevent cross-contamination between patients.
- 4.4 Can be used with or without needle.

5 Warnings/Precautions

- 5.1 Use once and discard immediately in accordance with local safety standards.
- 5.2 For single use only.
- 5.3 Not for Drug Storage.
- 5.4 Do not reuse. Reuse of syringe with needle can result in contamination or transmit disease.

6 Contraindications

- 6.1 There are no recognized contraindications for the use of Disposable Syringes.

7 Sterility

- 7.1 Disposable Syringes are sterilized by EO gas sterilization (indicated by **STERILE**) single use only. Do not re-use, (2) Do not re-sterilize. Inspect each package prior to use. Do not use the device

if any seal or cavity is damaged  or breached or if the expiration date has been exceeded. Once opened, the device must be used or discarded.

7.2 The 10-mL Curved Tip Syringe (Item 26267):

7.2.1 is offered in a bulk, non-sterile configuration.

7.2.2 Sterile barrier and endotoxin limits are not applicable.

8 Operating Instructions

8.1 Separate at upper package edge and peel apart to point below the hub.

8.2 If using with a needle, affix and tighten the syringe or female luer to the needle using aseptic technique. Grip the base of the needle and turn the syringe or luer clockwise.

8.3 Pull back on the plunger to fill the syringe with fluid according to established technique.

8.4 Push down on the plunger to express the fluid according to the procedure.

8.5 If used with a needle, remove the needle cap and perform the procedure according to established technique.

8.6 After the procedure, discard of the syringe (and needle) in a biohazard container according to local safety standards.

9 Storage

9.1 Store at room temperature 15°C-25°C.

9.2 Do not store in high temperatures and humidity.

9.3 Keep away from direct sunlight.

9.4 Keep away from rain.

9.5 Store in a cool and dry place.

LEGAL NOTICE

© 2023 Exelint International, Co. All rights reserved. and the EXEL logo are trademarks of EXELINT International, Co. EXELINT International, Co. disclaims any responsibility for possible consequences resulting from reuse, resterilization or improper use.

	EXELINT International, Co. 2500 Santa Fe Avenue, Redondo Beach, U.S.A. Tel: (310) 649-0707 Fax: (310) 649-1178 e-mail: info@med.com FOR U.S.A. INFO CALL: 800-940-3935 www.Exelint.com e-mail: info@Exelint.com
	EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem The Netherlands

Graphic Symbol Legend for Medical Device Labeling

	Medical Device
	Catalog number
	Federal law (USA) restricts this device to sale by or on the order of a physician.
	Consult instructions for use
	Important Cautionary Information
	Sterilized by Ethylene Oxide. Sterility guaranteed if package unopened and undamaged. (except 26267)
	Manufacturer's Lot Number
	Unique Device Identifier
	Use by
	For single-patient-use only. Do not reuse.
	Do not use if package is damaged
	Keep away from sunlight
	Keep Dry
	Temperature Limit
	Non-Pyrogenic (except 26267)
	Single Sterile Barrier (except 26267)
	This device is not manufactured with Natural Rubber Latex
	Manufacturer
	Product marketed in European countries
	European Representative