

Tubifast®

2-WAY STRETCH™



Lightweight, tubular dressing retention

Tubifast® is a lightweight, tubular bandage, ideal for dressing retention and skin covering for any part of the body.

Tubifast® bandages hold dressings securely, without constriction or compression

The light elasticity promotes freedom of movement

Tubular presentation — no requirement for pins or tape

Color coding for quick reference

Wide range of sizes to fit everything from small limbs to large adult trunks

Quick and easy to use

Light, radial and longitudinal elasticity allows patients complete freedom of movement, with added comfort¹


Less fray reduces the risk of fibers in the wound²




[Click here to view detailed product descriptions and prices.](#)

Call 800-397-5899 to order today!


How to use Tubifast®

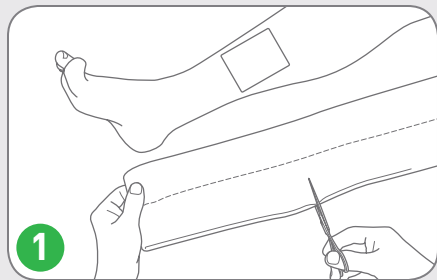

1 Tubifast® 2-Way Stretch is a lightweight conforming stockinette which can be used for dressing retention in dermatological areas or as an undercast stockinette. To apply Tubifast® 2-Way Stretch, choose the correct bandage width using the quick reference color coding (see below) and cut to length.

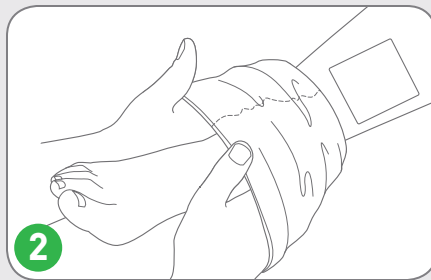

2 Stretch the piece of Tubifast® 2-Way Stretch over the hands and over the affected area, if securing a dressing. If full limb coverage is required, the Tubifast® 2-Way Stretch is simply pulled or rolled on as a stocking.

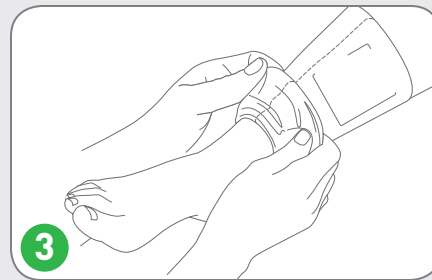

3 The Tubifast® 2-Way Stretch covers the dressing and gives it a light, even, non-constricting pressure. It may be applied in a double layer, especially over joints for extra security, or the edges may be tucked under. Whichever method is used, Tubifast® 2-Way Stretch is quick, easy and comfortable and stays in place.

How Tubifast® 2-Way Stretch works

Tubifast® 2-Way Stretch is manufactured from viscose with very fine elastane threads knitted into the fabric radially and longitudinally to provide light elasticity.

The Tubifast® bandage holds dressings securely, without constriction or compression. No pins or tapes are necessary, and its light elasticity promotes freedom of movement for the patient.

Tubifast® bandages are available in five sizes, designed for use on any part of the body.

Each size is identified by a colored line knitted into the bandage. It's quick and easy to use: cut to size and stretch over the dressing for an even, non-constrictive fit.

Benefits of Tubifast®

- Tubifast® bandages hold dressings securely, without constriction or compression
- The light elasticity promotes freedom of movement
- Tubular presentation—no requirement for pins or tape
- Color coding for quick reference
- Wide range of sizes to fit everything from small limbs to large adult trunks
- Quick and easy to use
- Light, radial and longitudinal elasticity allows patients complete freedom of movement, with added comfort¹
- Less fray reduces the risk of fibers in the wound²

Washing Instructions: 

¹ Published data shows that Tubifast® 2-Way Stretch exceeds the minimum requirement of longitudinal stretch for the actions of sitting, bending and flexing. Physical testing of textiles, B. P. Saville, Woodhead Publishing, 1999.

² Data on file.

Areas of use

Tubifast® bandages can be used for dressing retention and skin covering for any part of the body. They can also be used for patch wrapping and as an undercast stockinette.

Because of the tubular construction, Tubifast® bandages are particularly suitable for holding dressings in place on difficult areas of the body.

Use as secondary dressing over other Mölnlycke dressings such as Mepilex®, Mepilex® Lite, Mepitel®, Mepitel® One, Mepilex® Ag, Mepilex® Transfer, and Mepilex® Heel.

Material properties: Viscose, Elastane and Polyamide
Does not contain latex.

Tubifast® 2-way Stretch Ordering Information†

Product Code	Line	Product	Limb size	Width*
2434	RED	Small Limbs	3.5" - 7.1" (9 - 18 cm)	1.5" (3.5 cm)
2436	GREEN	Small and medium limbs	5.5" - 9.5" (14 - 24 cm)	2" (5 cm)
2438	BLUE	Large limbs	9.5" - 15.8" (24 - 40 cm)	3" (7.5 cm)
2440	YELLOW	Extra-large limbs, heads, children's trunks	13.8" - 25.2" (35 - 64 cm)	4.5" (10.75 cm)
2444	PURPLE	Large adult trunks	25.2" - 51.2" (64 - 130 cm)	8" (20 cm)

† Packaged in single packs

* Length for all Tubifast® 2-Way Stretch is 10 m.