



2701 Bartram Road • Bristol, PA 19007  
800-840-9041 • 215-788-2700  
www.gentell.com • info@gentell.com

## LoProfile Bordered Foam Dressings

<b>4" x 4"</b> (2"x2" pad with 1" adhesive border) 10/box 5 boxes/case 50 dressings/case <b>GEN-14400</b>	<b>6" x 6"</b> (4"x4" pad with 1" adhesive border) 10/box 5 boxes/case 50 dressings/case <b>GEN-14600</b>
--	--

- Absorbs twenty times its weight in exudates and drainage with no strike through
- Water-resistant barrier provides incontinent protection
- Excellent pressure relief and protection for skin that has recently healed



**Gentell LoProfile Bordered Foam** dressings are waterproof and highly absorbent foam with an island design. Their dense, sturdy structure protects the wound from external threats, and the porous texture is highly absorbent. A non-abrasive border holds the foam in place and creates a water-resistant barrier to protect the wound.

### Directions

1. Gentell Lo Profile Foam Plus may be used as a primary or secondary dressing. It may also be used as protection for wounds that have recently healed.
2. Clean the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. Select the optimum size Gentell Lo Profile Bordered Foam dressing providing a minimum of one-inch margin around the edges of the wound.
4. Remove paper backing from the dressing and apply directly over the surface of the wound.
5. Change dressing every other day or as ordered by a physician.