

# Male Urinary Pouch

(Retracted Penis Pouch)

## Flextend Skin Barrier

- **Extended wear time**  
Median wear time of nearly 40 hours in multi-site clinical study.\*
- **Enhanced skin protection**  
Resists breakdown; helps prevent skin irritation and maceration.

## Latex-Free Odour-Barrier Film

- **Contains odour effectively**  
Provides comfort for the patient and caregiver.
- **Prevents allergic reaction**  
For latex-sensitive patients and caregivers.
- **Improves patient monitoring**  
Transparent film allows easy assessment of discharge.

Male Urinary Pouches can be ordered either directly or through NHS supplies catalogue:

Available on Drug Tariff

Male Urinary Pouch				
Stock No.	NHS Product Code	Skin Barrier	Box Quantity	NHS Catalogue
9873	GCV100	Flextend	10	£30.35

\*Data on file. Hollister study 1946-IC, 1997

## Detailed Instruction Guide

### Preparing the skin

The patient should be in the supine position. Wash the genital area with mild soap and water. Trim the pubic hair. Rinse and dry thoroughly. Do not use oils, lotion, or powders as they will interfere with the adhesion of the barrier.

Be sure the drain cap is placed on the pouch drain. If necessary, cut an opening in the skin barrier slightly larger than the penis. **Do not** cut beyond the line indicated on the backing paper.




Remove the protective release paper from the skin barrier.



Place the penis in the opening of the pouch and press the barrier securely on skin. Avoid wrinkling, which can create channels for leakage.

Gently hold the barrier material against the skin for 60 seconds so that the patient's body heat will soften the barrier material and enhance its adhesive properties.

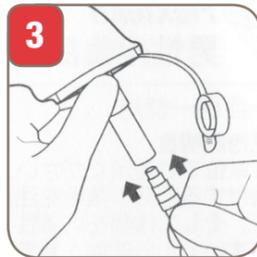

Remove the drain cap at the bottom of the pouch and attach the tubing to a drainage bag.

### Removing the Pouch

Disconnect the pouch from the drainage bag. Close the pouch drain cap. Ease the skin barrier away from the skin. To ensure proper adhesion, do not reuse. Dispose of used pouch according to protocol. **Do not discard into toilet.**

**Caution:** Discontinue use of this product if any of the following signs or symptoms occur.

- Changes in the skin colour of the penis
- Extreme skin temperature changes
- Swelling, burning, itching, pain, or loss of sensation

# Male Urinary Pouch

(Retracted Penis Pouch)

## Patient comfort and a secure fit in one easy to apply pouch



Your opportunity to see how simple and easy it is to fit.