

Re-Order #	Description	Size	Wt.	U/M
914380	Blue	2" x 60"	250 lbs.	Each
914382	Blue Bariatric	2" x 72"	350 lbs.	Each
914384	Stars/Stripes	2" x 60"	250 lbs.	Each
914386	Stars/Stripes Bariatric	2" x 72"	350 lbs.	Each
914387	Orange	2" x 60"	250 lbs.	Each
914388	Orange Bariatric	2" x 72"	350 lbs.	Each



Purpose

The Gait Belt is intended for use when lifting, ambulating or transferring patients. It is not to be used as a safety restraint in a chair or bed. Since the Pathoshield® Gait Belts minimize the risk of harboring pathogens, they are to be used in lieu of fabric gait belts when a reduced pathogen environment is desired.

Easy to Use

When an individual is at risk in ambulating without assistance gait belts should be used. The gait belt should be placed around the waist and when the buckle is closed the gait belt should fit snugly but not too tight to introduce patient discomfort.

The Delrin side release buckles are secured by inserting the male end of the buckle into the female side. To release the buckle, simultaneously press the tabs at the top and bottom of the buckle. Before ambulating the patient make certain that the buckle is positioned out of reach of the patient.

The patient should be ambulated by having the caregiver placing their hands through the belt palms up and grasping the belt. Before ambulating the resident, it should be confirmed that the number of caregiver's and their ability is sufficient for ambulation without introducing risk to either the caregiver or patient. The belt should be inspected before each use to assure that it isn't defective.

Easy Care

These devices may be wiped clean with a mild disinfectant.



[Click to see prices and more product details.](#)

Call to order now at 800-397-5899