



Re-Order Number	Descriptions	Size	Units
554025	30° Bed Wedge w/LSII Cover	7" x 12" x 24"	Pair
554026	30° Bed Wedge w/LSII Cover	7" x 12" x 34"	Pair

Purpose

Skil-Care's 30-Degree Positioning Wedge is designed to comply with the Agency for Healthcare Research and Quality (AHRQ) guidelines that recommend a 30-degree angle of incline as optimal for relieving pressure on the hipbone and coccyx.

Easy to Use

1. Place patient on his/side and slip tapered edge of wedge beneath patient, just below the shoulders. (Non-slip matting faces bed. It does not come in contact with patient).
2. Allow patient to roll back onto the wedge.
3. Reposition patient according to your facility's established procedures and policies.

Easy Care

The 30-Degree Positioning Wedge may be wiped clean using a spray-and-wipe cleanser. Disinfect using a non-chlorine disinfectant.

[Click here to view detailed product descriptions and prices.](#)

Call 800-397-5899 to order today!