


LUXE Bidet[®]


Assembly Instruction

NAMES AND FUNCTIONS OF PARTS

*Please note: illustrations may vary depending on the model

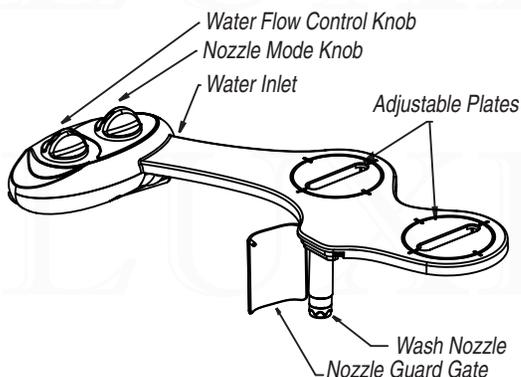


Installation Tools You May Need

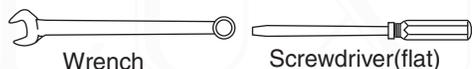

PARTS


Flexible Metal Braided Hose

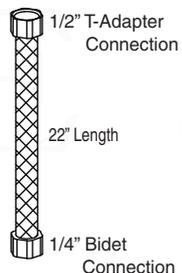


Plastic T-Adapter




Flexible Water Supply Hose

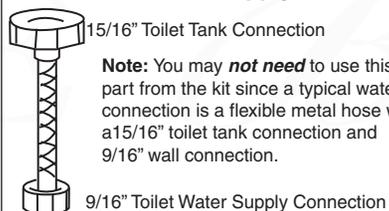

1 Shut off the toilet water supply.
Flush toilet to empty the tank.

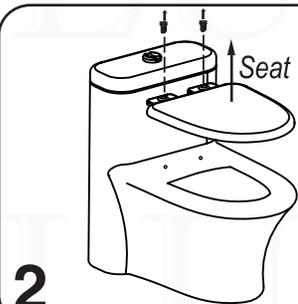

Remove the seat by unscrewing the seat.

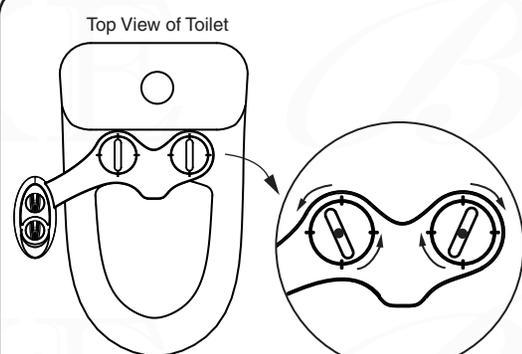


Place the bidet on the toilet. Plates may be adjusted depending on where the two holes in the back of the toilet are.



4 Bidet should now be aligned with the coupling holes.



5 Screw the seat back on to the bidet and the toilet.

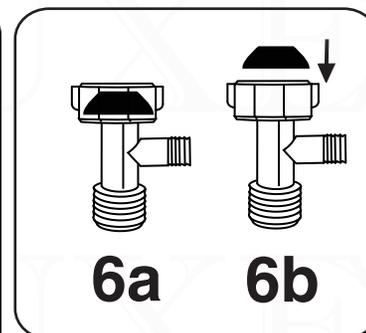


6a: Make sure that there is a black rubber washer inserted at the top of the t-adapter.
6b: If it is not already there, insert it as shown in figure 6b.


7 Connecting Water Supply

7a: Make sure you have shut off the toilet water supply and flushed the toilet to empty the tank.

7b: Connect the plastic T-adapter to the water supply as shown in figure 7b.

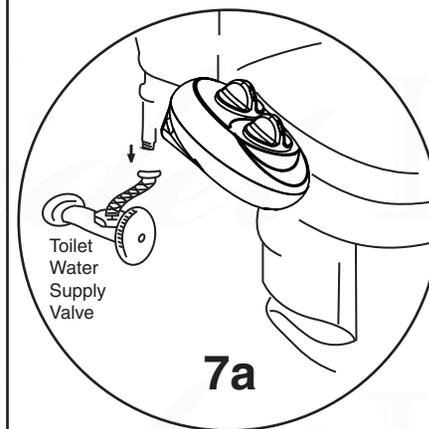


7a

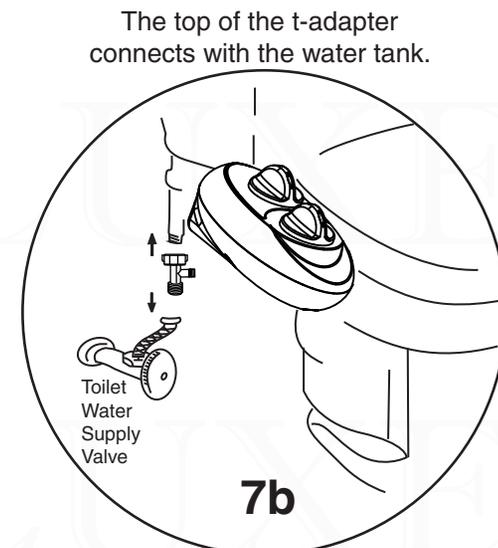


7b

Tip: Do not use excessive force while tightening the connection. Hand tighten then use a wrench to gently tighten one quarter or half-turn more.

LUXE Bidet® Assembly Instruction

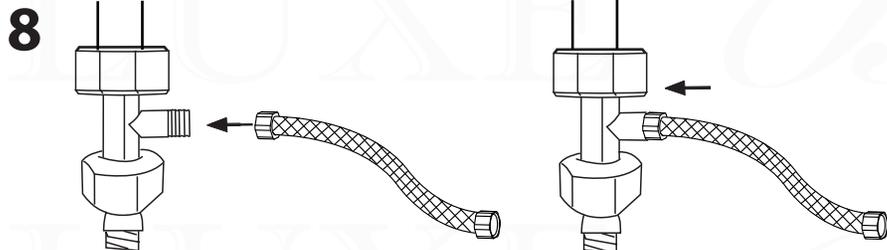


Connect the metal braided hose to the t-adaptor.

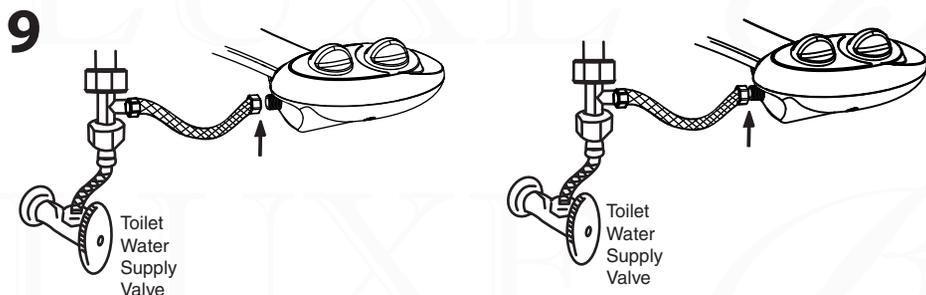

Connect the metal braided hose to the bidet inlet

ASSEMBLED UNIT

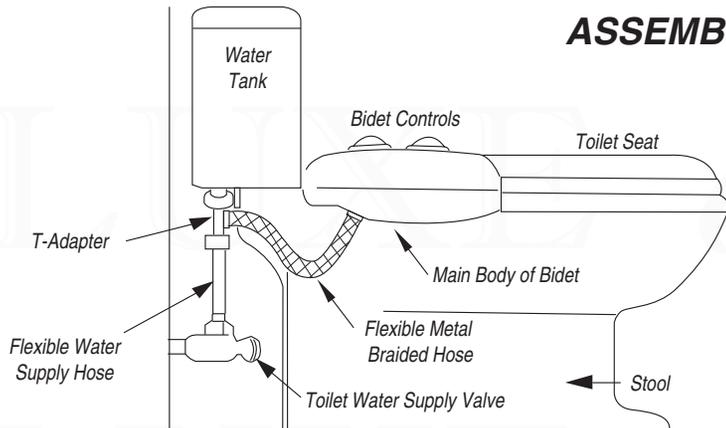

TRYING YOUR BIDET FOR THE FIRST TIME

Congratulations! You are ready to use the LuxeBidet for the first time. Gently turn the water pressure control knob to reach your desired spray pressure. The control only regulates water pressure, not the duration of the spray.

The working pressure of this product is 50±20 psi

LUXE Bidet® NEO110™, NEO120™, & NEO180™

TROUBLESHOOTING

For additional troubleshooting tips, please see our FAQs page at www.LuxeBidet.com.

After installing the bidet, the toilet seat sits at an angle and no longer rests on the toilet bowl.
Apply adhesive rubber bumpers to the bottom of the seat to eliminate the gap. Bumpers are available for purchase when you go to www.LuxeBidet.com/accessories.

When the bidet is on, no water comes out of the spray nozzle.

Solution 1: Gently pump the nozzle up and down by hand. It may simply be stuck.
Solution 2: Check that the t-adaptor has been drilled all the way through so water can flow through it. If the t-adaptor is blocked, please contact us for a replacement.

Water is leaking from a connection


Solution 1: Try reconnecting the t-adaptor to ensure that all threads are catching properly and that the connection is properly tightened.
Solution 2: If you have difficulty properly tightening the t-adaptor to your toilet tank connection, try wrapping a thin layer of plumbing tape over the toilet tank threads before connecting the t-adaptor. The plumbing tape will allow for added grip.
Solution 3: If you continue to experience a drip from the t-adaptor or any other connection, please contact us.

**For other questions or concerns, please contact us at
www.LuxeBidet.com**

OPTIONAL ACCESSORY

Available for Purchase at www.LuxeBidet.com/accessories

Metal T-Adapter with Shut-Off Valve (Cold Water)

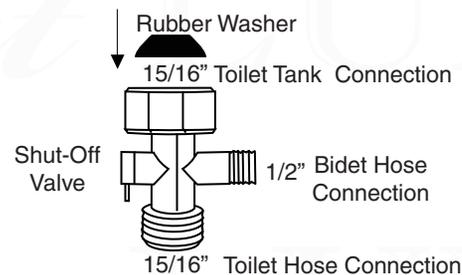

This metal t-adaptor has a shut-off valve to limit or cut off cold water to the bidet. It can be used instead of the plastic t-adaptor for the cold water connection.

For more information, visit www.LuxeBidet.com.

©2Go Products, LLC