



Instructions For Use

1



Select the appropriate Magellan™ Safety Needle for your procedure.

2



Open package by peeling back paper tab and remove safety needle.

3



Secure the needle by holding the needle and twisting the syringe clockwise to fully seat the needle. Remove protective needle sheath.

4



Draw up medication and administer injection using an aseptic technique in accordance with your facility's protocol.

Lock the safety shield using any of the following activation methods

5



...thumb

or



...finger

or



...flat surface

6



Verify locked position through audible and tactile "click". Locked position will completely cover needle tip.

7



Once safety shield is locked, immediately dispose of needle and syringe in an approved sharps container.

Safety Needle Gauge Colors

18
Pink

19
Cream

20
Yellow

21
Green

22
Black

23
Blue

25
Orange

All Magellan™ Safety Needles are ISO color-coded.

COVIDIEN, COVIDIEN with logo and the Covidien logo are U.S. and/or internationally registered trademarks of Covidien AG.
© 2010 Covidien AG.
H6077 xxxM 09/10