

Med-E-Lert™ XL

www.medelert.com

① TO SET UP YOUR ALARM

▲ (1) Tap button 1 - the hour should start blinking.
▲ (2) Tap button 3 repeatedly until the correct hour is selected.
▲ (3) To go back an hour, tap button 2.

▲ (4) Once the hour is set, tap button 1 and the minutes will blink.
▲ (5) Repeat step 2 until the correct time is reached.
▲ (6) Repeat step 3, if needed, until the correct time is reached.

Note: Once alarms and time is set and you're at the main menu, the colon between hours and minutes will continue to blink.

⑩ RECORD VOICE

When setting up each alarm time, press and hold the button 1 until the number "0" displays on the screen. (Note: Your finger must remain pressing button 1 for the duration of your recording; once you remove your finger, the recording will stop and automatically enter playback mode.) You can record up to 10 seconds. If you have not removed your finger for the entire 10 seconds, it will automatically cutoff and enter playback mode. If you would like to rerecord your audio, simply repeat the above steps on each alarm.

⑪ TONES

To select an alarm tone, press and hold buttons 1 and 3 simultaneously. You will need to repeat this process each time to move on to the next tone. Quickly press button 1 once you've reached the desired alarm tone to return to the main screen.

⑫ HOW TO LOCK AND UNLOCK MED-E-LERT

Insert key and turn it to the left 1/4 turn to lock.
Insert key and turn it to the right 1/4 turn to unlock.

⑬ HOW TO STOP ALARM

To stop an alarm and dispense the pills, pick up the Med-E-Lert™ XL and tilt it over, dumping the pills into a hand or suitable container.

⑭ LOW BATTERY ALERT & STANDBY WAKE UP

When the batteries are lower than 4.0V, the low battery icon will appear on the LCD screen and both the battery icon and LED will begin blinking. When the blinking LED and the low battery icon are visible, it's time to change all 4 AA batteries with 4 identical new ones or plug in by DC power supply. The system will cease to work until all batteries are replaced or the unit is plugged in. (If there is no operation for one minute, the Med-E-Lert™ XL will go into stand-by mode and the display will go blank to save battery life.)

⑮ ATTENTION

(1) If there is not any operation on the display after an additional 5 minutes and it is not time for a programmed alarm, the system will go to "sleep" and shut down the display. When it is time for a programmed alarm, the system will wake up, alerting you through both visual and audible alarms.
(2) If the pill dispenser is left upside-down for 1 minute, there will be a chirping sound coming from the Med-E-Lert™ XL. The chirping sound will continue until the Med-E-Lert™ is up-righted. If it is time for an alarm and the Med-E-Lert™ XL remains upside-down, the full alarm will sound and the blinking red light will also be displayed. After the Med-E-Lert™ XL is up-righted to the correct position, the pill tray will rotate granting access to the medications. Once the dispensing slot is emptied, the scheduled alarm will stop.
(3) DISCLAIMER
The Med-E-Lert™ XL Automatic Pill Dispenser is not a medical device and is not for use in the diagnosis of disease or other conditions, or in the cure, mitigation, treatment or prevention of disease.

www.medelert.com

⑩ RECORD VOICE

When setting up each alarm time, press and hold the button 1 until the number "0" displays on the screen. (Note: Your finger must remain pressing button 1 for the duration of your recording; once you remove your finger, the recording will stop and automatically enter playback mode.) You can record up to 10 seconds. If you have not removed your finger for the entire 10 seconds, it will automatically cutoff and enter playback mode. If you would like to rerecord your audio, simply repeat the above steps on each alarm.

⑪ TONES

To select an alarm tone, press and hold buttons 1 and 3 simultaneously. You will need to repeat this process each time to move on to the next tone. Quickly press button 1 once you've reached the desired alarm tone to return to the main screen.

⑫ HOW TO LOCK AND UNLOCK MED-E-LERT

Insert key and turn it to the left 1/4 turn to lock.
Insert key and turn it to the right 1/4 turn to unlock.

⑬ HOW TO STOP ALARM

To stop an alarm and dispense the pills, pick up the Med-E-Lert™ XL and tilt it over, dumping the pills into a hand or suitable container.

⑭ LOW BATTERY ALERT & STANDBY WAKE UP

When the batteries are lower than 4.0V, the low battery icon will appear on the LCD screen and both the battery icon and LED will begin blinking. When the blinking LED and the low battery icon are visible, it's time to change all 4 AA batteries with 4 identical new ones or plug in by DC power supply. The system will cease to work until all batteries are replaced or the unit is plugged in. (If there is no operation for one minute, the Med-E-Lert™ XL will go into stand-by mode and the display will go blank to save battery life.)

⑮ ATTENTION

(1) If there is not any operation on the display after an additional 5 minutes and it is not time for a programmed alarm, the system will go to "sleep" and shut down the display. When it is time for a programmed alarm, the system will wake up, alerting you through both visual and audible alarms.
(2) If the pill dispenser is left upside-down for 1 minute, there will be a chirping sound coming from the Med-E-Lert™ XL. The chirping sound will continue until the Med-E-Lert™ is up-righted. If it is time for an alarm and the Med-E-Lert™ XL remains upside-down, the full alarm will sound and the blinking red light will also be displayed. After the Med-E-Lert™ XL is up-righted to the correct position, the pill tray will rotate granting access to the medications. Once the dispensing slot is emptied, the scheduled alarm will stop.
(3) DISCLAIMER
The Med-E-Lert™ XL Automatic Pill Dispenser is not a medical device and is not for use in the diagnosis of disease or other conditions, or in the cure, mitigation, treatment or prevention of disease.

Med-E-Dispenser
Med-E-Lert™ XL
9265 S Highland Dr #900250
Sandy, UT 84093

EASY SET-UP
MANUAL

AUTOMATIC PILL DISPENSER

Med-E-Lert™ XL

www.medelert.com

① INSERT THE BATTERIES & PLUG IN

Insert batteries into the Med-E-Lert™ XL & plug in power supply. The compartment is located on the bottom of the machine.

② HOW TO OPEN

Open the compartment by pressing the lid's tab down on the front of the machine.

③ CLEANING INSTRUCTIONS

Wipe down the medication tray with a damp cloth.

④ LOADING YOUR MEDICATION

You will notice that there are nine dosage rings from which to select. Choose the ring that has the correct amount of doses per day for your medications. These doses can range up to nine-times per day. Place the appropriate ring for the medication regimen on the tray; fold tabs down to it secure it in the pill compartments, prior to filling it, and use this as a guide as to where to place pills.

Once the ring is in place, simply fill the slots with your medication. Note: It is normal, when your doses are less than 4 times a day, to have more than one day's worth of medication in each slot.

The refill frequency will depend upon the number of daily medications for example:

- > One time a day medication refills every 28 days
- > Two times a day medication refills every 14 days
- > Three times a day medication refills every 9 days
- > Four times a day medication refills every 7 days
- > Five times a day medication refills every 5 days
- > Six times a day medication refills every 4 days
- > Seven times a day medication refills every 4 days
- > Eight times a day medication refills every 3 days
- > Nine times a day medication refills every 3 days

⑤ DISPLAY LAYOUT

The below figure describes the meanings of symbols on the LCD screen:

Photo showing the LCD with all icons showing and lines drawn to each and the name printed at opposite side of the line where each icon is found.

Battery Icon	Mute & 3 Alarm Tones	12/24-Hour Time
12 Alarms	Clock: Hours	Clock: Minutes
Voice Alarms	Button 1	Button 2
	Button 3	

⑥ 12/24-HOUR TIME FORMATS

The Med-E-Lert™ XL automatically defaults to a 24-hour format. The AM or PM symbol will appear on the top of the display when the clock is in the 12-hour format. Choose from a 12- or 24-hour clock by pressing buttons 2 and 3.

▲ Press and hold buttons 2 and 3 simultaneously.
24-hour format 12-hour format

⑦ SLEEP MODE & WAKE UP DISPENSER

Press button 2 to put the Med-E-Lert™ XL into "sleep mode." After the Med-E-Lert™ XL goes into "sleep mode," it will shut down the screen to extend its battery life.

⑧ SETTING UP THE CLOCK TIME

Hold down button 1 until the hour starts blinking.
To scroll down the numbers, press button 2. A quick press of button 2 will go back 1 hour.
Hold down button 2 to scroll through more quickly.

Once the desired hour is reached, quickly press button 1 causing minutes to begin blinking. Click on button 2 to scroll down and button 3 to scroll up through the numbers until the correct minute is reached. Press button 1 for 5 seconds to complete setting the clock.

www.medelert.com

⑩ RECORD VOICE

When setting up each alarm time, press and hold the button 1 until the number "0" displays on the screen. (Note: Your finger must remain pressing button 1 for the duration of your recording; once you remove your finger, the recording will stop and automatically enter playback mode.) You can record up to 10 seconds. If you have not removed your finger for the entire 10 seconds, it will automatically cutoff and enter playback mode. If you would like to rerecord your audio, simply repeat the above steps on each alarm.

⑪ TONES

To select an alarm tone, press and hold buttons 1 and 3 simultaneously. You will need to repeat this process each time to move on to the next tone. Quickly press button 1 once you've reached the desired alarm tone to return to the main screen.

⑫ HOW TO LOCK AND UNLOCK MED-E-LERT

Insert key and turn it to the left 1/4 turn to lock.
Insert key and turn it to the right 1/4 turn to unlock.

⑬ HOW TO STOP ALARM

To stop an alarm and dispense the pills, pick up the Med-E-Lert™ XL and tilt it over, dumping the pills into a hand or suitable container.

⑭ LOW BATTERY ALERT & STANDBY WAKE UP

When the batteries are lower than 4.0V, the low battery icon will appear on the LCD screen and both the battery icon and LED will begin blinking. When the blinking LED and the low battery icon are visible, it's time to change all 4 AA batteries with 4 identical new ones or plug in by DC power supply. The system will cease to work until all batteries are replaced or the unit is plugged in. (If there is no operation for one minute, the Med-E-Lert™ XL will go into stand-by mode and the display will go blank to save battery life.)

⑮ ATTENTION

(1) If there is not any operation on the display after an additional 5 minutes and it is not time for a programmed alarm, the system will go to "sleep" and shut down the display. When it is time for a programmed alarm, the system will wake up, alerting you through both visual and audible alarms.
(2) If the pill dispenser is left upside-down for 1 minute, there will be a chirping sound coming from the Med-E-Lert™ XL. The chirping sound will continue until the Med-E-Lert™ is up-righted. If it is time for an alarm and the Med-E-Lert™ XL remains upside-down, the full alarm will sound and the blinking red light will also be displayed. After the Med-E-Lert™ XL is up-righted to the correct position, the pill tray will rotate granting access to the medications. Once the dispensing slot is emptied, the scheduled alarm will stop.
(3) DISCLAIMER
The Med-E-Lert™ XL Automatic Pill Dispenser is not a medical device and is not for use in the diagnosis of disease or other conditions, or in the cure, mitigation, treatment or prevention of disease.

Med-E-Dispenser
Med-E-Lert™ XL