

Product Category: Specialty Absorptive Dressing

Description:

Multipad is a unitized, multi-layered, non-adherent, highly absorbent dressing.

Action:

Multipad is a non-adherent, highly absorbent pad that can be used as a primary or secondary dressing for chronic wounds, surgical incisions, superficial lacerations and abrasions.

Indications:

- All dermal ulcers
- Diabetic ulcers
- Abdominal wounds
- Infected wounds
- Superficial wound
- Lacerations, cuts
- Abrasions
- Donor sites
- 2nd degree burns

Directions:

1. Peel open package.
2. Position **Multipad** over wound.
3. Secure with tape, roll gauze, or Stretch Net™ Tubular Elastic Bandage.
4. To remove, gently lift dressing from wound. Clean and re-dress wound.
5. Dispose of spoiled dressing per institution's policy.