**the ROHO group**

shape fitting technology®

## **Repair Kit Instructions**

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Reparationsssats bruksanvisning

Bruksanvisning for lappesett

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[www.therohogroup.com](http://www.therohogroup.com)

© 2000, 2008, 2012, ROHO, Inc.

## Repair Kit contents:

- (4) 7/8" (2 cm) square self-adhesive patches
- (2) 1 3/8" (3.5 cm) square self-adhesive patches
- (4) isopropyl alcohol pads

## You will also need:

A sink or basin of water, cloth, permanent ink-pen or marker, and a spoon (or other hard, curved surface).

## Locate the leak:

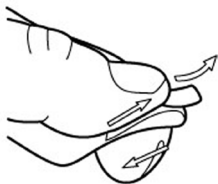
1. For QUADTRO SELECT®, CONTOUR SELECT®, AIRHAWK™ Pro Cushions: Open the ISOFLO Memory Control®.
2. Inflate the product and then turn all inflation valves clockwise to CLOSE. Immerse the product in water. To locate the leak, look for rising air bubbles. Remove the product from the water.

## Prepare the area of the leak:

3. Use a cloth to dry the area of the leak. Mark the area around the leak with a permanent marker or ink pen. **Note:** The ink will not weaken the patch.
4. Use an isopropyl alcohol pad to wipe the area of the leak. Use a clean, dry cloth to remove any excess alcohol. **Important:** Remove all water, oils, detergents, and soaps from the area around the leak. Air dry.
5. Turn all inflation valves counterclockwise to deflate the product. Remove enough air to easily apply the patch.

## Apply the patch:

6. Hold one edge of the patch between your thumb and forefinger. Starting at the opposite edge of the patch, use the other hand to pull up part of the paper backing.

**Note:** Touching the sticky surface of the patch will reduce its adhesive qualities.

7. Center the patch above the leak and apply the exposed edge of the patch to the product. Gradually remove the remaining paper backing as you press the patch to the area of the leak. **Suggestion:** Place the product on a surface that allows you to flatten the area of the leak when you apply the patch.

## Seal the patch:

8. Starting at the center and working toward the edges, use a hard, curved object, such as the back of a spoon, and firmly press down on all areas of the patch. Remove all trapped air until the patch appears uniformly dark.
9. After the patch is sealed, wait at least five (5) minutes before you inflate and use the product.

## Contact Information:

### In the U.S.A.:

ROHO Customer Service and Warranty Service:

Phone: 1-800-851-3449 • Fax: 1-888-551-3449

Customer Service: [cs@therohogroup.com](mailto:cs@therohogroup.com)

Warranty Service: [warranty@therohogroup.com](mailto:warranty@therohogroup.com)

### Outside the U.S.A.:

ROHO International:

Phone: 1-618-277-9150 • Fax 1-618-277-6518

Email: [intl@therohogroup.com](mailto:intl@therohogroup.com)

ROHO, Inc. has a policy of continual product improvement and reserves the right to amend this document. A current version of this document is available at [www.therohogroup.com](http://www.therohogroup.com).

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ROHO, Inc., 100 N. Florida Ave., Belleville, IL 62221-5249