



1

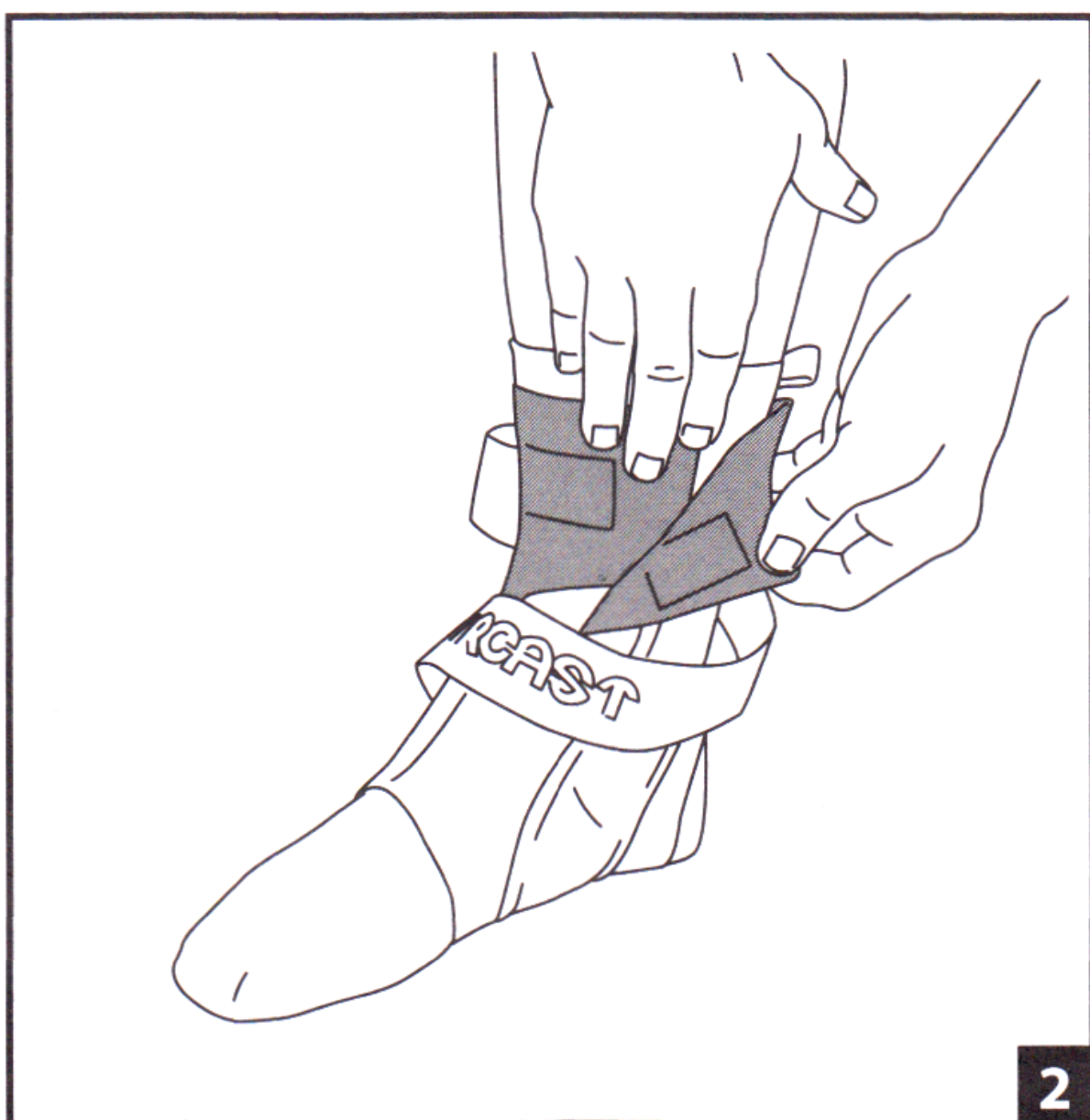

2

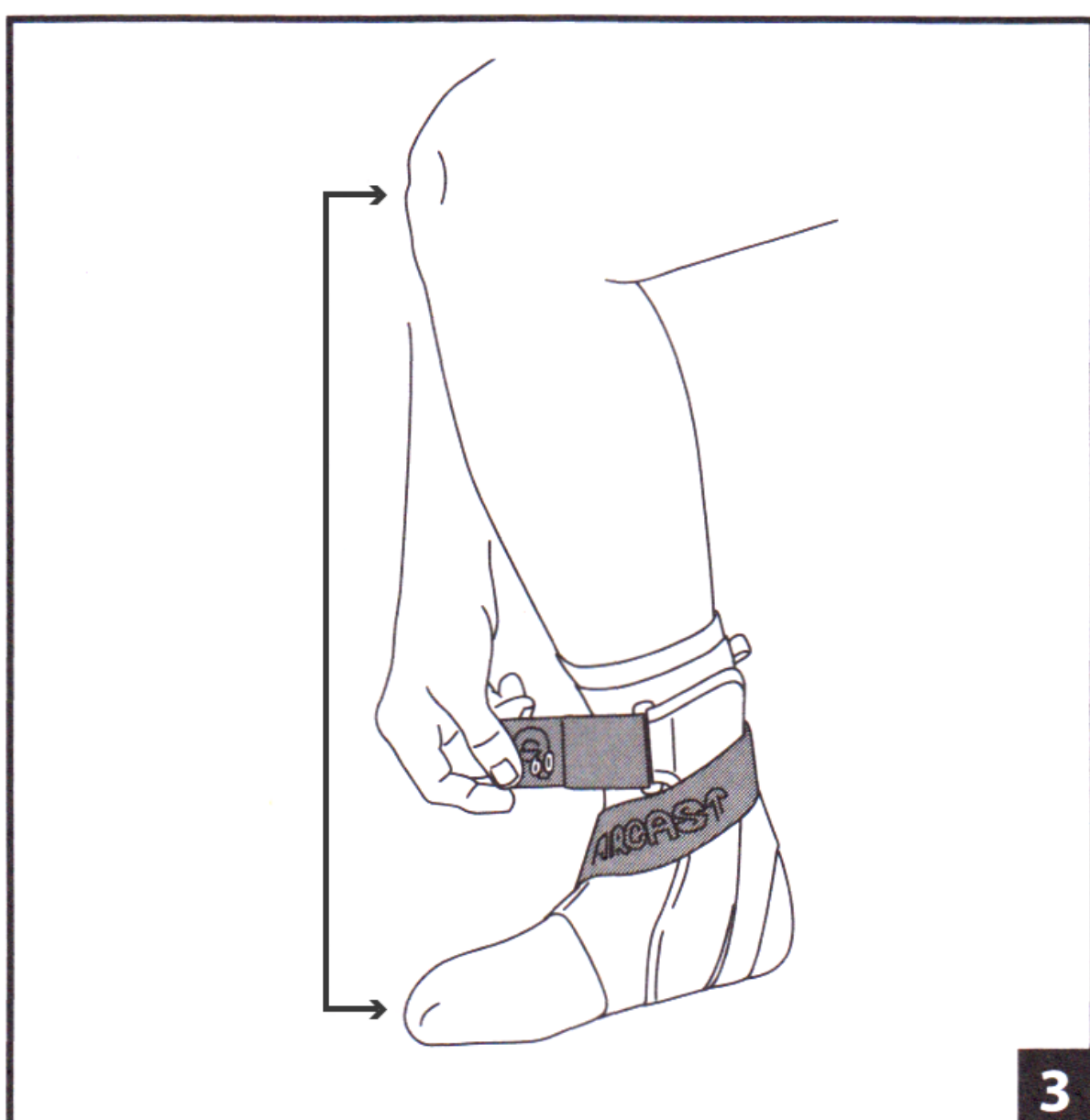

3

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USE/INDICATIONS:**

The Aircast A60™ Ankle Support is designed to provide support and protection for ankles.

**CON TRAINDICATIONS: N/A**

**WARNINGS AND PRECAUTIONS:** If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

**APPLICATION INFORMATION:**

1. Put on sock
  - Unfasten strap from black plastic ring, open top flaps, and open brace
  - Slide foot, toe first, into top of brace
  - Pull brace stabilizers upward and align stabilizers with sides of ankle
2. Wrap flaps around top of ankle and secure
3. Insert end of strap into black plastic ring
  - Bend knee and align over toes
  - Pull strap until snug and secure strap end onto body of strap

**CLEANING INSTRUCTIONS:**

The entire A60 Ankle Support can be hand washed in lukewarm water with mild soap and air dried.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

For single patient use only.

**LATEX FREE**