

## **WARNING: Please read before using this product.**

1. Serious injury or death can result if bed rails are not properly installed. Entrapment has resulted in injuries and death. Entrapment is a situation where an individual can become wedged between a bed rail and any other object. The possibility of an individual's head, body, arms, or legs could be trapped between the bars of a bed rail, between the bed rail and a head or footboard of a bed, or between the bed rail and a mattress. While there are no specific guidelines for bed rails, Standers Inc. is committed to providing safe products and informing the user of ways to prevent accidents.

### **SAFETY PRECAUTIONS:**

- ✓ Regularly check the bed rail to identify areas of possible entrapment.
  - ✓ Position the bed rail at least 18" away from head or footboards of a bed.
  - ✓ Make sure that there is no gap between the bed rail and the side of the mattress. A gap could allow the user to become wedged between the bed rail and the mattress.
  - ✓ Do not use this product with light weight foam or air mattresses. Light weight foam or air mattresses will allow the bed rail to move and may allow users to become trapped between the bed rail and bed.
  - ✓ Additional safety measures should be taken for users that are a high risk. Such users include those who are not mentally conscious (including those individuals under medication), users who have size or weight conditions that may change the gap between the bed rail and the mattress, and users who are generally restless while awake or asleep.
  - ✓ If you have any concerns or questions regarding these precautions, please contact Standers at 1-800-506-9901.
2. If you are suffering from an ailment or injury or are taking medications that affect your balance or ability to stand or sit down, you should first consult with a physician before using this product. If the product is not used properly, serious injury or harm could result.
  3. The product was not intended to carry the full weight of an individual. This product is only intended to provide balance and support while sitting and standing.
  4. The product instructions must be carefully followed to ensure that the product is properly assembled and positioned. Failure to follow the instructions could result in serious injury to the user(s) of the product.
  5. Children should not be allowed to hang on or play with the product.
  6. It is important that the weight of the mattress is adequate to support the product. The bed rail is not well suited for and should not be used with light-weight foam or other light-weight mattresses.
  7. **DISCLAIMER:** Neither Standers, Inc. nor the manufacture is liable for any harm or injury that may result or has resulted from the improper use or assembly of the product.

### **Limited Warranty**

Congratulations on your purchase of your Standers brand product. We hope that it will ease your daily living. Please read the following warranty statement for your protection.

Standers, Inc. warrants to the original purchaser that this product will be free from defects in material and/or workmanship for 1 year from the date of purchase. During the warranty period, Standers, Inc. will correct any defects in material or workmanship, at no charge. The original purchaser, however, must prepay shipping cost. Any replacement parts will be new and are warranted for the remainder of the original warranty or thirty (30) days from the date of shipment, whichever is longer. The original purchaser must notify Standers, Inc. during the warranty period in writing of the defect in workmanship or material. This warranty is not transferable and covers only normal use. **NO SHIPPING CHARGES ARE REFUNDABLE.**

The warranty does NOT cover the following:

- Damage during shipment other than original shipment to retailer.
- Damage caused by unauthorized attachments, alterations, or modifications.
- Damage caused by the use of the product for purposes other than those for which the product was designed or intended.
- Damage caused by any other abuse, misuse, mishandling or misapplication.

Under no circumstances shall Standers, Inc. be liable for any special, incidental, or consequential damages based upon breach of this limited warranty, breach of contract, or strict liability.

**THERE ARE NO WARRANTIES WHICH EXTEND BEYOND THE DESCRIPTION ON THE FACE HEREOF. ALL OTHER WARRANTIES, EXPRESS OR IMPLIED (INCLUDING ALL IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE), ARE HEREBY DISCLAIMED.**


Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This warranty gives you specific legal right, and you may also have other rights, which vary from state to state.

To obtain service under this warranty, you can reach Standers Inc. at 74 Canterbury Circle, Logan, Utah 84321.




# Assembly Instructions

1. Remove all parts from the box and become familiar with the parts as shown in Figure 1.

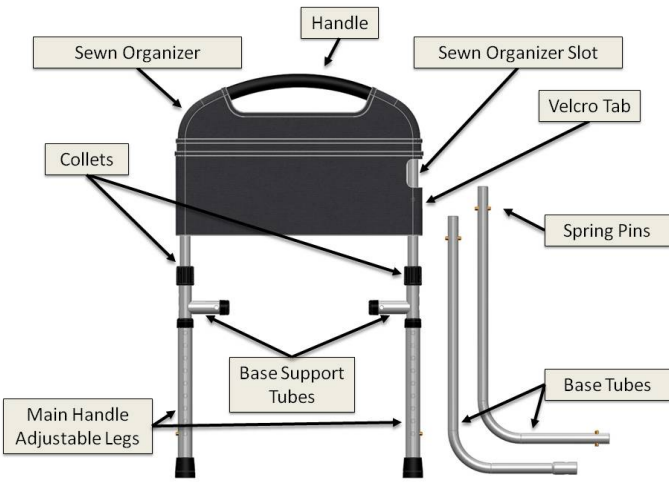


**FIGURE 1**

2. Connect Base Tubes together as shown in Figure 2.

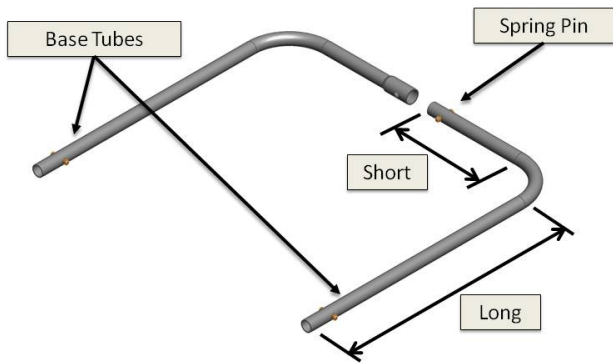


**FIGURE 2**

3. Attach Base Tubes into the Base Support Tubes as shown in Figure 3. Make sure the Spring Pins engage the hole in the Base Support Tubes.

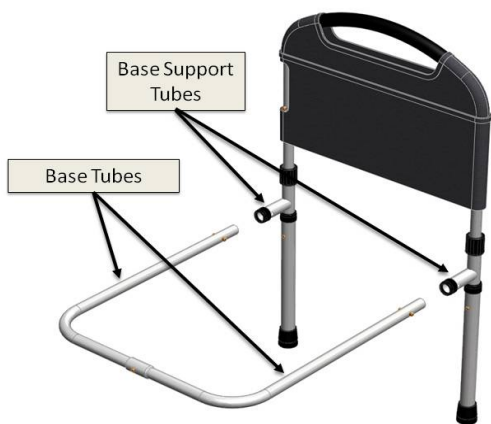

**FIGURE 3**

4. As shown in Figure 4, fully insert the assembled product between the mattress and box spring of the bed. Note that product should be at least 18" away from the head of the bed and the Handle should be 4" to 6" above the top of the mattress.

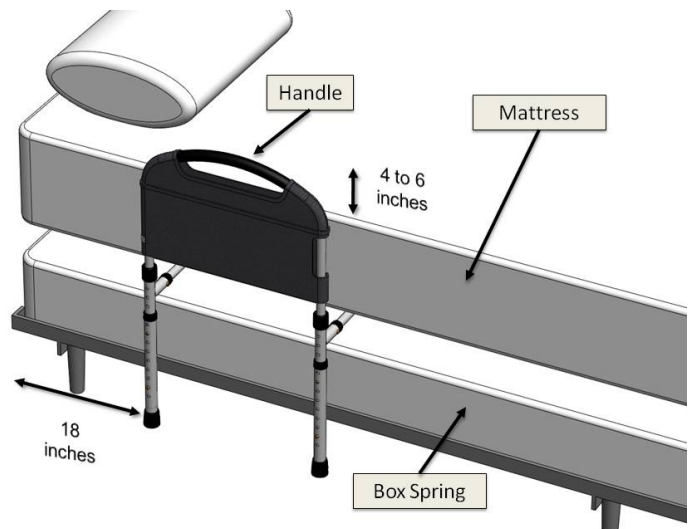


**FIGURE 4**

5. **Adjusting height of Main Handle:**

As shown in Figure 5, the height of the handle can be adjusted up or down by loosening the Collets by rotating counter clockwise. Depress both spring pins so the handle can move up or down to the desired height. After adjusting the Main Handle to the desired height, tighten Collets by rotating clockwise.

6. **Adjusting height of Main Handle legs:**

As shown in Figure 5, the Main Handle Adjustment Legs can be adjusted up or down to engage the floor by depressing the spring pins so the legs can move up or down to the desired height.

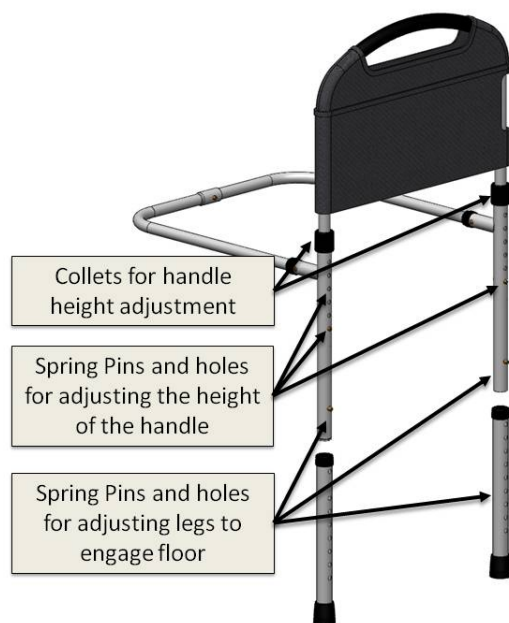

**FIGURE 5**