

CAREBAG®

MEN'S URINAL

With Super Absorbent Pad

Product Description

The CareBag® Men's Urinal is designed to replace traditional plastic or stainless steel urinals. Ideal for those in wheelchairs, patients confined to bed, or when toilets are not readily available. The CareBag® Men's Urinal is also safe and convenient for ER or transport use.

Benefits

- Safe, convenient handling, collection, transport, and disposal of urine
- Single-use, more hygienic than traditional men's urinal
- Quickly absorbs urine
- Holds approximately 16 oz (450 ml)
- Eliminates unpleasant odors from urine when completely closed

Instructions for Use

1. Open box and remove bag.
2. Unfold and open the bag, see the super absorbent pad inside.
3. Hold the bag open and urinate into the bag.
4. Urine turns to gel in seconds.
5. To close and seal: pull the blue string on one side of the bag and the white string on the other side of the bag until the opening of the bag closes tightly. Tie a knot to seal the bag opening.

Disposal: like diapers

- If patient is not infectious, dispose of closed liner with regular waste
- If patient is infectious, discard infectious waste according to facility infection control guidelines.
- Do NOT flush.

When collecting and measuring urine using the Cleanis® CareBag Men's Urinal, simply weigh the bag (in grams) and deduct the volume of urine.

Weight of bag + pad before use = 22 grams (gr)

1 gram = 1 cc of water

eg.: If after use, the CareBag® Men's Urinal weight = 100 gr, then the volume of urine collected = 100 - 22 = 78 gr

Precautions

Single-use product. Do NOT reuse (⊗)
Supervise children using the Urinal Bag
DO NOT FLUSH the CareBag® Men's Urinal

Product Composition

Bag: Polyethylene liner with colored strings
Pad: Non-woven composite of pulp and acrylate polymer

Absorption Capacity

Absorbs up to 16 oz (450 ml) of organic liquids

Identification & Expiration Date

Each box is identified by a reference/lot number and expiration date (5 years)

Safety Data

- This product is not considered dangerous according to the European Union regulation
- MSDS sheets available upon request

Conformity

- Compliant with the FDA regulation Class I, 510(k) exempt medical device
- Compliant with the 93/42/CEE regulation for medical device - Class I
- Eco-friendly: Compliant with anti-pollution legislation on incineration risks. During incineration, the product does not release any chloride element or byproduct that might create dioxin.
- Customs code: 5603149090
- Company certification: ISO 9001:2008 and ISO 13485:2003

Reorder Number: 7733993

Unit	Box of 20 bags	Liner: 6 ^{5/16} " x 14 ^{15/16} ", Pad: 5 1/8" x 9 ^{7/16} "
Case	21 boxes of 20 bags (total of 420 bags)	30 ^{1/16} " (L) x 14 ^{1/2} " (W) x 6 ^{5/8} " (H); 22.05 lbs
Pallet	33 cases (693 boxes, total of 13,860 bags)	31 ^{1/2} " (L) x 47 ^{1/4} " (W) x 77 ^{3/16} " (H); 750 lbs



1375 Broadway Avenue, Floor 11, Suite 1100, New York, NY 10018
Website: www.Cleanis.com • Phone: 646-278-5627 • Fax: 646-253-1258