

Nurse Call Adapter Cable Set

REF 8235NCA	REF 8235NCC	REF 8235NCD	REF 8235NCF
REF 8235NCG	REF 8235NCHH	REF 8235NCJ	REF 8235NCJJ
REF 8235NCK	REF 8235NCL	REF 8235NCM	REF 8235NCMM
REF 8235NCNN	REF 8235NCP	REF 8235NCPP	REF 8235NCR
REF 8235NCRR	REF 8235NCS	REF 8235NCVV	REF 8235NCWW
REF 8235NCXX			

You can connect the following Posey alarms: Sitter Elite (8345), KeepSafe Deluxe (8374, 8374NP) and Sitter Select (8361) to your facility's nurse call system to communicate with the nursing station in one of two ways:

1. The 8282 will connect the alarm to a nurse call system if the wall outlet has a ¼" jack and you want to connect the alarm only.
2. If the wall outlet is not a ¼" jack or if you need a nurse call button in conjunction with the alarm, you will then require the Nurse Call Adapter Cable Set (adapter and nurse call cable). We offer the following set of part numbers that include both adapter and nurse call cable for use in tandem. (See list above.)



Alarm with Nurse Call Cable

To Connect the Nurse Call Cable Only

Insert one end of cable into aperture labeled "Nurse Call" jack on the appropriate side of alarm (8345 right, 8374 left, 8361 left).

Insert other end of cable into wall jack of nurse call panel.

Connecting the Adapter to Nurse Call Cable

If you require an adapter to connect to the nurse call cable to communicate with the nursing station:

Insert one end of the nurse call cable into one socket of the adapter. Then place the nurse call cable system connector into the wall outlet. Plug your existing call cord into the other socket of the adapter.


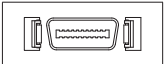
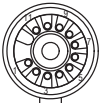
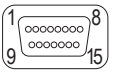
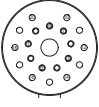
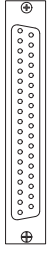
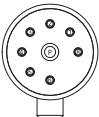

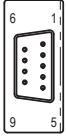
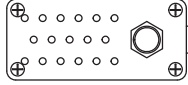
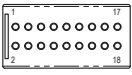
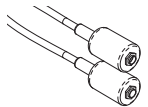
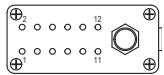
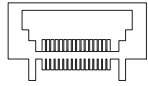
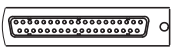


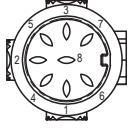




⚠ WARNING When using a nurse call cable, ensure the nurse call cable is plugged into both the alarm and wall jack before leaving the patient unattended. Verify that an alert is received at the nursing station if the cable is unplugged from the wall jack.

NOTE: There will be no alert at the nursing station or at the bedside if the nurse call cable is unplugged from the alarm.

⚠ WARNING

- Do not stretch or strain cable to avoid possible damage and possible malfunction.
- Do not attach cable to moving parts of the bed or chair that will cause strain or damage if the bed or chair is repositioned.
- Always position the cable so that moving parts (side rails, wheels, etc.) will not cause strain or damage the cable.
- Do not run over the cable with carts or equipment.
- Do not wrap the cable tightly during storage.
- Always remove the cable by pulling on the jack. Do not pull on the cable.
- Always secure the cable out of the way so that it will not be a tripping hazard.
- Always test alarm and nurse call function prior to leaving the patient unattended. Activate the alarm (remove pressure from sensor, unfasten chair belt sensor) and make sure the nurse call light for the proper bed and room activate in the hall and at the nurses' station.
- Do Not use the alarm sensor if they don't activate each time weight is removed from the sensor or the chair belt is unfastened. Replace alarm and sensor with working units and retest before leaving patient unattended.

Cat. No.	Adapter Description	Plug Configuration	Cat. No.	Adapter Description	Plug Configuration
8235NCA	Executone 7-Pin Adapter (C/P Not Wired)		8235NCNN	Hill-Rom Adapter Box D35M to T35F & ¼" Jack w/Dummy	
8235NCC	Rauland 11-Pin M-F to S12F Adapter		8235NCP	Fisher Berkeley Adapter Box D51M to T51F & ¼" Jack w/Dummy	
8235NCD	Philips/Sylvania 20-Pin M-F to S12F Adapter		8235NCPP	Dukane Box Adapter DB37P Male to DB37P Female & ¼" Bed Exit Jack	
8235NCF	Executone 7-Pin Futura M-F Adapter to ¼" Phono Jack		8235NCR	Wescom Adapter C59M to S59F & ¼" Jack w/Dummy	
8235NCG	Zettler 9-Pin, Sentinel 400, Y-Adapter, Zettler 9-Pin Male to Female & ¼" Jack		8235NCRR	Dukane B27 17-Pin w/Phone Plug Adapter	
8235NCHH	Dukane 18-Pin Adapter		8235NCS	¼" Phone Plug Y-Adapter	
8235NCJ	Dukane 12-pin K97 M-F to S12F Adapter		8235NCVV	Executone 16-Pin Lifesaver Box Adapter D16M to T16F & S12F	
8235NCJJ	Hill-Rom 37-Pin Adapter		8235NCWW	Wescom 10-Pin Modular Adapter M-F w/ 1/4" Jack & Dummy	
8235NCK	Fisher Berkeley 8-Pin Adapter C54M to S54F & S12F		8235NCXX	Intercall Adapter C17M to S17F and S12F with ¼" Dummy	
8235NCL	Executone 21-Pin M-F to S12F Adapter Assy				
8235NCM	Jeron 8-Pin Adapter (Din Plug)				
8235NCMM	Rauland Responder 8-Pin Adapter	