

Please write us anytime at [office@tzmousa.com](mailto:office@tzmousa.com)  
or call **(470) 377-4942** Monday to Friday 8 AM – 4.30 PM EST  
if you would like to learn more about our products and order  
free samples. More details about the product and other  
practical information about incontinence at [www.seni-usa.com](http://www.seni-usa.com)

Please visit also our **YouTube** channel '**TZMO Group**'  
to understand more about incontinence and how to deal with it easier.



## Testimonials

Since my husband is wearing Seni at night no more accidents happen. I do not have to take extra clothing with me in case the pads do not hold. We can recommend them for everybody who is heavily incontinent. They are excellent. My husband will not wear any other brand but these.

Erika, Family caregiver, FL

As a geriatric Care Manager I have many clients who are incontinent or experience other urinary issues. Seni products are absolutely AMAZING. My clients and families love them. They allow them not to fear going out and provide a sense of security and comfort. They also prevent any skin breakdown. Until you try them you cannot believe how wonderful they are. They are especially good if a person suffers from some type of dementia and gets agitated if they need changing. Due to the design and absorbency they don't need to be changed as often. I never thought I would become so enthusiastic about an incontinence / bladder product but they deserve high praise. You use less products so they are very cost effective.

Lacy, LMSW, GA

I am a professional Care Manager, and I have started using the briefs for my own mother who is living with dementia. I am so impressed with the quality and design elements of these products that I am telling all of my clients' families and other service providers. The briefs help prevent skin issues as they remain dry, and are super absorbent. They don't leak, and best of all, my mother likes wearing them because they do not make her too hot like other brands do. Her comfort and cleanliness are just as important as her dignity, and the Seni products allow me to provide for all three. Thank you Seni!

Lisa, M. Geriatric Care Manager, Atlanta, GA



# seni®


better care  
for your  
loved ones



 TZMO SA

[tzmoglobal.com](http://tzmoglobal.com)  
[seni-global.com](http://seni-global.com)

[seni-usa.com](http://seni-usa.com)

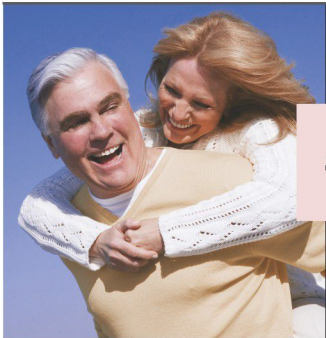




Did you know that every

1 in 4 women  
and every  
1 in 8 men

at different ages suffer  
from incontinence?

Involuntary loss of urine might happen to everyone during everyday activities such as physical activity, sneezing, laughing or coughing. It can also be caused by Parkinson's disease, Alzheimer's disease, multiple sclerosis (MS), spinal cord injuries, obesity, several childbirths or just growing old.

Incontinence degree is not the same in everyone. It is calculated on the base of amount of urine lost within 4 hours. The simplest division of incontinence degrees is into light, moderate and heavy, as follows:

Degrees of incontinence	LIGHT	MODERATE	HEAVY
			
Volume of urine involuntarily passed /4h	up to 3 fl oz / 100 ml	3-7 fl oz / 100-200 ml	over 7 fl oz / 200 ml

Seni offers a full range of incontinence products. Choose the one you need.



Free samples available, see last page.



Seni products are designed with the users' comfort in mind. They have many features that help prevent leakages and keep the skin dry. Using Seni, there is no need to give up your social activities – you can live as you like.

SIDE GATHERS



Hydrophobic standing side gathers that protect from side leakage and ensure additional protection. Many leaks happen on the side area and this is why side gathers are very important.

VAPOR PERMEABILITY



Seni's outer layer allows vapors to go through helping sensitive skin to breathe. Millions of microscopic pores allow vapor to flow in and out, but keep liquid and odor in.

SAP



Superabsorbent inside the double absorbent core locks in moisture so that it doesn't irritate delicate skin. Locking urine decreases also the bacterial growth and helps contain urine odor.

EDS



EDS (Extra Dry System) gives a nice, comfortable dry feeling next to the skin. Delicate, non-woven EDS distributes wetness quickly along the product and then locks it in so it reduces moisture coming back in contact with the skin.

SOFTNESS



Seni products are delicate for compromised skin because they're made with soft cloth-like materials rather than harsh or stiff plastics. This also helps make them quiet and discreet.

ANATOMICAL SHAPE



Perfect adjustment to the body and comfort of use. Better fit, less chances of leakage, better protection.

Not made with natural rubber latex which can irritate skin.



Why it is worth to use  
Seni Premium Quality  
absorbent products?



### Benefits of Seni:

- Less sweating thanks to vapor permeable outer layer
- Lower risk of leakages
- Less frequent underpads and/or bedding changes
- Reduction of the risk of skin irritations
- Bedsores prevention
- Less frequent medical appointments = time and money savings
- Comfort and safety

The high absorbency and vapor permeable outer layer of Seni Premium Quality products make it possible to use them longer and at the same time keeping the skin dry to prevent irritation or breakdown. This helps you save time and money.


### Changing product less frequently means:

- No need to disturb sleep so often
- More dignity
- Less pain for chronically ill
- More time to spend with the loved ones on talking, walking, playing or reading etc.
- The freedom of going out and enjoying life without being afraid of embarrassing leaks
- Lower use of briefs/underwear and underpads
- Lower use of washcloths and latex gloves for changing
- Lower expenses associated with laundry loads and linen changes
- Less physical work for caregivers




When using premium incontinence products you also dispose less and make your contribution to protect the Earth!

To know when to change the product observe wetness indicator.

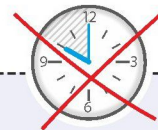

Seni absorbent products perform very well and **there is no need to change them every 2 hours.** Typically they can stay in place longer. This always depends on the individual needs (e.g. products can be used for 4-6 hours; up to 8 hours for products with high absorbency or "night products").


To find out what is the proper time to change the soiled product and avoid leakages observe the wetness indicator performance – yellow stripes and/or inscription along the briefs or pads. When the stripes turn green/blue and the inscription fades, the product is soiled. **When 50-80% of wetness indicator has changed the color, it is time to change the product.**

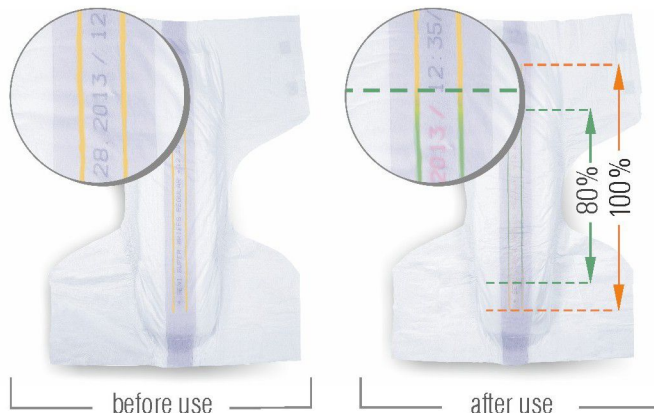

The only situation when the absorbent product should be changed immediately – without looking at the indicator – is the release of feces.

# Seni Lady


for light to moderate incontinence

 Premium Quality bladder control pads for women available in seven sizes.



Side gathers, anatomical shape and wide adhesive strip that secure the pad to the underwear help to prevent leakage. Many leaks happen on the side area so the Seni Lady **side gathers** ensure extra protection. The pads are made of soft, **vapor permeable** fabrics and have **EDS** that distributes wetness quickly along the pad and helps protect the skin. They are individually wrapped for your convenience and discretion.

## Seni Lady selection

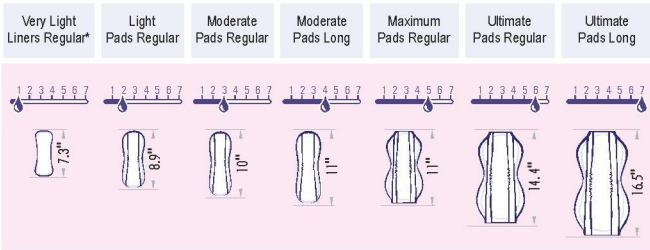

Item #	Seni Lady size	Length	Pieces in Pack	Packs per Case
S-1L36-PL1	Very Light Liners Regular*	7.3"	36	30
S-2P30-PL1	Light Pads Regular	8.9"	30	16
S-3P28-PL1	Moderate Pads Regular	10"	28	16
S-4P26-PL1	Moderate Pads Long	11"	26	20
S-5P30-PL1	Maximum Pads Regular	11"	30	16
S-6P28-PL1	Ultimate Pads Regular	14.4"	28	8
S-7P26-PL1	Ultimate Pads Long	16.5"	26	8

\* Do not have side gathers and EDS.

# Seni Man

for light to moderate incontinence

 Premium Quality guards for men available in two types.

## Seni Man Active



Side gathers and the special shape of Seni Man Active together with the wide adhesive strip secure the guard to the underwear and help to prevent leakage. Many leaks happen on the side area so the Seni Man Active **side gathers** ensure extra protection. **EDS** distributes wetness quickly along the guard and then it is locked in so it reduces coming back of wetness in contact with the skin. They are made of soft **vapor permeable** materials.

## Seni Man Fit



**"3D" shape** of Seni Man Fit together with the wide adhesive strip secure the guard to the underwear and help to prevent leakage. Many leaks happen on the side area so the Seni Man Fit special shape is a perfect solution. They are made of soft **vapor permeable** materials.

To be used with close body fitting underwear for proper performance. They are individually wrapped for your convenience and discretion.

Item #	Seni Man style	Length	Pieces in Pack	Packs per Case
S-AC30-PM1	Active Guards	11.2"	30	12
S-FT20-PM1	Fit Guards	15.7"	20	8

## Seni Active

Premium Quality underwear available in many sizes.



Seni Active pull-on underwear is a perfect solution for mobile people, people leading an active life and during bladder training. It fits better to better prevent leaks. Leaks are usually caused by the lack of standing side gathers or by gapping due to an ill-fitting product. That's why Seni offers its pull-on underwear with **side gathers** and **leg cuffs**.

Dual layer absorbent core together with **EDS** provide greater absorbency and a feeling of dryness. They are delicate for compromised skin because they're made with soft vapor permeable materials. This also helps make them quiet and discreet. Tear-away side seams are a comfortable and convenient way to remove the underwear.

Choose lavender Seni Active for Women or gray Seni Active for Men for moderate absorbency needs, or Seni Active Super and Seni Active Super Plus in case you need higher absorbency.

for moderate incontinence  
**Seni Active for Women**



Item #	Size	Waist / Hip size	Pieces in Pack	Packs per Case
S-MS18-AW1	Small / Medium	28"-40"	18	4
S-LA16-AW1	Large	38"-50"	16	4
S-XL14-AW1	X-Large	47"-63"	14	4

for moderate incontinence  
**Seni Active for Men**



Item #	Size	Waist / Hip size	Pieces in Pack	Packs per Case
S-MS18-AM1	Small / Medium	28"-40"	18	4
S-LX16-AM1	Large / X-Large	38"-63"	16	4

for moderate to heavy incontinence  
**Seni Active Super**



Item #	Size	Waist / Hip size	Pieces in Pack	Packs per Case
S-SM22-AS1	Small	22"-33"	22	4
S-ME20-AS1	Medium	31"-43"	20	4
S-LA18-AS1	Large	39"-53"	18	4
S-XL14-AS1	X-Large	47"-63"	14	4

for heavy incontinence  
**Seni Active Super Plus**



Item #	Size	Waist / Hip size	Pieces in Pack	Packs per Case
S-SM10-AP1	Small	22"-33"	10	4
S-SM22-AP1			22	4
S-ME09-AP1	Medium	31"-43"	9	4
S-ME20-AP1			20	4
S-LA08-AP1	Large	39"-53"	8	4
S-LA18-AP1			18	4
S-XL07-AP1	X-Large	47"-63"	7	4
S-XL14-AP1			14	4

See details on page 14 about choosing the correct size.  
Is your size not listed? Look at page 11 for the Seni Shaped Pads solution.

**NOTE**




## Seni Super Briefs

Premium Quality briefs available in many sizes.




Seni Super Premium Quality Briefs are excellent and reliable protection dedicated both for active people and those with limited mobility, in urinary and bowel incontinence. Seni Super briefs are made from soft, yet enduring materials. **Vapor permeable** backsheet helps **the skin to breathe easily**. Their standing gathers together with leg cuffs protect better from the side leaks, and elastics in the waist and closing tabs provide better fit. Equipped with **double absorbent core** and **EDS** briefs have moisture distributed and locked in the core increasing the dryness feeling for the user. Double wetness indicator on the outer layer helps to determine when it is time to change the product.

 for heavy incontinence  
**Seni Super Briefs**




Item #	Size	Waist / Hip size	Pieces in Pack	Packs per Case
S-XS25-BS1	X-Small	16"-24"	25	3
S-SM25-BS1	Small	22"-31"	25	3
S-ME25-BS1	Medium	30"-43"	25	3
S-RE25-BS1	Regular	40"-50"	25	3
S-LA25-BS1	Large	45"-59"	25	3
S-XL25-BS1	X-Large	55"-67"	25	3

 for heavy to severe incontinence  
**Seni Super Plus Briefs**



Item #	Size	Waist / Hip size	Pieces in Pack	Packs per Case
S-SM12-BP1	Small	22"-31"	12	4
S-ME10-BP1	Medium	30"-43"	10	4
S-ME25-BP1			25	3
S-RE10-BP1	Regular	40"-50"	10	4
S-RE25-BP1			25	3
S-LA09-BP1	Large	45"-59"	9	4
S-LA25-BP1			25	3
S-XL08-BP1	X-Large	55"-67"	8	4
S-XL25-BP1			25	3


 for severe incontinence  
**Seni Super Quatro Briefs**



Item #	Size	Waist / Hip size	Pieces in Pack	Packs per Case
S-SM12-BQ1	Small	22"-31"	12	4
S-ME10-BQ1	Medium	30"-43"	10	4
S-RE10-BQ1	Regular	40"-50"	10	4
S-LA09-BQ1	Large	45"-59"	9	4
S-XL08-BQ1	X-Large	55"-67"	8	4

### For the best briefs performance activate them before use:

hold the edges and stretch a few times vigorously and fold lengthwise so that the inscription is in the middle. Make sure that the side gathers are not folded to the inside.

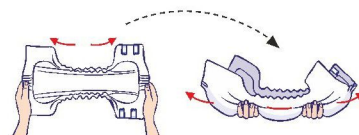

See details on page 14 about choosing the correct size.  
Is your size not listed? Look at page 11 for the Seni Shaped Pads solution.

**NOTE**

## Seni Shaped Pads



Premium Shaped Pads are perfect for active and immobile people. Designed without the side wings clinging onto the hips provide freedom of movement. The outer layer that is made from soft, **vapor permeable fabric helps the skin to breathe** easily and assures increased discretion. Perfect **for those who can't find proper size** among briefs or underwear. Anatomical shape together with **standing gathers** protect better from leakage. Equipped with **double absorbent core** and **EDS** shaped pads have moisture distributed and locked in the core increasing the dryness feeling for the user. The double wetness indicator on the outer layer helps determine when it is time to change the product. They should be worn with elastic, snug fitting underwear such as Seni Supporting Pants.



for moderate to heavy incontinence

### Seni Shaped Pads



Item #	Product Name	Length	Pieces in Pack	Packs per Case
S-UN30-PS1	Seni Day	24.6"	30	3
S-PL30-PS1	Seni Night	27.2"	30	3



### Seni Supporting Pants



Item #	Size	Waist / Hip size	Pieces in Pack	Packs per Case
S-LA02-SP1	Large	31"-47"	2	40
S-XX02-SP1	XX-Large	39"-61"	2	40

## Underpads & Washcloths

Seni Soft hygienic underpads are designed to be used as an additional protection which helps to keep the wheelchair, bed, and bedding clean and dry. Seni Soft is perfect to use when changing a brief.

Seni Soft Super with Wings has side wings to be folded under the mattress to keep underpad in place.

Seni Soft Super Dry with superabsorbent material locks the moisture in and helps to contain urine odor.



### Seni Soft Super



### Seni Soft Super Dry



Item #	Product Name	Size	Pieces in Pack	Packs per Case
S-0130-US1	Seni Soft Super	23" x 15"	30	4
S-0230-US1	Seni Soft Super	23" x 23"	30	4
S-0330-US1	Seni Soft Super	23" x 35"	30	2
S-0430-US1	Seni Soft Super with Wings	35" x 67"	30	2
S-0330-UD1	Seni Soft Super Dry	23" x 35"	30	2

Seni Care washcloths gently, yet effectively cleanse the skin. Alcohol and paraben free to satisfy the needs of sensitive skin. Recommended for frequent cleansing of perineal areas.



### Seni Care Washcloths



Item #	Product Name	Size	Pieces in Pack	Packs per Case
S-WA48-C11	Seni Care Washcloths	12" x 8"	48	12

# Choose the right size



Proper size is very important to have products well fitted and to prevent leakages (which occur when product is too big, too loose) or irritations (which occur when product is too tight).



**too big**  
risk of leakages



**too small**  
irritations possible




**perfect fit**


When selecting the correct size it is not enough to know the weight. It is important to measure the waist and hips.

**The measurement of waist and hips should be taken with the use of a tape measure.**

If both measurements (waist and hip size) are in one range, pick the size this range indicates. See chart below:

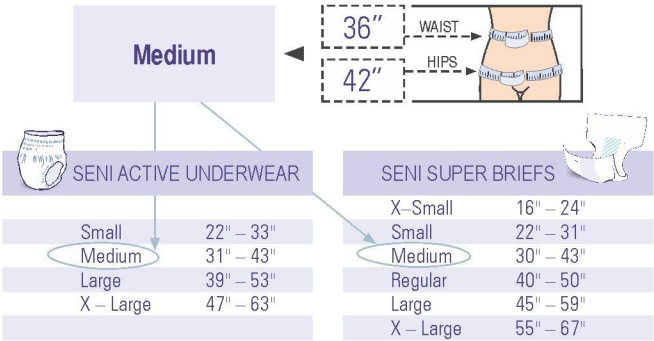

If measurements are in two different ranges, defer to the larger size. See chart below:

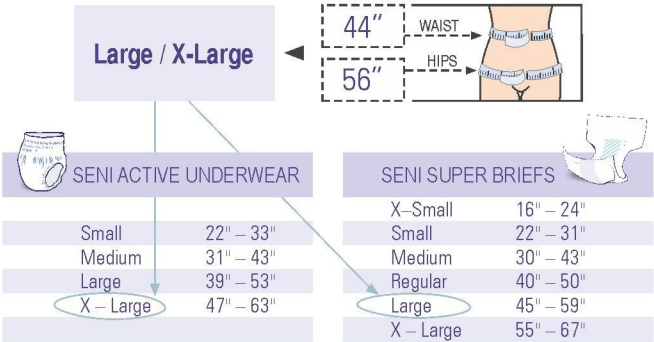





Please provide your information to receive  
free samples and mail it to\*:

**TZMO USA, Inc. 1827 Powers Ferry Road, Building 5,  
Atlanta, GA 30339  
In attn. Marketing Department**

Name: \_\_\_\_\_

Address: \_\_\_\_\_

\_\_\_\_\_

City: \_\_\_\_\_

\_\_\_\_\_

State: \_\_\_\_\_ Zip Code: \_\_\_\_\_

Phone Number: (.....) \_\_\_\_\_

E-mail: \_\_\_\_\_

What product you currently use: \_\_\_\_\_

\_\_\_\_\_

How did you hear about us: \_\_\_\_\_

\_\_\_\_\_

### **\*Choice Regarding Collection of Personal Information / How We Use and Share Information Collected**

You may choose not to provide us with personal information. However, in order to provide you with Seni samples as described above, your personal and contact information is required. We do not connect your personal information we collect by this form with any and all aggregated impersonal information regarding customers' use of our products.

We may use personal information collected from you to - among other things - respond to you, contact you, and send you Seni samples. In addition, we may contact you in the future to provide you with information or services regarding our products.

Your privacy is important to us. When you give us your personal data, we will not sell, trade, or rent your personal information to third parties or share it without your consent. Your personal information may be stored and processed in the United States or any other country where the TZMO Group, its affiliates, or its agents are located.

# Free Sample Request

We will be happy to send you Seni samples to try.  
Please limit your request to **2 items**  
and fill in the address form on the other side.

## Seni Lady:

- ☐ Maximum Pads Regular
- ☐ Ultimate Pads Regular
- ☐ Ultimate Pads Long

## Seni Man:

- ☐ Active Guards
- ☐ Fit Guards

## Seni Active for Women:

- ☐ Small/Medium
- ☐ Large
- ☐ X-Large

## Seni Active for Men:

- ☐ Small/Medium
- ☐ Large/X-Large

## Seni Active Super:

- ☐ Small
- ☐ Medium
- ☐ Large
- ☐ X-Large

## Seni Active Super Plus:

- ☐ Small
- ☐ Medium
- ☐ Large
- ☐ X-Large

## Seni Super Briefs:

- ☐ X-Small
- ☐ Small
- ☐ Medium
- ☐ Regular
- ☐ Large
- ☐ X-Large

## Seni Super Plus Briefs:

- ☐ Small
- ☐ Medium
- ☐ Regular
- ☐ Large
- ☐ X-Large

## Seni Super Quatro Briefs:

- ☐ Small
- ☐ Medium
- ☐ Regular
- ☐ Large
- ☐ X-Large

## Seni Shaped Pads:

- ☐ Day
- ☐ Night

## Seni Soft:

- ☐ Super (23"x35")
- ☐ Super Dry (23"x35")