

Connecting the Humidifier

Warnings

- Do not overfill humidifier.
- Do not reverse the oxygen input and output connections. Water from the humidifier bottle will travel through the cannula back to the patient.
- Do not use the humidifier while the device is in Pulse mode.

Note: Only connect the humidifier if prescribed. Only use the humidifier if the device is in Continuous Flow mode.

To connect the humidifier:

1. Remove the cap from the humidifier bottle.
2. Fill the humidifier with distilled water to the level indicated by the manufacturer.
3. Replace the cap and make sure it is tightened securely.
4. Attach the humidifier bottle adapter to the humidifier bottle by turning the wing nut on the bottle counterclockwise until it is fastened securely.
5. Place the humidifier bottle assembly in the accessory bag. Attach the accessory bag to the pull cart.
6. Rotate the bottle so the humidifier adapter hose is away from the body of the portable oxygen concentrator.
7. Attach the oxygen tubing from the humidifier bottle assembly to the oxygen outlet connector on the portable oxygen concentrator.
8. Attach the cannula tubing to the humidifier bottle outlet.
9. After assembly is complete, make sure the oxygen is flowing through the cannula.

Caring for your SimplyGo System

Cleaning the Device and Rechargeable Battery

Warning: To avoid electrical shock, do not remove the SimplyGo covers. The covers should only be removed by authorized service personnel. Do not apply liquid directly to the covers. Do not use alcohol, solvents, polishes or any oily substances on the device, as they are flammable.

Caution: Do not allow liquids into any of the controls, the interior of the case, or the oxygen tubing connector. If this occurs, contact your home care provider for assistance.

The exterior covers of the device and rechargeable battery should be cleaned as needed:

1. Turn the device off and disconnect from the power source before cleaning.
2. If the battery is removed, wipe the battery bay and battery with a dry cloth.
3. Clean the device exterior using a damp cloth with a mild household cleaner and wipe it dry. Or, if you use medical disinfectants, be sure to follow the manufacturer's instructions. For disinfection, Philips Respironics recommends the following:
 - Discide Ultra Towlelettes: Manufacture - Palmero, www.palmero-health.com (or equivalent)

If you are using a humidifier, clean your device according to your home care provider's or manufacturer's instructions.

Cleaning the Accessories

Follow the instructions below when cleaning the accessories.

Carrying Case and Mobile Cart

If necessary, the carrying case can be washed in a washing machine using a mild laundry detergent. Air dry.

To wash the carrying case and mobile cart by hand, use only warm water and a mild liquid dish detergent.

1. Remove the SimplyGo device from the carrying case or mobile cart.
2. Dampen a cloth in the detergent and water solution and wipe the outside surfaces of the case clean.

AC and DC Power Supplies/Power Cord

The AC and DC power supplies and power cord should be cleaned as needed.

1. Disconnect cords from device before cleaning.
2. Clean the cords using a cloth with a mild household cleaner and wipe dry.

Cannula and Tubing

Clean and replace the cannula and tubing as instructed by the manufacturer and your home care provider.

Device Storage

Store your device in a place where it will remain clean and dry.

Caution: Do not store your device or accessories in extreme temperatures, below -4° F (-20° C) or above 140° F (60° C).

Device Disposal

Dispose of the device in accordance with local regulations or contact your home care provider.

SimplyGo Portable Oxygen Concentrator




Mobility

Your SimplyGo system has been designed to provide you with a source of oxygen that can be carried with you. To enable you to do so, your SimplyGo device has a carrying case bag, accessory bag, and a mobile cart for use with it.

SimplyGo Carrying Case


The SimplyGo carrying case allows you to take your supply of oxygen with you as you go about your normal, daily activities. It provides protection for your SimplyGo as well as a carrying strap to transport the device. To place the SimplyGo into its carrying bag:

1. Position the bag and the SimplyGo so that the front of the device corresponds with the surface of the bag that has a zippered accessory pouch. This will ensure that the inlet air vent on the rear of the device, the outlet air vent on the front, bottom of the device and the power input connect on the side of the device line up with their openings provided in the bag. If the SimplyGo is not placed in the bag properly the air vents will be blocked, and the device will overheat during operation and shut off.
2. Insert the device into the top of the bag and pull the snugly fitting bag up around the device. Check to ensure that the air vents and power input connector align with their openings in the case.
3. Wrap the handle cover around the carrying handle of the SimplyGo and secure it in place by pressing the two Velcro together.

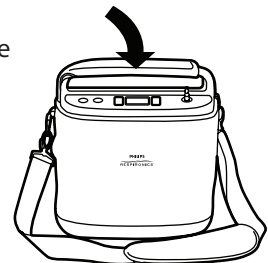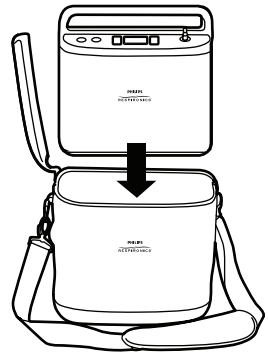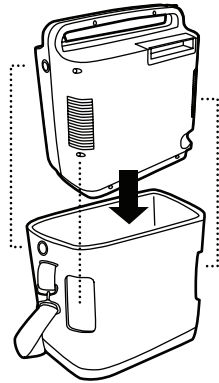

SimplyGo Accessory Bag

The accessory bag is a separate, zippered bag that provides extra space for storing and carrying items such as additional batteries, power supplies, connecting cables, nasal cannulas and personal items you need that cannot be accommodated by the storage pocket in the SimplyGo carrying case.

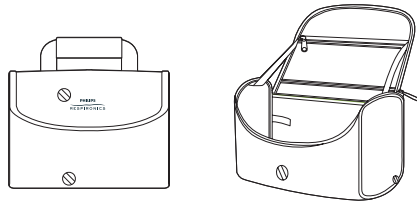

This bag can be attached to the mobile cart along with your SimplyGo device as described below.

SimplyGo Mobile Cart

The SimplyGo mobile cart provides you an alternative method to carrying the SimplyGo in its carrying case. The cart will accommodate both the SimplyGo in its carrying case and the accessory bag. Both can be placed on the mobile cart as shown in the instructions included with the mobile cart.

Traveling With Your System

With your SimplyGo system and proper advance planning, you can enjoy traveling within your community and beyond. Before you leave, make sure you pack the following:

- Fully charged battery (and extra batteries for a long trip).
- AC power supply and connector cord
- DC power supply
- Carrying bag
- Mobile cart

Also, be sure to take the telephone numbers of your home care provider and physician in case of an emergency.