

SILVER ALGINATE PACKING DRESSING

# ALGIDEX™ Ag PACKING GAUZE

## ALGIDEX™ Ag PACKING GAUZE



Unique combination of ionic silver, alginate, and maltodextrin in the simplicity of a packing gauze.

- Effective bacterial barrier to help prevent contamination from external bacteria
- Ideal dressing for deep or tunneling wounds
- Provides immediate and sustained antimicrobial activity against a broad spectrum of pathogens
- Does NOT require activation



## SILVER ALGINATE PACKING DRESSING

# ALGIDEX™ AG PACKING GAUZE

### ANTIMICROBIAL ANALYSIS

Algidex Ag is a technologically advanced wound dressing that has been uniquely formulated with a combination of ionic silver, alginate, and maltodextrin to provide immediate and sustained antimicrobial activity against a broad spectrum of pathogens without inducing bacterial resistance. Algidex Ag has been clinically documented as a cost-effective dressing that provides excellent wound management with a longer wear-time and fewer dressing changes. The silver in Algidex Ag is completely and evenly distributed over the dressing in the ionic state.

### FEATURES & BENEFITS

- Reduces bacterial colonization
- Absorbs wound exudate
- Decreases wound odor
- Specially woven gauze will not ravel, lint, or fray
- Provided in 5 yd. strips
- Can be cut to desired length
- Releases active ionic silver for broad antimicrobial effectiveness
- Convenient reclosable bottle

### INDICATIONS

Algidex Ag Packing Gauze is used for the management of "deeper" wounds that require packing such as tunneling or undermining wounds (pressure ulcers, venous ulcers, diabetic ulcers, and surgical wounds).

### DIRECTIONS FOR USE

1. Thoroughly cleanse wound with normal saline.
2. Remove appropriate amount of packing gauze from bottle.
3. Completely fill wound with packing gauze (lightly packing).
4. Cover with appropriate secondary dressing such as gauze, hydrocolloid, foam, or film dressing. Secondary dressing selection should be based on the amount of wound drainage.
5. Dressing may be worn up to 5 days or until cover dressing reaches saturation.



### REMOVAL

1. Remove secondary dressing.
2. Gently remove packing gauze.
3. Thoroughly cleanse wound to remove wound drainage or residue from Algidex Ag.
4. Reapply as necessary following directions above.

Product #	Size	Qty/Cs
46-PG12	½" x 5 yds.	12
46-PG14	¼" x 5 yds.	12



200 DeBusk Lane  
Powell, Tennessee 37849  
[www.deroyal.com](http://www.deroyal.com)

TO PLACE AN ORDER OR FOR MORE INFORMATION  
PLEASE CALL: **800-251-9864**  
865.938.7828 (OUTSIDE USA)  
800.327.0304 (FAX)

Reprint #0-2162  
Rev. 1/09  
©2009 DeRoyal