




3M™ Tegaderm™ I.V. Advanced Securement Dressing with Comfort Adhesive Technology

1683 P.I.V. APPLICATION TECHNIQUE



1.

Prepare the site according to your facility's protocol. Allow all preps and skin protectants to dry completely. Open package and remove sterile dressing.

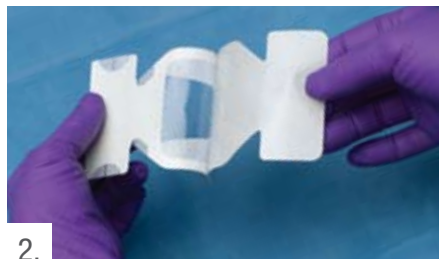

2.

Remove liner to expose adhesive.



3.

Hold end of dressing with adhesive side facing skin. Bend slightly with thumb.



4.

Position dressing with bottom of notch placed directly below luer lock hub. Pinch in place around catheter.



5.

Apply firm pressure over film to establish adhesion to hub and skin.



6.

Wrap ends of notch beneath catheter hub and apply firm pressure.



7.

Slowly peel back frame while simultaneously pressing down dressing edges.



8.

Remove precut sterile tape strips from paper frame. Apply each tape strip as shown above. Recommended application (a–d):

- Apply strips parallel and adjacent to each other over the luer hub and continue straight out across dressing.
- Tape strips are applied over the hub, not under.
- Pinch tape around luer hub and apply firm but gentle pressure over all surfaces to maximize adhesion.
- Apply tape label.

1683 P.I.V. REMOVAL TECHNIQUE



1.

Carefully remove tape strips from dressing. Then gently grasp soft cloth on both sides of notch and separate.



2.

Slowly peel dressing back over itself while stabilizing catheter and supporting surrounding skin.