




# 3M™ Tegaderm™ I.V. Advanced Securement Dressing with Comfort Adhesive Technology

## 1688 APPLICATION TECHNIQUE



1.

Prepare the site according to your facility's protocol. Allow all preps and skin protectants to dry completely. Open package and remove sterile dressing.

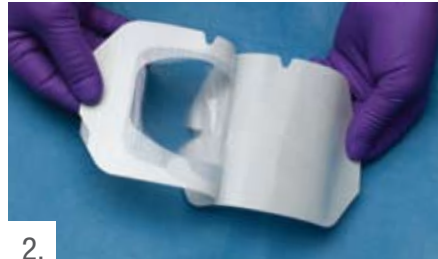

2.

Peel liner from dressing, exposing adhesive surface. Flip dressing over so adhesive faces skin.



3.

Positioning notched edge of dressing over catheter hub, place stabilization border over catheter "wings." Smooth down all dressing surfaces. Device should be stabilized according to facility protocol.



4.

Slowly remove paper frame while continuing to smooth down outer edges of transparent dressing.



5.

Smooth dressing from center toward edges using firm pressure to enhance adhesion.



6.

Remove the large precut notched sterile tape. Apply tape strip across the stabilization border of the dressing over the catheter hub. The notch opening should face away from the catheter insertion site, or be parallel with the dressing notch. Press all surfaces firmly but gently to enhance adhesion. Label dressing according to our facility's protocol. Secure catheter lumens or extensions.

## 1688 REMOVAL TECHNIQUE



1.

Remove tape strip over dressing.



2.

Gently grasp the edge of the dressing and slowly peel dressing back over itself toward insertion site or in direction of hair growth. Keep removal "low and slow." Support catheter hub and surrounding skin with other hand.