

3M™ Tegaderm™ I.V. Advanced Securement Dressings

1689



1689 Application Technique



1 Prepare the site according to your facility's protocol. Allow all preps and skin protectants to dry completely. Open package and remove sterile dressing.



2 Remove liner to expose adhesive. Place dressing adhesive side up on sterile field or inside of dressing package.



3 Position notch edge of dressing over catheter hub, placing stabilization border over catheter "wings." Do not stretch dressing over skin surface.



4 Apply dressing to skin applying firm, gentle pressure to window film. Pinch film firmly around catheter tubing connection. Gently lift tubing connection and press border firmly under catheter lumens.



5 Slowly peel frame away from border edge while simultaneously applying firm pressure to border edge. Smooth entire dressing into skin to ensure optimal adhesion.



6 Remove securement tape strip with notch from liner. **Two small adhesive free tabs from liner will remain on tape strip liner.** Partially fold securement tape strip in half at notch.



7 Apply tape strip's pre-cut notch completely underneath catheter's molded junction. The notch opening should face the insertion site. Peel paper tabs from liner corners. Smooth entire dressing surface; working from catheter insertion site out toward dressing edge.



8 Prepare and apply documentation label over the top of the exposed molded junction. Press first half of strip against the junction; then adhere the other side.



9 Apply firm pressure to entire dressing to ensure optimal adhesion.

1689 Removal Technique



1 Remove all securement tape strips from dressing.



2 Gently grasp soft cloth edges below keyhole notch.



3 Lift dressing on each side from skin using low and slow technique.



4 Peel dressing toward catheter insertion site until removed.

3M™ Tegaderm™ I.V. Advanced Securement Dressings

Provides secure hold

- Reinforced border, deep notch, and tape strips work together to enlist the entire dressing in securement
- Innovative adhesives hold strongly, manage moisture, and release gently
- Designed to maintain securement by preventing edge lift, flexing with patient movement, and managing moisture

Excellent value


- Can potentially reduce the number of dressing changes and restarts
- May be worn for the life of a PIV catheter - Provides up to 7 days of wear time for CVC's**

Easy to apply and remove

- Designed to promote consistent application
- Frame delivery makes placement accurate and easy

Supports infection prevention goals

- A waterproof film coating on all pieces provides a barrier to external contaminants including liquids, bacteria and viruses*
- Allows continuous monitoring

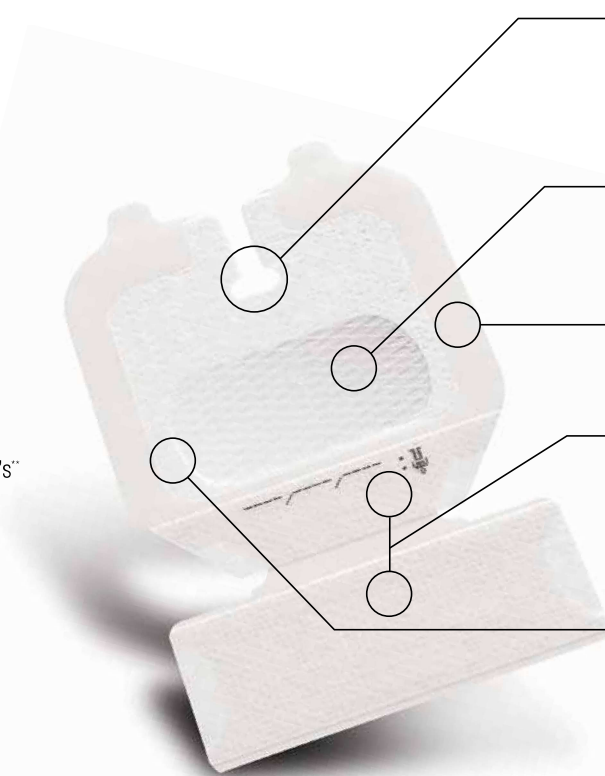

Deep notch

- Fits easily and conforms around hubs and lumens
- Designed to discourage edge lift
- Designed to reduce the risk of catheter manipulation and dislodgement

Highly breathable transparent film

- Designed to release moisture quickly to provide better wear time and securement
- A waterproof, sterile barrier to external contaminants*

Picture-frame delivery

- Minimizes potential to stick to gloves or to itself
- One-handed application

Securement tape strips

- Enhance securement while promoting consistent application
- Waterproof film coating
- Pre-printed documentation label for dressing changes

Stabilization border

- Maximizes securement, breathability and wear time
- Flexible and conformable
- Reduces edge lift to maintain securement
- Waterproof film coating

**In vitro* testing shows that the transparent film of 3M™ Tegaderm™ I.V. Advanced Securement Dressing provides a viral barrier for viruses 27 nm in diameter or larger while the dressing remains intact without leakage.

**O'Grady, N et al. (2011). Guidelines for the Prevention of Intravascular Catheter-Related Infections, 2011. Retrieved from <http://www.cdc.gov/hicpac/BSI/BSFguidelines-2011.htm>. (pp. 32-35)

Not made with natural rubber latex.

To learn about Tegaderm™ I.V. Advanced Securement Dressing or the full line of Tegaderm™ Brand I.V. Site Dressings, visit us at www.3M.com/TegadermIVBrochure, contact your 3M Critical & Chronic Care Solutions representative or call the 3M Health Care Customer Helpline at **1-800-228-3957**. Outside of the United States, contact the local 3M subsidiary.

3M™ Tegaderm™ Dressings SIMPLE. DEPENDABLE. TRUSTED.



Critical & Chronic Care Solutions Division

3M Health Care
2510 Conway Avenue
St. Paul, MN 55144
USA
1 800 228-3957
3M.com/c3sd

Please recycle. Printed in U.S.A.
© 3M 2011, 3/2014. All rights reserved.
3M, Cation and Tegaderm are trademarks of 3M.
Used under license in Canada.
70-2010-8383-2