


Get instant COVID test results in 15 minutes.

New variants have shown that anyone can be vulnerable to COVID-19 even if they are vaccinated or young. With iHealth COVID-19 Antigen Rapid Test, you can test yourself after being out in public. Early and regular testing helps you better care for yourself and protects your friends, family, and community members from potential exposure.

4 Steps. 15 Minutes. Zero Discomfort.


iHealth COVID-19 Antigen Rapid Test is the simplest way to find out if someone has COVID-19. It only takes 4 steps and 15 minutes to complete the test. The test is also non-invasive. You won't need to collect a sample from deep in your nasal cavity to get accurate results.



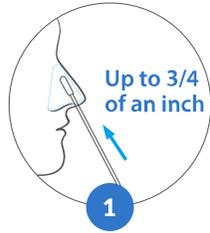

Swab

Brush against the inner wall of both nostrils 5 times each in a circular motion with a non-invasive nasal swab. Our nasal swab is soft, highly absorbent, and only needs to be inserted $\frac{1}{2}$ - $\frac{3}{4}$ inches so you can test yourself comfortably.

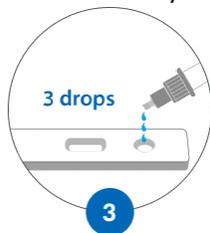


Drip

Put 3 drops of the mixed solution onto the sample port of the



Dip

Insert the swab with the sample into the bottom of the tube and stir it in the fluid 15 times. Squeeze the sides of the tube around the swab as you pull it out.

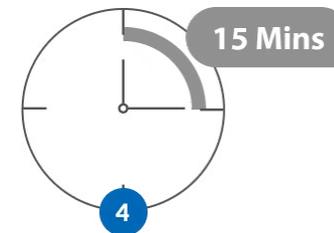

Wait 15 minutes

Start the timer. Your results will be ready in 15 minutes.

Flu or COVID, 15 mins to get peace of mind.

In case of physical discomfort such as fever, sneezing, cough or fatigue, you can quickly know if you have COVID-19 within 15 minutes by using iHealth COVID-19 Antigen Rapid Test.



Fever



Sneezing



Fatigue

Quick in-home testing takes away the worry of developing symptoms.

Back to school, back to the office, or just back from travel or public gatherings, a rapid, 15-minute, self-test is waiting at home for when you get back.



Each iHealth COVID-19 Antigen Rapid Test kit includes 2 tests

Its compact size makes it easy to store it away or take it with you in a travel bag.



**An in-home test designed to
get accurate results.**

Not only is the iHealth COVID-19 Antigen Rapid Test easy and non-invasive, it's a small engineering beauty. The test has a highly-absorbent swab that is individually sealed to help prevent contamination so you can view the results of your in-home test with confidence.



Highly-Absorbent swab

Made of PU foam, the absorbent tip of the swab makes sample collection more efficient than a regular swab.




Double-Cap Security Design


The double-cap design allows you to safely store the liquid and then precisely drip the mixed sample onto the test card.



Easy-to-Understand Results

The unique test card shortens the time to get your results and makes them easy to read. If only the C line shows up, the test is negative. If both C and T lines show up, the test is positive.

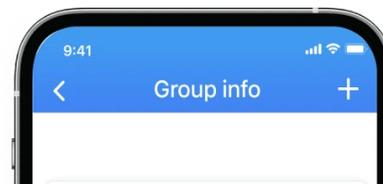

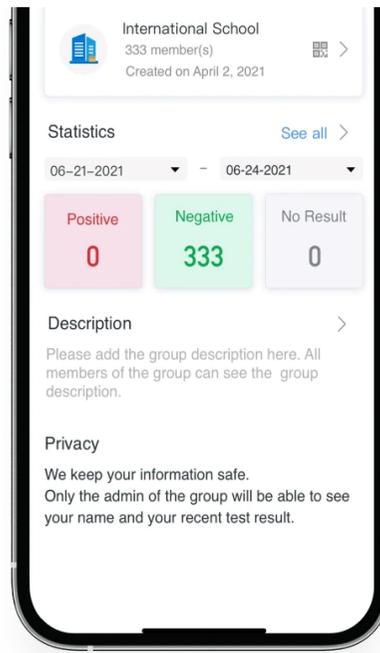

Group COVID-19 testing for schools, event planners, administrators, and organizers everywhere.

Test and monitor results for large groups of people using iHealth in-home testing and the iHealth COVID-19 Test app.

Results Management

+

iHealth Pass





+

Vaccine Proof

+

Independently-sealed packaging ensures safe and hygienic testing.

The swabs and test cards of iHealth COVID-19 Antigen Rapid Test are sealed and independently packaged to ensure the safety and hygiene of the product.

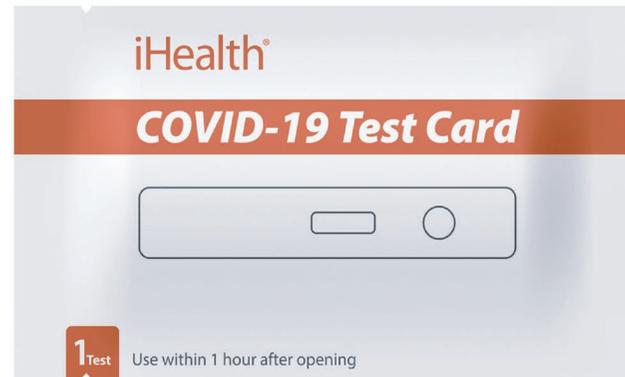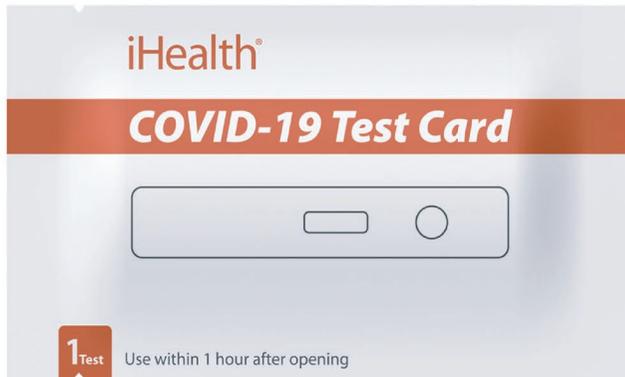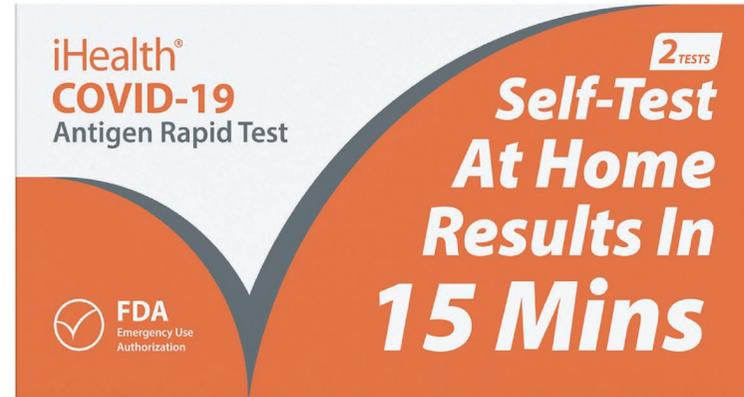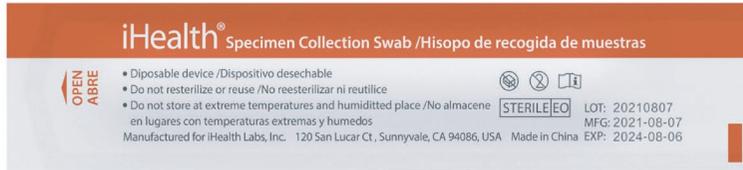

Self-test at home. Results in 15 minutes.

- This product has been authorized only for the detection of proteins from SARS-CoV-2, not for any other viruses or pathogens; and,
- The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

Disclaimer: If you're currently experiencing COVID-19 symptoms, please contact your primary care physician (PCP) for immediate guidance.

iHealth® COVID-19 Antigen Rapid Test may take several days to ship. This is not a PCR test and not valid for travel. The manufacturer of the iHealth® COVID-19 Antigen Rapid Test requires test kits be stored at temperatures between 36° and 86°F (2° -3°C). Any extended exposure of the test kit to temperatures beyond this range may affect quality of test results. If you have ordered home delivery of the test kit and expect temperatures in your area to be outside of this range at time of delivery, it is recommended that you arrange for someone to be home at or near time of delivery to receive the test kit.