Intelli-Flo Polar Pad Ankle FITTING INSTRUCTIONS

for Patients and Medical Professionals

⚠ WARNING

The Intelli-Flo Polar Pad can be cold enough to seriously injure the skin. The Intelli-Flo Polar Pad is non-sterile and for single patient use only. Do not attempt to sterilize the Pad. Follow these instructions and the Operating Instructions and Product Inserts provided with the Polar Care Kodiak Motorized cold therapy units (available at www.breg.com or 1-800-321-0607).

1. **Discuss treatment with your licensed health care practitioner.** Provide a complete medical history including any reactions to cold. Certain medical conditions make cold-induced injury more likely. Ask your practitioner about potential adverse reactions and cold-induced injuries.
2. **Use only according to your practitioner's instructions** regarding the frequency and duration of cold application and breaks, proper temperature, how and when to inspect the skin, and total length of treatment. Federal law restricts this device to sale by or on the order of a licensed health care practitioner.
3. **Apply an insulation barrier** (such as BREG Polar Dressing, Webril, Kerlix, cast padding, elastic bandage) that provides complete coverage between the Intelli-Flo Polar Pad and skin. **Do not let any part of the Intelli-Flo Polar Pad touch skin.** If a sterile dressing has been applied to the treatment site that does not completely cover the skin under the pad, use an additional insulation barrier. The Intelli-Flo Polar Pad alone is too cold to be applied directly to the skin.
4. **Apply the Intelli-Flo Polar Pad** with the gray plastic channeled side towards the skin. Refer to the graphics below. Do not wrap pad so tightly as to restrict normal circulation, as this can cause injury.
5. **Connect the Intelli-Flo Polar Pad** couplings to the Polar Care Kodiak unit couplings. The Intelli-Flo Polar Pad hose couplings are only compatible with the Polar Care Kodiak unit couplings.
6. **Follow the Polar Care Kodiak Operating Instructions and Product Insert** inside the pouch; both are affixed to the unit.
7. **Inspect the skin** under the Intelli-Flo Polar Pad (by lifting the edge) as prescribed, typically every 1 to 2 hours. Do not use the Polar Care Kodiak if dressing, wrapping, bracing, or casting over the Intelli-Flo Polar Pad prevents skin inspections. Stop using and contact your practitioner immediately if you experience any adverse reactions, such as: increased pain, burning, increased swelling, itching, blisters, increased redness, discoloration, welts, other changes in skin appearance, or any other reaction identified by your practitioner.
8. **To end cold therapy,** disconnect electrical power and press the two metal tabs on the hose coupling and gently pull apart or remove the Intelli-Flo Polar Pad from your body.

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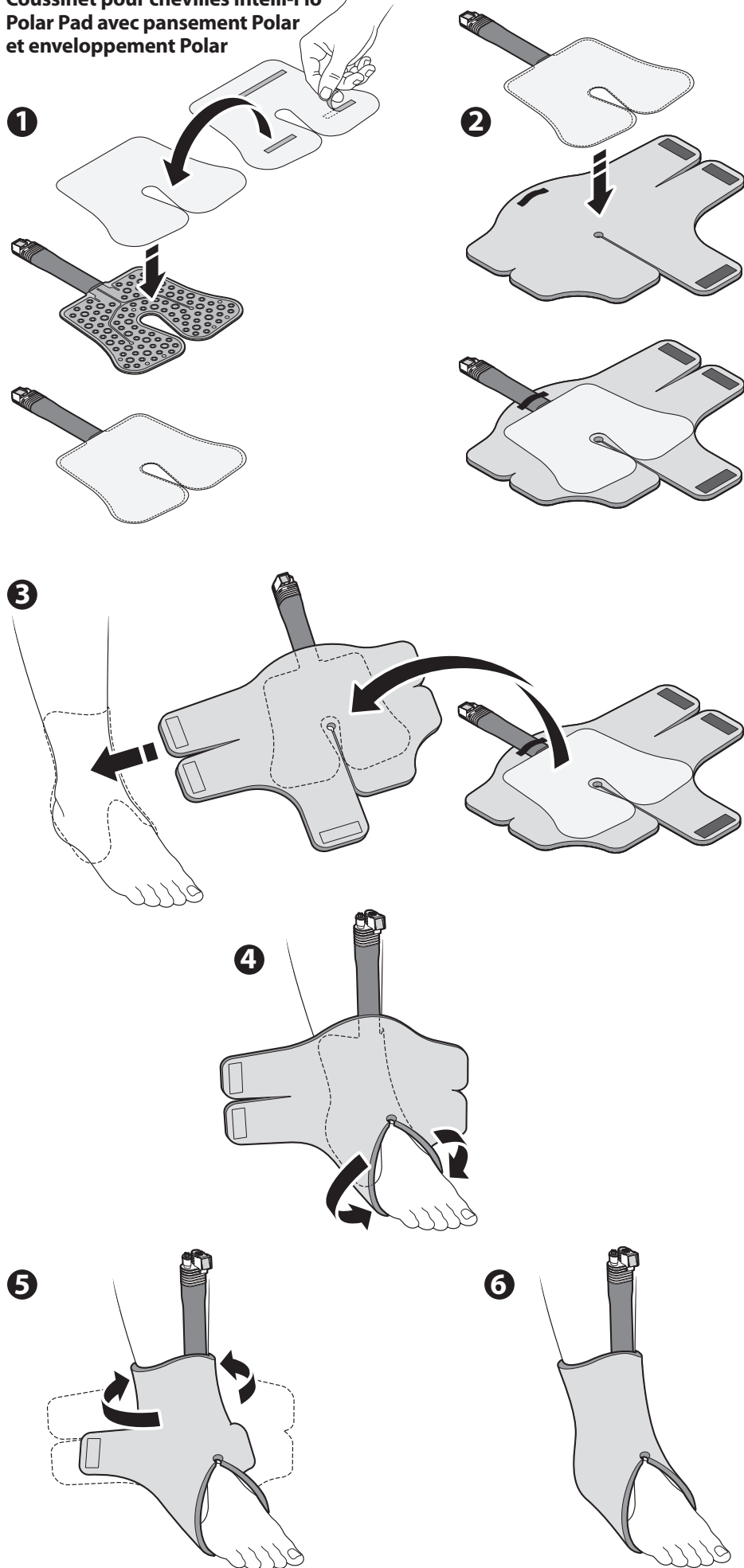
Intelli-Flo Polar Pad Ankle with Polar Dressing and Polar Wrap

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Intelli-Flo Polar Pad Ankle with Polar Dressing and elastic bandage

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BREG Inc.
Vista, CA 92081 U.S.A.