

Clorox Healthcare[™] Portfolio of Bleach Germicidal Cleaners



Ready-to-Use Bleach Products Most Trusted by U.S. Hospitals

Meet the standards of your healthcare peers and leading institutions:

- 1:10 bleach dilution
- Meet CDC and APIC guidelines for surface disinfection of the toughest healthcare pathogens
- Used by 5 of U.S. News and World Report's Top 10 Most Influential Hospitals
- One-step cleaning and disinfecting; no precleaning required
- Premixed and ready to use, ensuring proper concentration every time

Fast-acting and effective

Clorox Healthcare™ Bleach Germicidal Wipes

are EPA-registered to kill 51 microorganisms, including:

- C. difficile spores in 3 minutes*
- Representatives of ESKAPE pathogens in 30 seconds
- Norovirus in 1 minute
- TB in 3 minutes

Clorox Healthcare™ Bleach Germicidal Cleaners

are EPA-registered to kill 42 microorganisms, including:

- C. difficile spores in 5 minutes*
- Representatives of ESKAPE pathogens in 1 minute
- Norovirus in 1 minute
- TB in 30 seconds

*Use as directed on precleaned hard, nonporous surfaces.

Efficiency and value

Our portfolio of bleach germicidal cleaners helps your team do the job fast and do the job right.

Premixed and Ready to Use

Clorox Healthcare™ Bleach Germicidal Cleaners eliminate the labor costs associated with mixing and labeling bleach concentrates because they are premixed, ready to use and stable until the expiration date on the package.

• One-Step Cleaner Disinfectant

Clorox Healthcare™ Bleach Germicidal Cleaners contain detergent, so they clean and disinfect in just one step; no precleaning is required.

Versatile

May be used for instrument precleaning and disinfection of noncritical instruments. Can be applied directly onto surfaces, or Clorox Healthcare TM bleach sprays may be applied with a clean cloth, mop or sponge.

Broad Surface Compatibility

Clorox Healthcare™ Bleach Germicidal Cleaners contain an anticorrosive ingredient that makes them less damaging to surfaces and equipment than conventionally prepared bleach solutions.

Designed for user and patient comfort

Clorox Healthcare™ Bleach Germicidal Cleaners are made with a proprietary odor-masking formula for patient and staff comfort. There are several reasons your team can use bleach with confidence.

- 1. Multiple leading hospital studies noted that when patients and Environmental Services personnel were exposed to bleach, little or no concern about the odor of bleach was expressed.¹
- 2. Bleach's odor is caused by the chemical reaction that occurs when bleach begins to break down proteins, such as those that make up infection-causing pathogens.
- 3. When bleach is used as directed, the chlorine species emitted are lower than recognized levels that cause respiratory irritation and are below the EPA's occupation exposure limits for chlorine.

Broad-spectrum surface compatibility

Clorox Healthcare™ Bleach Germicidal Cleaners are multipurpose, broad-spectrum cleaner disinfectants for use on hard, nonporous surfaces. To minimize hypochlorite's impact on surfaces, they are specially formulated with an anticorrosion agent and are compatible on the following:

Surfaces:

fiberglass
 (sealed) surfaces
 glass

 glased porcelain
 hard plastics
 laminated plastic
 glazed ceramics

 glazed porcelain
 porcelain
 stainless steel
 countertops

Medical:

autoclaves respiratory mammography equipment equipment bed railings phlebotomy traysstretchers blood glucose monitors plastic mattress tables covers carts ultrasound transducers reception dialysis machines counters wash basins exam tables wheelchairs IV poles

Laboratory:

autoclavescountertops

Dental:

chairslight lens coverscountersoperatory hardgurneyprobesprobes

Veterinary:

animalcagesclinic surfacesequipmentfountains

General Use:

keyboards

doorknobs

- changing tablesdeskshighchairsdesktoplight switches
 - light switchesplaypens
- telephonestoiletstrash cans

sinks

To ensure that the salt left behind by bleach does not build up and cause surface damage over time, wipe down these surfaces after disinfecting with a fresh, clean, damp cloth.

Use as directed on hard, nonporous surfaces. For any use site that includes electronics, refer to manufacturers' instructions before use.

^{1.} Patient and Environmental Service employee satisfaction and tolerance of using germicidal bleach wipes for patient room cleaning. Presentation by Kimberly Aronhalt, BSN, RN, Infection Preventionist, Mayo Clinic, Rochester, Minn., at APIC, New Orleans, July 13, 2010.

Convenient forms to meet your needs

 $Clorox\ Healthcare^{\tau_M}\ Bleach\ Germicidal\ Cleaners\ are\ ready\ to\ use\ in\ convenient\ liquid\ and\ wipe\ forms.\ No\ mixing\ or\ labeling\ of\ bleach\ solutions\ is\ required.$

CLOROX HEALTHCARE™ BLEACH GERMICIDAL WIPES						
CLINICAL SURFACES	MULTIPURPOSE	TERMINAL CLEANING				
6" x 5" wipes	6.75" x 9" wipes	12" x 12" wipes				
The Control of Control	HAAD HAAF BAA GO MACAD W 90 Water and the second of the	MEAN STOLE A.E. Brack Germicidal Wight The Control of the Contro				
DESIGNED FOR: Nurses	DESIGNED FOR:	DESIGNED FOR: Environmental Services				
	Nurses, Environmental Services					
 PERFECT FOR: Equipment disinfection: Everyday disinfection of clinical surfaces such as glucometers and BP monitors by nurses. 	 PERFECT FOR: Versatility: Disinfection and cleaning of high-touch surfaces and medical equipment. 	PERFECT FOR: Terminal cleaning				
 High frequency: High consumption and frequent usage. 	 Spot cleaning and spills 	 Reducing labor costs and preventing spills: Ready-to-use application eliminates need for formula mixing. 				
 Small surfaces: Disinfection and cleaning of high-touch surfaces and medical equipment. 	 Daily patient room cleaning: Safely disinfect carts, wheelchairs, countertops, desktops, bed rails and dialysis machines. 	• Large surfaces: Such as patient beds, counters, windowsills, table tops, chairs, toilets, showers and sinks.				

CLOROX HEALTHCARE™ BLEACH GERMICIDAL CLEANERS							
TRIGGER SPRAYS	PULL-TOP	REFILLS					
22 fl. oz., 32 fl. oz.	32 fl. oz.	64 fl. oz., 128 fl. oz.					
THE REAL PROPERTY OF THE PROPE	Taranta Caranta Carant	THE STATE OF THE S					
PERFECT FOR: Convenient application: Trigger sprays offer quick and easy surface coverage.	PERFECT FOR: • Direct application: Should be directly applied to hard, nonporous surfaces. Compatible with microfiber cloths.	PERFECT FOR: • Easy refilling: Economical refill bottles minimize packaging waste and meet the needs of heavy users					
• Large surfaces: Efficient coverage of large surfaces such as countertops, gurneys, stretchers, plastic mattress covers, shower stalls and exam tables.	 Patient care areas: Offers broad, direct application for minimized aerosolization. 	Safe dispensing: A vented spigot allows for easy refill of trigger spray or pull-top bottles with no spillage, as well as into bucket as is sometimes needed in Environmental Services.					

Clorox Healthcare™ Clorox Healthcare™ **Bleach Germicidal** Bleach Germicidal

Cleaners

Wipes

Clorox Healthcare™ Bleach Germicidal Cleaners

		Bleach Germicidal Wipes	Clorox Healthcare™ Bleach Germicidal Cleaners
	Effective against:	Kill Time	Kill Time
	Acinetobacter baumannii	30 sec	1 min
	Bordetella pertussis	30 sec	1 min
	Campylobacter jejuni	30 sec	1 min
	Clostridium difficile spores	3 min	5 min
	Enterobacter aerogenes	30 sec	1 min
	Enterobacter cloacae	30 sec	
	Enterobacter cloacae NDM-1		1 min
	Vancomycin-resistant Enterococcus faecalis (VRE)	30 sec	
	Multidrug-resistant Enterococcus faecium	30 sec	1 min
	Vancomycin-resistant Enterococcus faecium (VRE)		1 min
	Escherichia coli	30 sec	
	Escherichia coli 0157:H7	30 sec	
	ESBL-producing Escherichia coli	30 sec	1 min
	Escherichia coli NDM-1	30 sec	1 min
	Klebsiella pneumoniae	30 sec	1 min
'	ESBL-producing Klebsiella pneumoniae	30 sec	1 min
	Carbapenem-resistant Klebsiella pneumoniae	30 sec	1 min
ro	Klebsiella pneumoniae NDM-1	30 sec	1 min
Bacteria	Legionella pneumophila	30 sec	
Вас	Listeria monocytogenes	30 sec	
	Proteus mirabilis	30 sec	1 min
	Pseudomonas aeruginosa	30 sec	1 min
	Salmonella enterica	30 sec	1 min
'	Serratia marcescens	30 sec	1 min
	Shigella dysenteriae	30 sec	
'	Staphylococcus aureus	30 sec	1 min
	Linezolid-resistant <i>Staphylococcus aureus</i> (LRSA)	30 sec	
	Methicillin-resistant Staphylococcus aureus (MRSA)	30 sec	1 min
	Methicillin-resistant <i>Staphylococcus aureus</i> (CA-MRSA) (USA 300)	30 sec	
	Methicillin-resistant <i>Staphylococcus aureus</i> (CA-MRSA) (USA 400)	30 sec	
	Vancomycin-resistant Staphylococcus aureus (VRSA)	30 sec	1 min
	Vancomycin intermediate-resistant Staphylococcus aureus (VISA)	30 sec	1 min
	Stenotrophomonas maltophilia		1 min
Fungi, Vet	Streptococcus pneumoniae	30 sec	1 min
	Streptococcus pyogenes	30 sec	1 min
	Candida albicans	3 min	1 min
	Canine Parvovirus	3 min	1 min
	Feline Panleukopenia Virus	3 min	1 min
	Trichophyton mentagrophytes (Athlete's Foot Fungus)	3 min	1 min
TB	Mycobacterium bovis	3 min	30 sec

	Effective against:	Kill Time	Kill Time	
	Adenovirus Type 2	1 min		
	Avian Influenza A Virus	1 min	1 min	
	Hepatitis A Virus (HAV)	1 min	1 min	
	Hepatitis B Virus (HBV)	1 min	1 min	
	Hepatitis C Virus (HCV)	1 min	1 min	
	Herpes Simplex Virus (HSV-2)	1 min	1 min	
Viruses	HIV-1	30 sec	1 min	
	Human Coronavirus	1 min		
	Influenza A Virus	1 min	1 min	
	Influenza A Virus (H1N1)		1 min	
	Norovirus*	1 min	1 min	
	Poliovirus	1 min	1 min	
	Respiratory Syncytial Virus (RSV)	1 min		
	Rhinovirus	1 min	1 min	
	Rotavirus	1 min	1 min	
	TOTAL:	51 Claims	42 Claims	
Claray Haaltheara™ Product Information				

Clorox Healthcare™ Product Information

Bleach Germicidal Wipes

EPA Reg. No. 67619-12

6" x 5" Clinical Wipes,

6/150 ct., Item No. 30577 6.75" x 9" Multipurpose Wipes,

6/70 ct., Item No. 35309 12" x 12" Terminal Wipes,

2/110 ct., Item No. 30358

12" x 12" Terminal Wipes Refill.

2/110 ct., Item No. 30359



Bleach Germicidal Cleaners

EPA Reg. No. 56392-7

6/32 fl. oz. Spray, Item No. 68970

6/32 fl. oz. Pull-Top, Item No. 68832

8/22 fl. oz. Spray, Item No. 68967

6/64 fl. oz. Refill, Item No. 68973

4/128 fl. oz. Refill, Item No. 68978



Use as directed on hard, nonporous surfaces.

* Obtained via surrogate.

For more information, contact your Clorox sales representative or call 800-234-7700. email: healthcare@clorox.com visit us: www.cloroxhealthcare.com

© 2012 Clorox Professional Products Company, 1221 Broadway, Oakland, CA 94612.

