

MINIMIZE
RISK OF SKIN INJURY
MAXIMIZE
PATIENT SAFETY

DETACHOL® ADHESIVE REMOVER BOTTLE APPLICATION INSTRUCTIONS

Medical Adhesive-Related Skin Injury (MARSII) is an under-recognized but serious complication that contributes to negative patient safety and satisfaction¹

Minimize the risk of skin injury and discomfort associated with removal of dressings, tapes, and sticky residue from the skin with DETACHOL

Follow facility protocol or provider instructions for device/dressing change procedures.

1 Saturate

- > Saturate a cotton swab or gauze with Detachol.

2 Apply

- > Apply Detachol, beginning at the edge of the tape or device.

3 Remove


- > Once the corner is permeated, gently lift up, continuing to apply Detachol while gently lifting the tape or device until the adhesive is completely removed.

4 Cleanse

- > Wash the area with soap and water or skin cleanser.



detachol[®]
ADHESIVE REMOVER

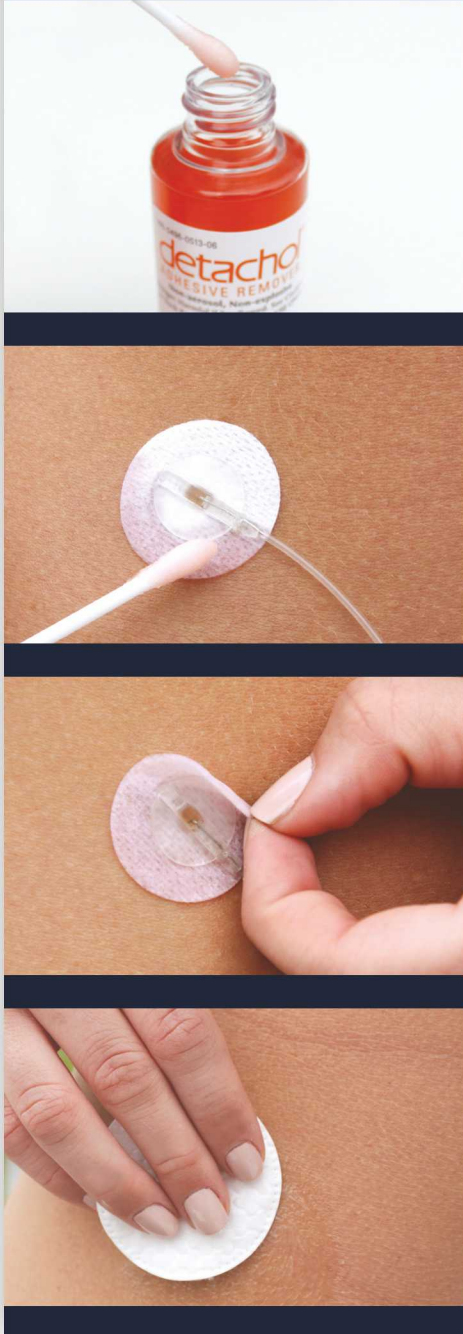

Warnings: For external use only. Contains: Specially Refined Paraben Hydrocarbons. Caution: Flammable. Keep out of reach of children. If swallowed, do not induce vomiting. Call physician immediately.

For more information, please contact your sales consultant or Eloquest Healthcare, Inc.

 877-433-7626  www.detachol.com

Eloquest
HEALTHCARE™