

# detachol®

## ADHESIVE REMOVER


When you need it off quickly,  
painlessly, safely, and entirely.




---

Detachol is a non-irritating adhesive  
remover used to aid in the removal  
of dressings, tapes, devices and  
sticky residue from the skin.

---

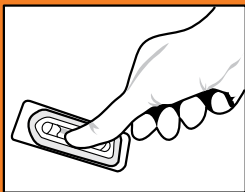

1 CRACK

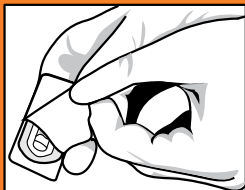

2 PEEL



3 APPLY

## 3 EASY STEPS FOR PAINLESS REMOVAL


Place an individual vial in the blister-pak on a hard surface (label facing down) and gently apply pressure to crush the inner ampule, releasing Detachol.

Peel back the lid and remove the individual vial using sterile or aseptic technique.


By gently squeezing the vial, apply Detachol to the corner of the tape or dressing. Once the corner is saturated, carefully lift the tape. Continue squeezing the vial, while gently pulling the tape or dressing at the same time. Continue application until adhesive is completely removed. Clean the area with mild soap and water. Avoid contact with non-intact skin.

**Warnings:** For external use only.  
**Contains:** Specially Refined Parafin Hydrocarbons  
**Caution:** Flammable. Keep out of the reach of Children.  
If swallowed, do not induce vomiting.  
Call physician immediately


## COMMON DETACHOL APPLICATION AREAS

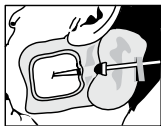

**CENTRAL  
LINE IV  
DRESSING  
REMOVAL**

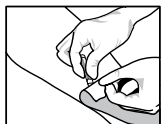

**NPWT &  
OTHER WOUND  
DRESSINGS  
REMOVAL**

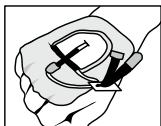

**PERIPHERAL  
IV DRESSING  
REMOVAL**

- DETACHOL CAN ALSO REMOVE:**
- **LEFTOVER ADHESIVE RESIDUE**
  - **ET TUBE ADHESIVE**
  - **NG TUBE ADHESIVE**
  - **CATHETER ANCHOR ADHESIVE**

**Eloquest**  
HEALTHCARE®

780 West Eight Mile Road, Ferndale, MI 48220  
877-433-7626 • [www.eloquesthealthcare.com](http://www.eloquesthealthcare.com)

Detachol is a registered trademark of Ferndale IP, Inc.  
DE04 R4/12