


Shower Transport Chair with Rear Wheel Locks



Item#: 522-1702-1900

Assembly Instructions

1. Secure the rear brace (1) to both side supports (right & left) (2). Start by aligning the holes of both members and using a 1-3/4" screw (3) two washers (4) and a lock nut (5) on each side as shown. The washers are contoured to fit against the tubing. Tighten by hand.
2. Attach the front brace (7) to both side supports (2). Align the holes of both members. Insert a 2-1/4" screw (6) two washers (4) and a lock nut (5) into the upper and lower holes on one side as shown. The washers are contoured to fit against the tubing. Tighten by hand. Repeat steps above to secure the other side of the front brace.
3. Next, attach the backrest (8) to both side supports (2). Align the holes of both members. Insert a 2-1/4" screw (6) two washers (4) and a lock nut (5) into the upper and lower holes on one side as shown. The washers are contoured to fit against the tubing. Tighten by hand. Repeat steps above to secure the other side of the backrest.
4. Position the seat (9) over the rear (1) and front brace (7) then firmly push down until the clips lock into place on the tubing. **MAKE SURE THE SEAT CLIPS ARE FIRMLY ATTACHED TO THE CHAIR FRAME.**
5. Finally, tighten all screws securely using a Phillips screwdriver and pliers or wrench. **DO NOT OVER TIGHTEN AS IT COULD STRIP THE THREADS.**

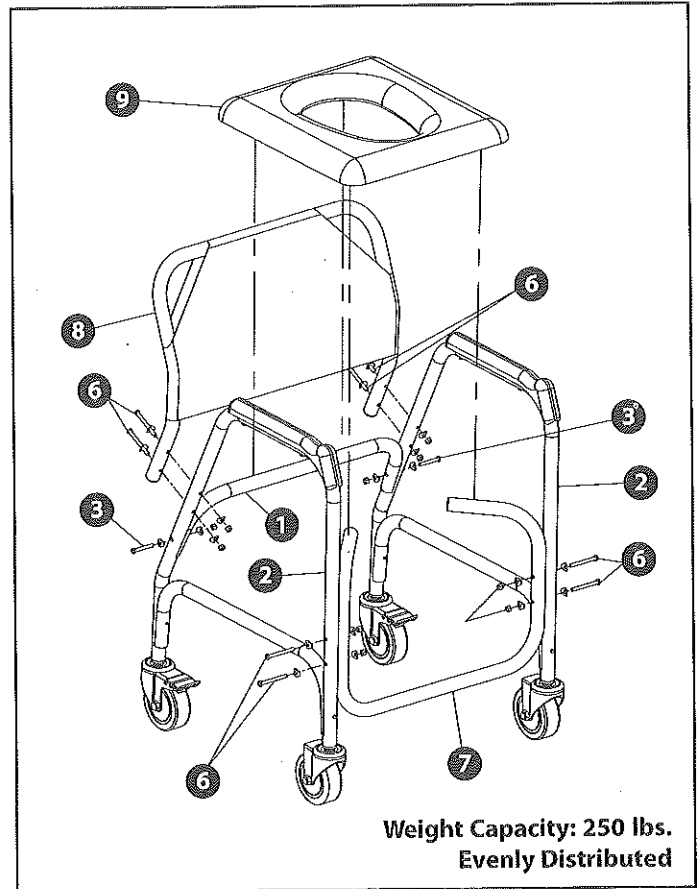

WARNING

WARNING: HealthSmart® International assumes no responsibility for any damage or injury caused by improper assembly or use of this product.

Make sure all screws and nuts are tightened securely. For your safety, the rear wheels should always be in the locked position when ever possible. Failure to do so can cause serious injury.

This product should not be used without the advice or instruction from an authorized healthcare professional.

To help prevent falls and serious injury, persons having limited physical capabilities should be assisted or supervised at all times.

All wheels **MUST** be on a flat surface at all times during use.

Do not exceed the maximum weight capacity of 250 lbs. (evenly distributed).

Do not continue to use this product if any of the components are missing or damaged.

Maintenance

1. Before each use, check to make sure that all four seat clips are securely attached to chair frame.
1. Periodically check chair assembly to make sure all screws and nuts are tightened securely.
2. To clean, use a mild soap and water solution, rinse with clean water, and dry with a soft cloth. Never use solvents or abrasive cleaners on any part of the chair.

Assembly Code:

- 1 - Rear Brace
- 2 - Side Supports (Right & Left)
- 3 - Screws, 1-3/4" (2ea.)
- 4 - Washers (20ea.)
- 5 - Nylon Lock Nuts (10ea.)
- 6 - Screws, 2-1/4" (8ea.)
- 7 - Front Brace
- 8 - Backrest
- 9 - Seat

Note: If any parts are missing, do not return to the place of purchase. Contact us at 877-505-6190.

Tools Needed For Installation:

Phillips screwdriver and pliers or wrench.

LATEX Contains Latex