

ALBAHEALTH® DRESSING RETAINERS INSTRUCTIONS FOR USE



Albahealth® dressing retainers are made of the highest quality latex-free materials to improve comfort, promote compliance and insure the best possible outcome for all patients.

ALBAHEALTH® DRESSINGS AND DRESSING RETAINERS




CareNett® and X-Span® Dressing Retainers are designed to keep wound dressings in place for difficult to tape areas of the body as well as for patients sensitive or allergic to medical tape. Both brands provide a softer and more comfortable alternative to keeping dressings in position and feature greater stretch to accommodate the full patient population.

Additional features include:

- Reduced linting
- Available in full 25 YD rolls
- Soft and comfortable
- Latex-free

Application is quick and easy:

- Cut the length of product needed from the convenient dispenser box
- Position dressing in place
- Place hands inside open end of dressing retainer and stretch open
- Place product over body part
- Place index fingers in netting where openings are needed
- Pull apart netting creating openings to fit appendages, limb, etc.

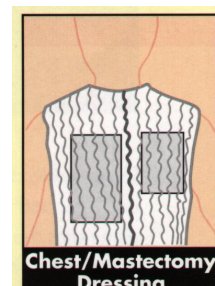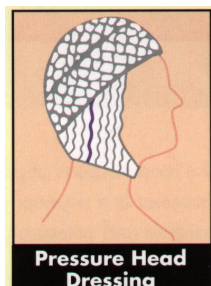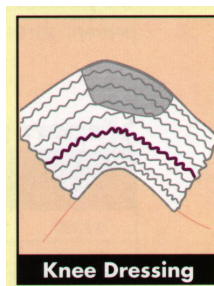

CARENETT	X-SPAN	SIZE	DESCRIPTION	QTY/CS
CN01	821	1	Medium - Fingers/Toes	1 RL
CN02	822	2	Large - Fingers/Multiple	1 RL
CN03	823	3	Small & Medium - Hand/Ankle All - Wrist	1 RL
CN04	824	4	Small - Lower Thigh/Knee/Calf Small & Medium - Neck/Upper Arm Medium - Elbow/Forearm/Foot Large - Hand/Ankle	1 RL
CN05	825	5	Medium & Large - Lower Thigh/Knee/Calf Large - Neck/Upper Arm	1 RL
CN06	826	6	Small - Wrist All - Head/Upper Thigh	1 RL
CN07	827	7	Small & Medium - Hips/Bust/Chest/Axilla Medium & Large - Waist	1 RL
CN08	828	8	Large - Hips/Bust/Chest/Axilla	1 RL
CN09	830	9/10	X-Large - Hip/Bust	1 RL



Better care starts
with safety and comfort.

Albahealth is a subsidiary of Encompass Group LLC
800.284.4540 | www.encompassgroup.net

© 2010 Encompass Group, LLC
All Rights Reserved
Lit. No. AC10176 REV. 8/10