

Easy Set Up

1: Powering on your Device

Turn the unit over and open the battery lid. If locked, insert the key into the lock and turn the key clockwise to unlock the battery compartment. Insert 4 AA batteries. This should power on the unit right away.

Note: The XL unit uses either batteries or an optional power supply card that you can purchase separately.



2: Open the Device & Setting Time Format

Press the front latch button to release and open the lid. If the lid doesn't open, ensure it isn't locked by inserting the key inside the lock and turning the key clockwise until it unlocks. The unit comes in 24-hour format. To set up, change, or toggle between 12-hour or 24-hour format press and hold buttons 2 and 3 simultaneously until the format changes to the desired one. If you would like to change it back, repeat the process..



3: Set the Time

Step 1: Hold button 1 until hours blink (about 3 seconds).

Step 2: Press button 2 to decrease the hour or button 3 to increase the hour.

Step 3: Press button 1 to move to the minutes while the hour is still blinking. Press button 2 to decrease the minutes or button 3 to increase the minutes.

Step 4: Hold button 1 to exit the setup.



4: Setting up Alarms

There are 12 alarms available labeled A1 to A12.

Step 1: Press button 1 to enter setup (do not press and hold, just quickly tap) for the first alarm which is A1.

Step 2: Press button 2 to decrease the hour or button 3 to increase the hour.

Step 3: Press button 1 to move to the minutes while the hour is still blinking. Press button 2 to decrease the minutes or button 3 to increase the minutes.

Button 1 cycles through each alarm (A1-A12). Keep pressing to reach the desired alarm. To set additional alarms (A2-A12), repeat steps 2 and 3 on the desired alarm.

Note: To exit alarm setup, tap button 1 until you see current time displayed.



5: Selecting Alarm Tone

The unit comes with three different alarms and a silent mode. To setup, change, or toggle between each of them, press and hold **buttons 1 and 3 simultaneously** for three seconds until the tone changes to the desired one. If you would like to change it to a different tone, repeat the process.

Note: If no tone is heard, the unit is on silent mode. Press and hold buttons 1 and 3 to advance to the next alarm tone.



6: Voice Over Alarm (Optional)

Tap button 1 to enter the alarm setup menu. Scroll through alarms A1-A12 by tapping button 1. Keep tapping button 1 until the desired alarm is reached for voiceover recording. Once the alarm is selected, press and hold button 1 for 5 seconds until the screen becomes blank. Record the desired voiceover while screen is blank. When finished, press button 1 again to set voiceover. Repeat the process for other alarms, if desired.

**To exit alarm setup, tap button 1 until current time is displayed.*

Note: When finished, you should see a microphone symbol next to the set alarm on the main menu.

