


BD E-Z Scrub™ Preoperative Surgical Scrub Brush

4% chlorhexidine gluconate solution

SKU: 371073

Directions for use

- 1 Wet hands and forearms to the elbows with warm water
- 2 Use nail cleaner and apply scrub solution from sponge side
- 3 Work up lather
- 4 Scrub nails, cuticles and interdigital spaces thoroughly for 3 minutes with brush side and hands and forearms with the sponge side
- 5 Rinse with warm water
- 6 Repeat scrub for 3 more minutes, use the sponge side only. Add water as necessary to produce the desired level of suds
- 7 Rinse hands and arms thoroughly
- 8 Dry thoroughly

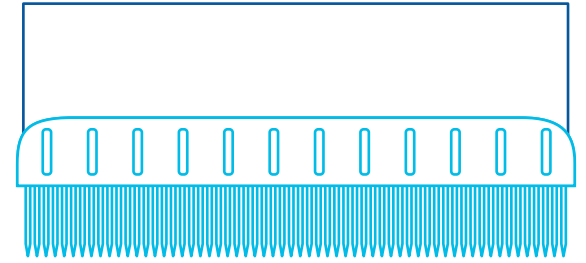

For more detailed instructions, please see the product Instructions for Use.

Drug Facts

Active ingredient 4% chlorhexidine gluconate w/v

Purpose Antiseptic

Use

Preoperative cleaning of healthcare personnel arms and hands.

Warnings

For external use only.

Do not use

- As a preoperative skin preparation of the face or head
- In eyes, ears and mouth
- If you have a sensitivity to chlorhexidine gluconate or its components

When using this product

- avoid contact with meninges
- chlorhexidine gluconate has been reported to cause deafness when instilled in the middle ear through perforated ear drums
- chlorhexidine gluconate solution has been reported to cause serious and permanent eye injury when it has been permitted to enter and remain in the eye during surgical procedures
- if chlorhexidine gluconate solution should contact these areas, rinse out promptly and thoroughly with water

Drug Facts (continued)

Stop use and ask a doctor if adverse reactions occur. Irritation, sensitization and generalized allergic reactions have been reported with chlorhexidine-containing products, especially in the genital areas.

If swallowed, gets in eye or ear, get medical help or contact a Poison Control Center immediately.

Other information

- avoid excessive heat 40 °C (104 °F)
- discard after single use

Inactive ingredients

FD&C red #40, hydrochloric acid, lauramide DEA, nonoxynol-12, peg-6000 distearate, peg-75 lanolin, purified water USP, sodium hydroxide

Questions?

1.800.453.4538 Monday to Friday, 8 a.m. to 5 p.m. MST

BD, Vernon Hills, IL, 60061, U.S.

bd.com

BD, the BD Logo and E-Z Scrub are trademarks of Becton, Dickinson and Company or its affiliates. © 2021 BD. All rights reserved. BD-24471 (0121) 5951

