Adapt [™]	REF 7917
---------------------------	-----------------

www.hollister.com/symbolglossary

	Hollister [®]
	©2020 Hollister Incorporated. 912921-620

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

Contents: Skin protective wipes
Product Description: Skin protective wipe. Alcohol free – no sting! Helps provide a protective film on the skin. May help prevent irritation associated with removal of adhesive.

Indications and Intended Use: This product is an accessory to an ostomy pouching system that provides a protective layer between skin and bodily wastes and fluids

INGREDIENTS: WATER, PVP, BUTYLENE GLYCOL, VP/VA COPOLYMER, ALOE VERA DECOLORIZED WHOLE LEAF EXTRACT, SODIUM HYALURONATE, OCTOXYNOL-9, PEG-12 DIMETHICONE, DISODIUM EDTA, DIAZOLIDINYL UREA, IODOPROPYNYL BUTYLCARBAMATE.

WARNINGS Reuse of a single use product may create potential risk to the user such as poor adhesion.

Precautions For external use only. Use only as directed. Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, and vagina). Discontinue use if irritation occurs. To be used on intact skin only. Avoid contact with eyes.

Users with any known sensitivities to any of the ingredients should not use this product.

Notes:

Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

INSTRUCTIONS FOR USE

Skin should be clean and dry prior to application of the Adapt skin protective wipe.

Apply a thin uniform coating. Let dry. Protective coating forms, which bonds to skin. Coating does not have to be removed from skin when changing appliance. If the Adapt skin protective wipe is used in an area with skin folds or other skin-to-skin contact, make sure the skin contact areas are separated to allow the coating to thoroughly dry before returning to normal position.

Please follow all local disposal regulations.

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

	Hollister [®]
	

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