


# IMPROSAFE

## SAFETY BLOOD COLLECTION SYSTEM

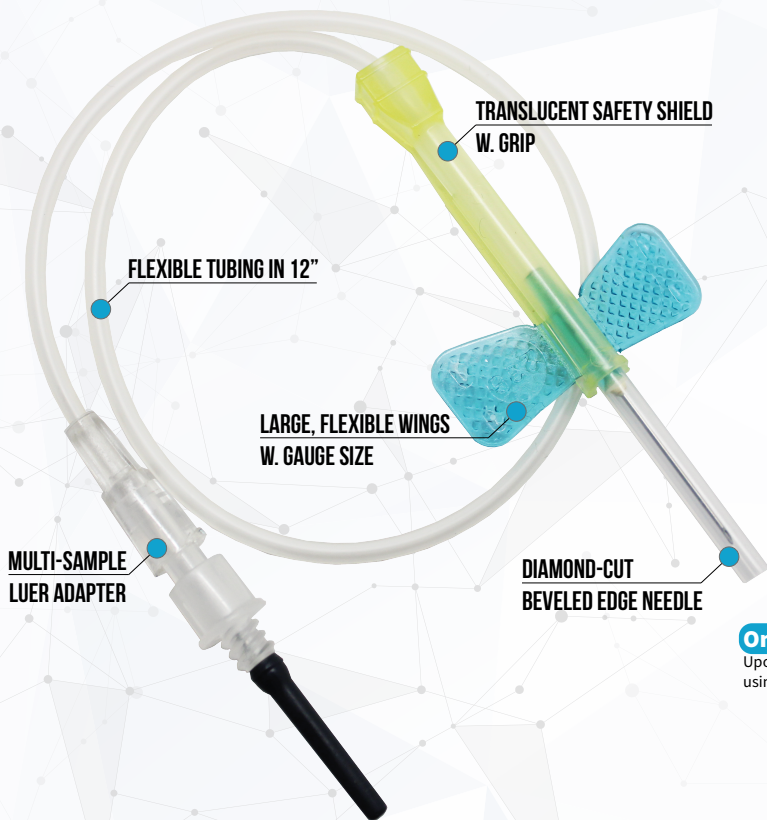

IMPROSAFE Blood Collection Set is a simple and easy to use set. It is a safety-engineered design protecting health-care workers from needlestick injury. It is available with pre-attached holder for ready to use situation, no assembly required after taking it out of the package.

### TWO METHOD PROCEDURE

#### One-Handed Technique

Upon completion of collection, apply light pressure to site using three fingers as shown

#### Two-Handed Technique

Upon completion of collection, apply light pressure to site using three fingers as shown

### Opening and Assembly Method



**1**  
Peel back packaging at arrow mark till the wing set be exposed.




**3**  
Twist and pull to remove needle cover.



**2**  
Screw holder to Luer Adapter. Firmly twist and press holder to lock male/Female Luer Adapter connector.



Withdraw safety blood collection set by holding the yellow safety shield grip area with the thumb and index finger.

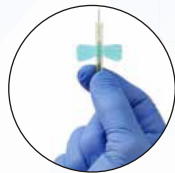


While holding the tubing securely with the other 3 fingers, push the yellow shield forward with thumb and index finger.



You will stop pushing the needle until the needle is completely covered and a click sound is heard., indicating that the shield is locked and in place over the needle tip.




Withdraw safety blood collection set by holding the yellow safety shield grip area with the thumb and index finger.

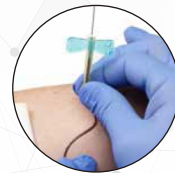

Withdraw safety blood collection set With the opposite hand, hold the tubing between thumb and index finger.



You will stop pushing the needle until the needle is completely covered and a click sound is heard., indicating that the shield is locked and in place over the needle tip.

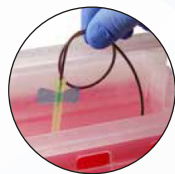

Discard immediately into an approved sharps container.



Discard immediately into an approved sharps container.

Product Code	Needle Gauge x Length	Tube Length	Wings Color	Quantity
IMP-21PE	21 G x 3/4"	12"	Green	Box: 100   Case: 2,000
IMP-21AH (w. Holder)				Box: 25   Case: 200
IMP-23PE	23 G x 3/4"	12"	Blue	Box: 100   Case: 2,000
IMP-23AH (w. Holder)				Box: 25   Case: 200
IMP-25PE	25 G x 3/4"	12"	Orange	Box: 100   Case: 2,000
IMP-25AH (w. Holder)				Box: 25   Case: 200