


BD Nexiva™
Closed IV Catheter System




Secure Connections

- **Vent plug**
 - Must remain in place during insertion for
 - Flashback and
 - Blood containment
- **BD Q-Syte™ Luer access split-septum**

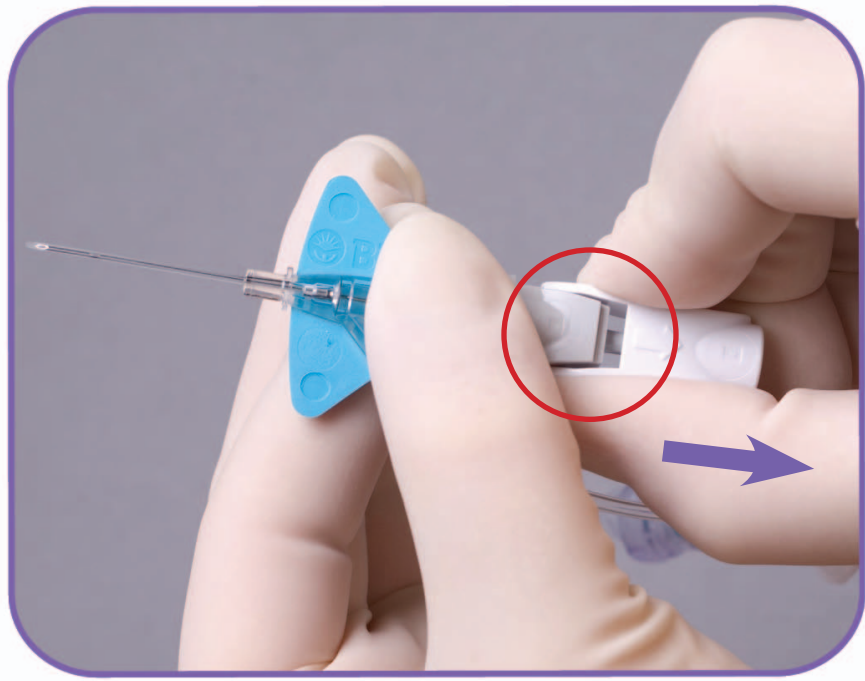

Release Seal



- Loosens catheter for easier advancement
- Pull back ~ 1/8 inch

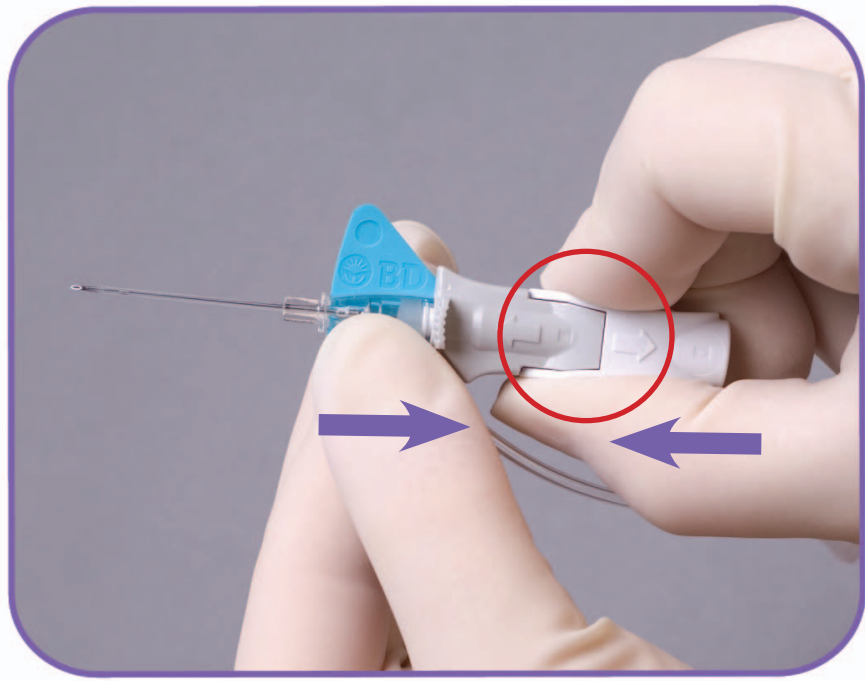


Colour to White Everything Tight

- Push coloured platform and white grip back together

Keep it *Sharp*

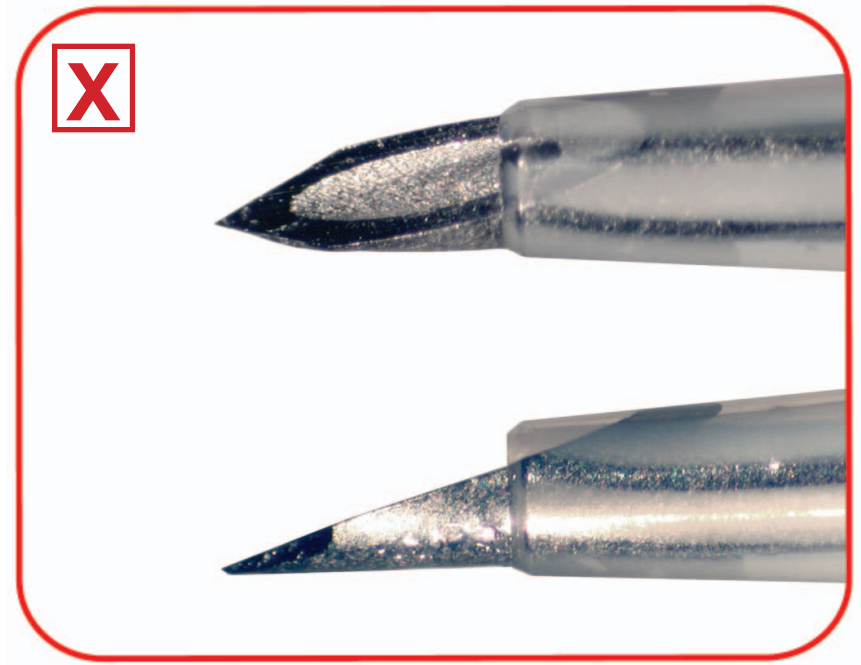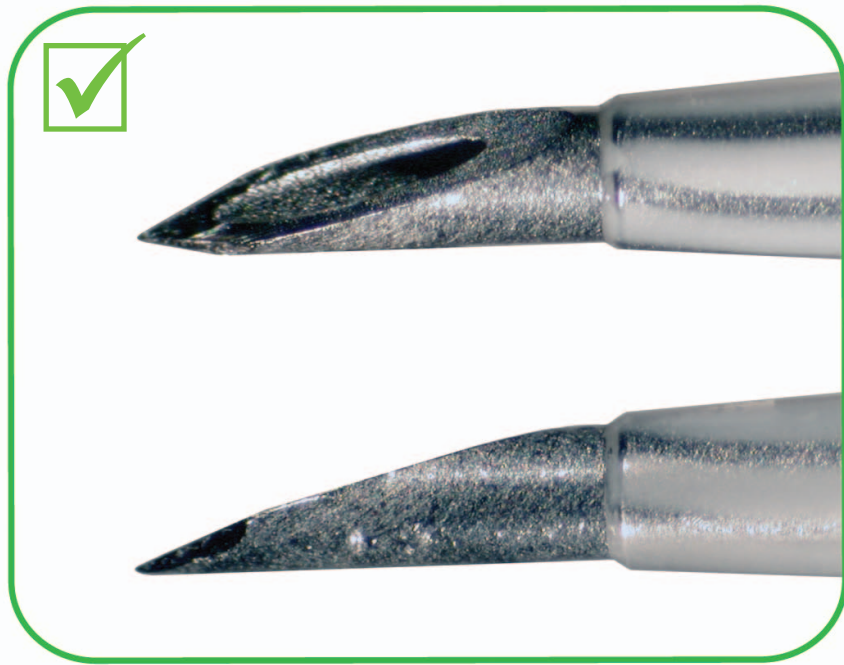


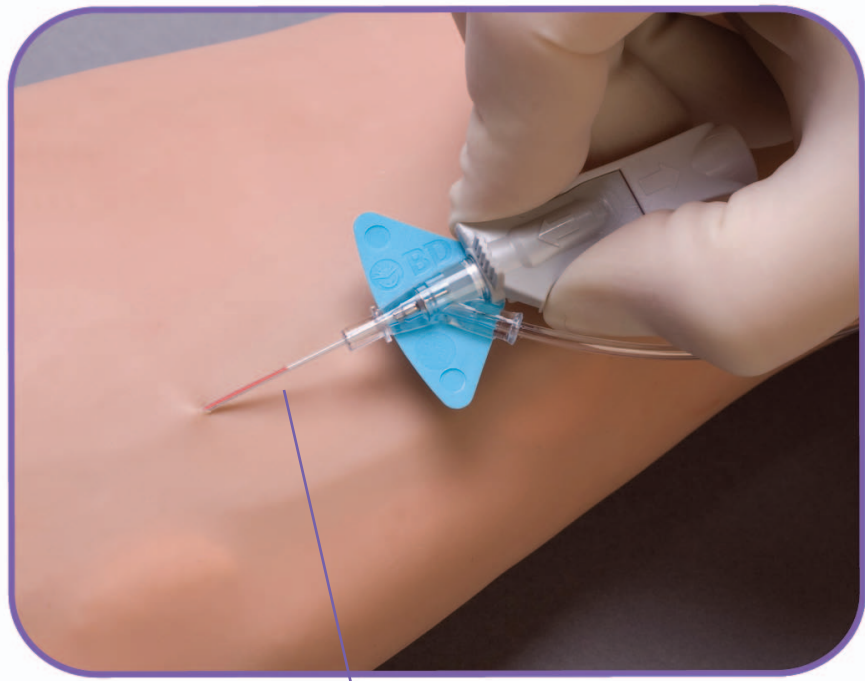

Initial Blood Return

BD Instaflash™ Needle Technology


- Immediate blood return is seen within the catheter

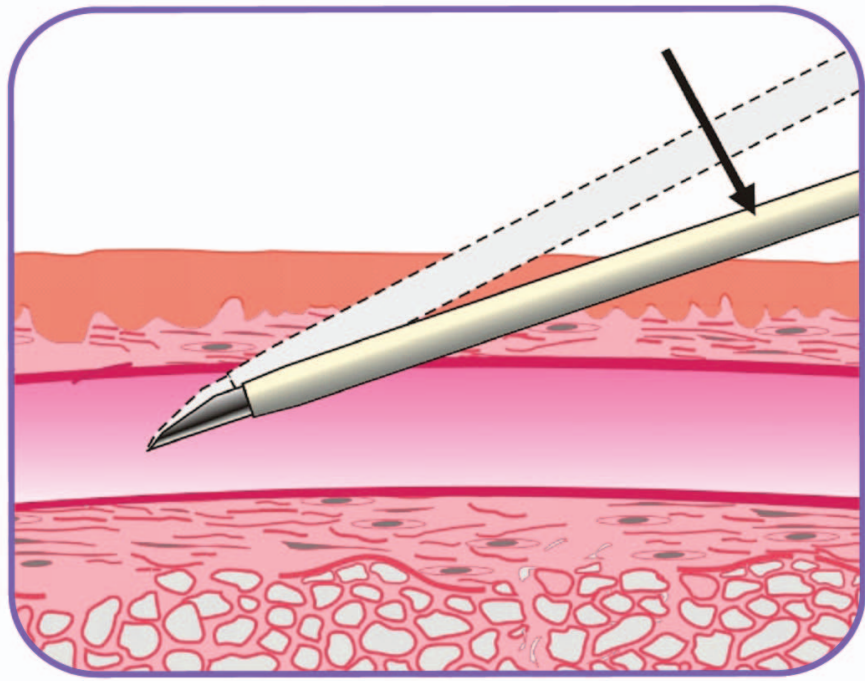

Lower and Advance

- Lower catheter almost parallel to skin
- Slightly advance entire unit to ensure catheter tip is within vessel


Forward First

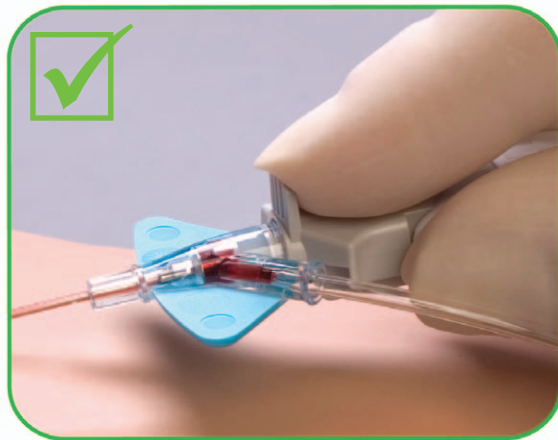

Pad to tab





Not fingernail

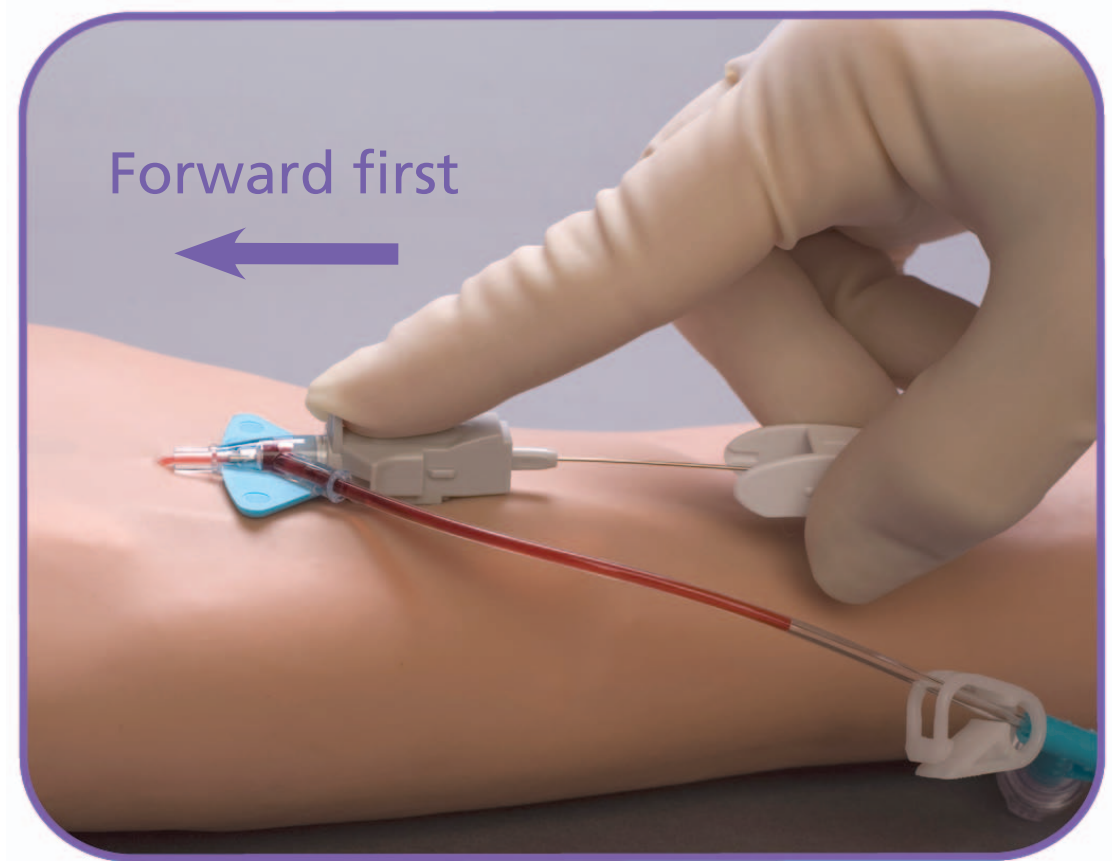

Don't pull back

Forward First

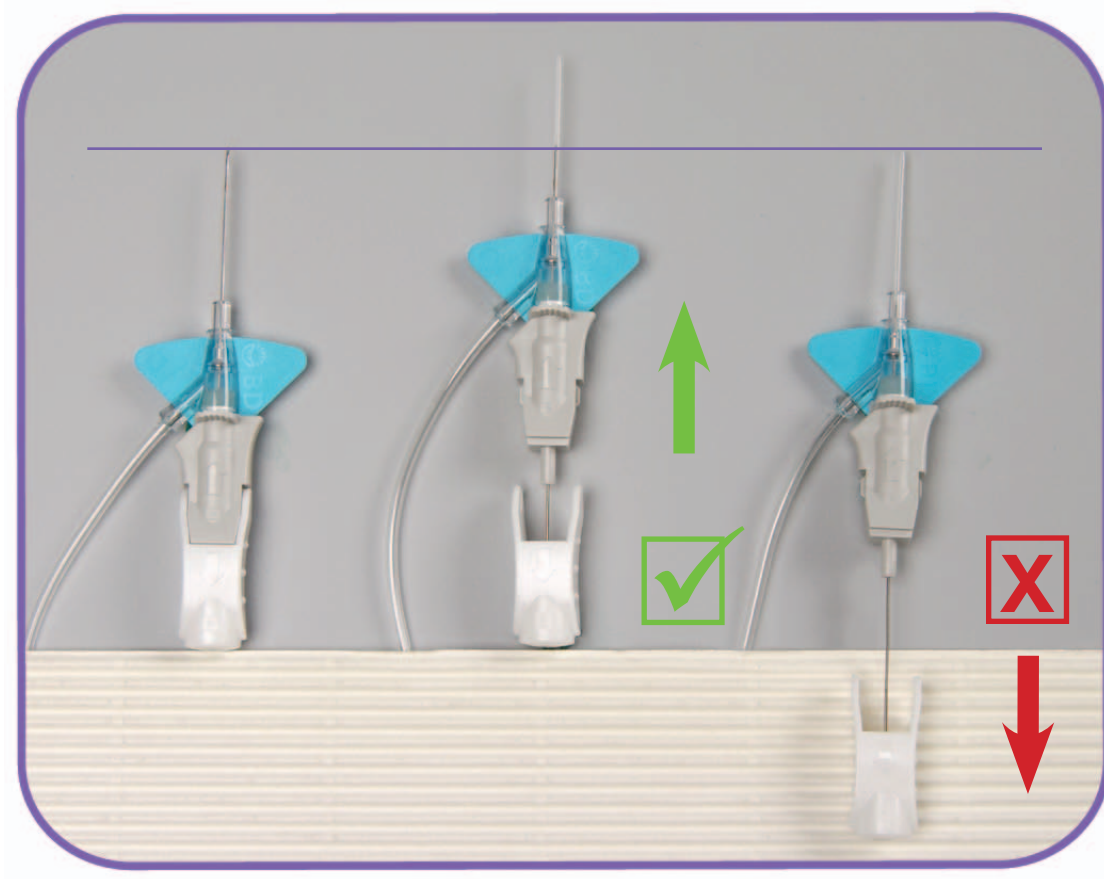


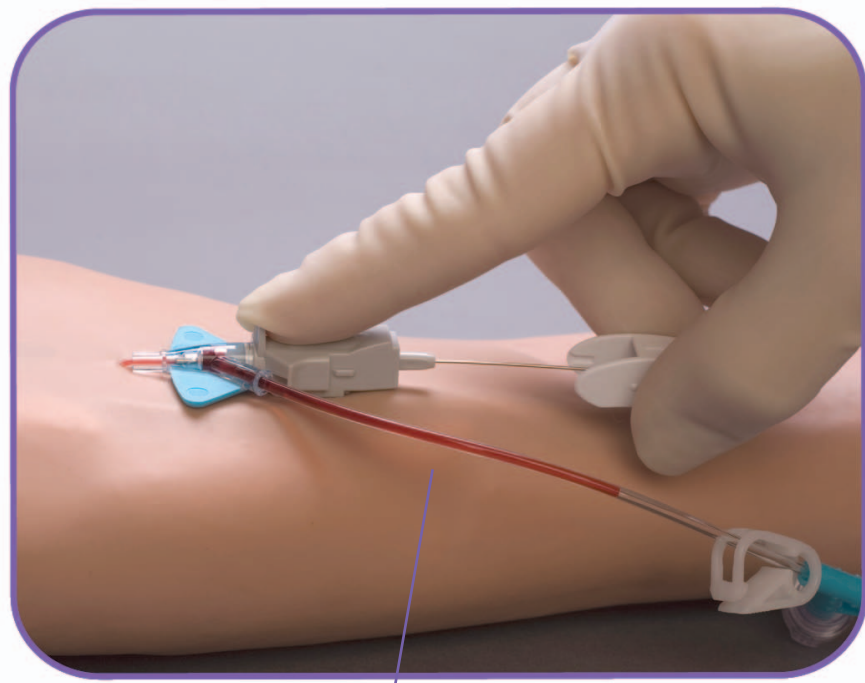



Blood Return

Secondary Flash

- Blood return continues up the extension
- Continuous flow confirms the catheter tip is within the vessel lumen
- Quickens as needle is withdrawn

Stabilize & Remove

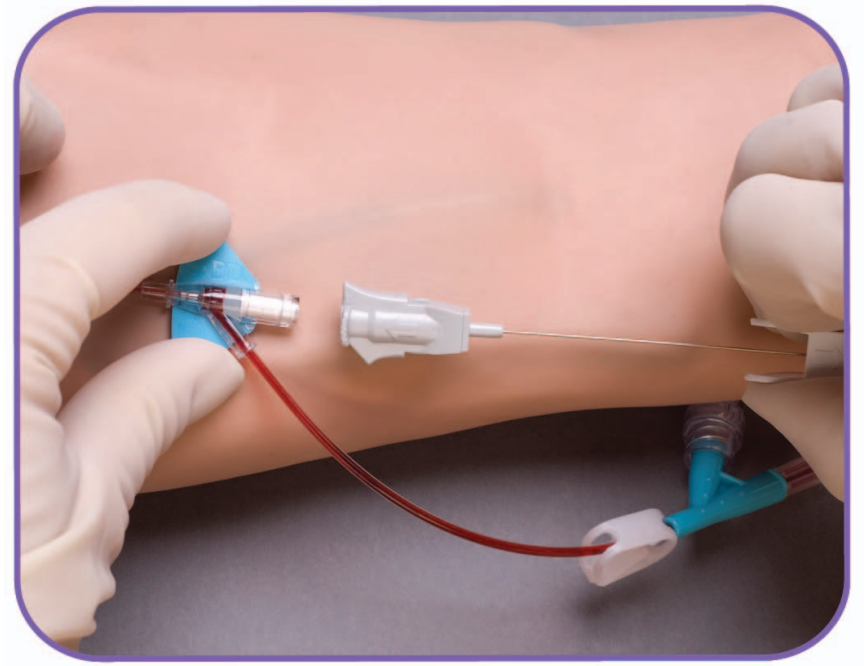
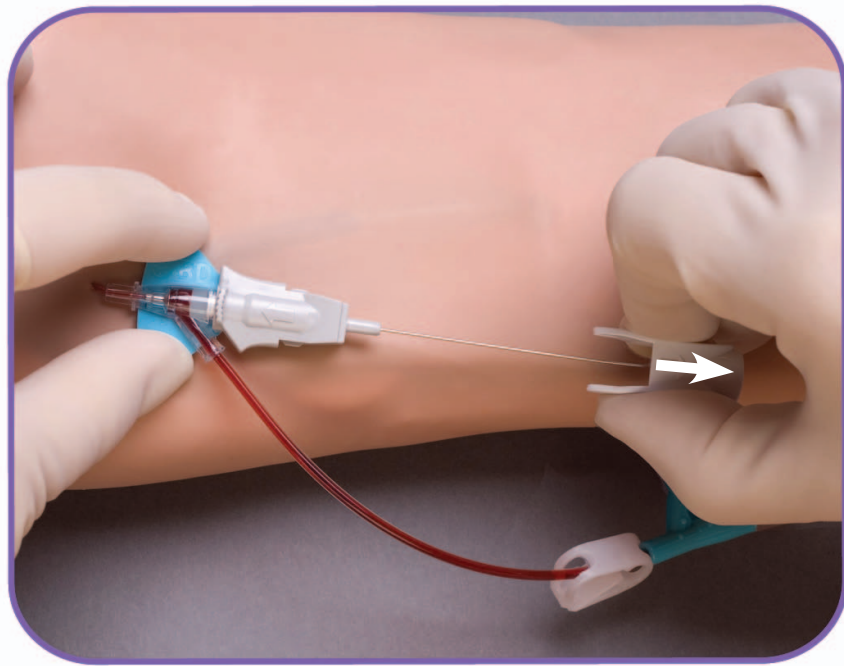


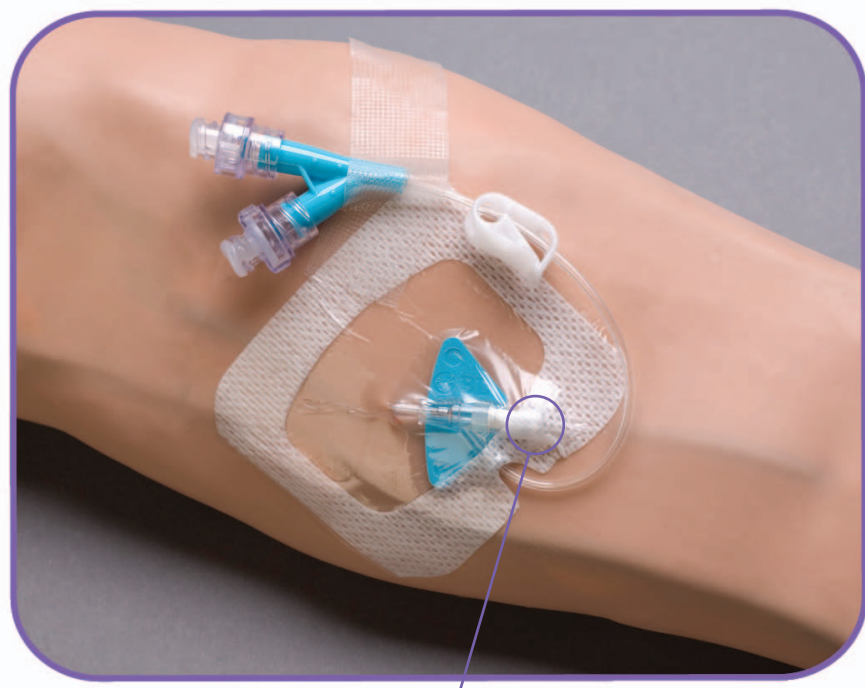

Septum

Stabilize Site

- Transparent dressing should cover the septum

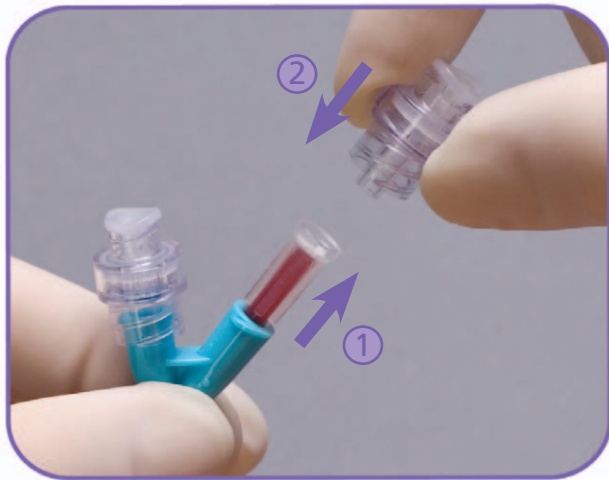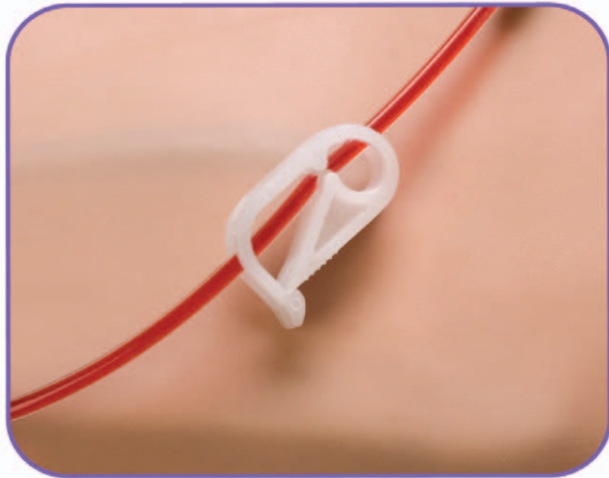

Clamp & Replace Vent Plug

- **Clamp extension**
 - Prevents blood leakage
- **Replace vent plug with BD Q-Syte™ device; flush this same port**



Unclamp & Flush

- Unclamp
- Flush
- **CLAMP BEFORE DISCONNECT**
 - Helps prevent blood reflux into catheter tip
- Keep clamped when not in use
- Single lumen catheter
 - Compatible meds only



BD Medical
2771 Bristol Circle
Oakville, Ontario
L6H 6R5
www.bd.com/ca