


BD Insyte™ Autoguard™ Shielded IV Catheter

QUICK REFERENCE CARD


1

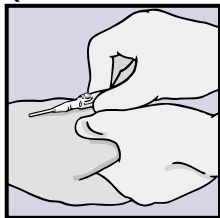

Venipuncture

- Prior to venipuncture, rotate catheter hub 360°
- Approach vein slowly at a low angle
- Observe flashback


2

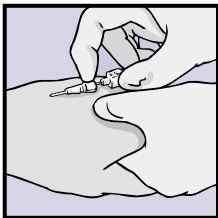

Advancement

- Upon flashback visualization, lower catheter almost parallel to the skin
- Before threading, advance ENTIRE UNIT slightly to ensure catheter tip is in the vein
- Thread catheter into vein while maintaining skin traction

3

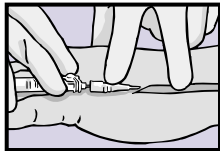

Needle Retraction

Before activating the safety device:

- Remove tourniquet
- Apply digital pressure beyond the catheter tip
- Stabilize catheter hub
- Press the white button

4 Securement

- Secure catheter and apply sterile dressing according to policy and procedure

BD Insyte™ Autoguard™ Shielded IV Catheter

QUICK REFERENCE CARD

Things to Remember:

- Do not bend the needle prior to insertion
- Never reinsert needle into the catheter as this could shear the catheter
- Do not withdraw needle from catheter hub before pressing the white button
- Do not use scissors at or near the insertion site
- Aseptic technique and proper skin preparation of the site are essential
- Discard the needle in the nearest sharps collector. Follow your institution's policy for safe disposal of all medical waste
- Obtain appropriate medical attention in the event of an accidental needlestick injury. Follow your institution's protocol.