



## Camel Lifting Cushion

The Camel Lifting Cushion will both sit up and lift a fallen person.

### Easy to fit

- The lifting cushion is inflated using an Airflo Compressor with a simple hand control
- Simple to use, requiring very little training
- Suitable for indoor and outdoor use
- Can be used independently or assisted

### Simple to maintain

- Clean using a proprietary non-abrasive mild liquid cleaner.
- Easy to deflate, fold away and store in the carry bag provided
- The material is designed to exceptionally high standards with extraordinary durability, strength & performance in the toughest of environments

### Safe to use

- Lifts the frailest person up to 700 lbs from the floor
- Stable and supportive
- Ideal for carers of individuals with neurological disorders who fall regularly

### Improves well-being

- Provides a safe, dignified lift for a fallen person
- Reduces the risk of musculoskeletal injury for the carer
- Individuals with a cognitive impairment are less distressed when lifted with a Camel
- The backrest gives additional support and is particularly suitable for bariatric patients

### Technical Specifications

Max user weight	700 lbs
Max height	22"
Min height	Flat
Seat depth	18 1/2"
Total length	44"
Width	28 1/2"
Total product weight	14 lbs
Back/headrest length	32"
Airflo Compressor	24
Lifting performance per charge	up to 4 lifts




Product code: MPCA060400

## Features

- 1. Easy to setup** - connect the four hoses from the hand control to the corresponding numbers and coloured connectors on the Camel.
- 2. The head should lie on the Camel badge** and the body should lie centrally.
- 3. The backrest** may be partially inflated before the lower sections.
- 4. Made from very strong polyurethane** coated nylon material.
- 5. Each layer inflates individually** ideal for single-handed care.

## Accessories

- Slidesheet
- Camel Carry Bag

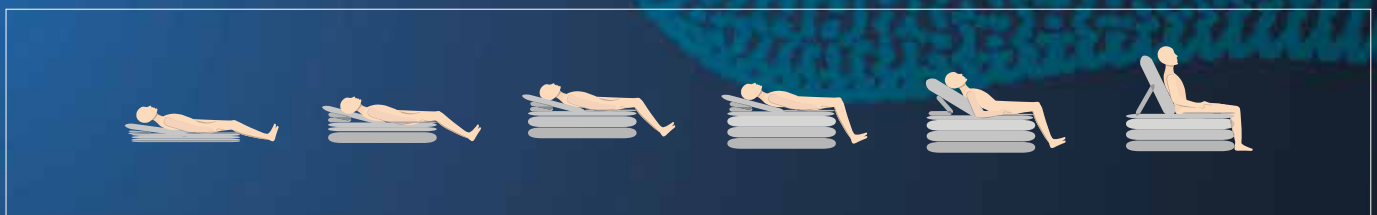

**For more information visit**  
[www.mangarhealth.com](http://www.mangarhealth.com)  
 Email: [sales@mangarhealth.com](mailto:sales@mangarhealth.com)  
 Phone: 0800 2800 485



ML1067 Issue 1