

## Anti-Embolism Stockings

McKesson anti-embolism stockings can help reduce the threat of blood clots forming in legs. Blood clots can form in anyone, but the threat of their formation is much greater during a hospital stay. McKesson anti-embolism stockings improve blood circulation in the leg veins by applying graduated compression. The stockings are designed for easy application and comfort while wearing. They are made of nylon/spandex with a graduated compression, inspection toe hole and a formed heel pocket. Color-coded for easy sizing. The stockings provide the standard compression of 20–30 mmHg from the ankle upward.

### Indications:

- To help prevent formation of deep vein thrombosis.
- To promote increased blood flow velocity in the legs by compression of the deep venous system.

### Contraindications:

Stockings are not recommended for patients with the following:

- Any local leg condition in which stockings would interfere, such as dermatitis, immediate post-operative vein ligation, gangrene or recent skin graft.
- Severe arteriosclerosis or other ischemic vascular disease.
- Massive edema (swelling) of legs or pulmonary edema from congestive heart failure.
- Extreme deformity of leg.



### Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

### Never Compromise on Care<sup>SM</sup>

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.


Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

## ORDERING INFORMATION

DESCRIPTION	MFR #	QTY
Anti-Embolism Stockings, Knee-High, Regular, Small	84-01	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Knee-High, Regular, Medium	84-02	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Knee-High, Regular, Large	84-03	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Knee-High, Regular, X-Large	84-04	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Knee-High, Long, Small	84-11	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Knee-High, Long, Medium	84-12	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Knee-High, Long, Large	84-13	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Knee-High, Long, X-Large	84-14	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Thigh-High, Short, Small	84-21	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Thigh-High, Regular, Small	84-22	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Thigh-High, Long, Small	84-23	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Thigh-High, Short, Medium	84-31	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Thigh-High, Regular, Medium	84-32	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Thigh-High, Long, Medium	84-33	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Thigh-High, Short, Large	84-41	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Thigh-High, Regular, Large	84-42	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Thigh-High, Long, Large	84-43	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Thigh-High, Regular, X-Large	84-44	1 pair/pack, 12 packs/cs
Anti-Embolism Stockings, Thigh-High, Long, X-Large	84-45	1 pair/pack, 12 packs/cs

## SIZING GUIDE

		MFR #	SIZE	CALF CIRCUMFERENCE	LENGTH	TOP COLOR	TOE PORT COLOR
KNEE	REGULAR	84-01	Small	Up to 12"	Up to 16"	White	Yellow
		84-02	Medium	12" to 15"	16" to 17"	White	White
		84-03	Large	15" to 17"	17" to 18"	White	Blue
		84-04	X-Large	17" to 19"	18" to 19"	White	Green
	LONG	84-11	Small	Up to 12"	Up to 18"	Blue	Yellow
		84-12	Medium	12" to 15"	18" to 19"	Blue	White
		84-13	Large	15" to 17"	19" to 20"	Blue	Blue
		84-14	X-Large	17" to 19"	20" to 21"	Blue	Green
THIGH	SHORT	84-21	Small	Up to 12"	Up to 29"	Yellow	Yellow
		84-31	Medium	12" to 15"	Up to 29"	Yellow	White
		84-41	Large	15" to 17"	Up to 29"	Yellow	Blue
	REGULAR	84-22	Small	Up to 12"	29" to 33"	White	Yellow
		84-32	Medium	12" to 15"	29" to 33"	White	White
		84-42	Large	15" to 17"	29" to 33"	White	Blue
		84-44	X-Large	17" to 19"	29" to 33"	White	Green
	LONG	84-23	Small	Up to 12"	33" to 36"	Blue	Yellow
		84-33	Medium	12" to 15"	33" to 36"	Blue	White
		84-43	Large	15" to 17"	33" to 36"	Blue	Blue
		84-45	X-Large	17" to 19"	33" to 36"	Blue	Green

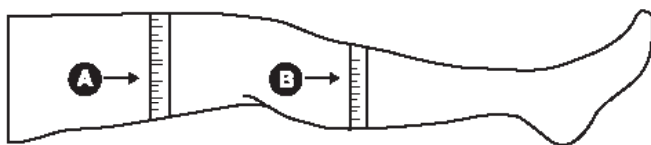

### FOR PROPER FIT – KNEE LENGTH

Size based on: Circumference of "B"

Length based on: Distance from bend of knee to bottom of heel

### FOR PROPER FIT – THIGH LENGTH

Size based on: Circumference of "A" and "B"

Length based on: Distance from "A" to bottom of heel

### LAUNDERING GUIDELINES

Machine wash in lukewarm water. Air dry/Tumble dry on low setting. Do not iron.

McKesson Medical-Surgical  
9954 Mayland Drive  
Richmond, VA 23233  
[mms.mckesson.com/mckesson-brands](https://mms.mckesson.com/mckesson-brands)

Post-Acute Care and Home Health  
800.347.2456

Home Medical Equipment  
888.822.8111

Primary Care  
866.625.2679

McKesson Brands Support  
800.777.4908