# **GUARDIAN®**

Family-owned. Trusted by hospitals for over 50 years.  
Entreprise familiale. Jugée fiable par les hôpitaux depuis plus de 50 ans.  
Empresa familiar. Cuenta con la confianza de hospitales por más de 50 años.

## **BARIATRIC COMMODE WITH BENCH SEAT**

**USER INSTRUCTIONS AND WARRANTY**  
Weight capacity 650 lbs (294 kg)

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**LIFETIME  
LIMITED  
WARRANTY**

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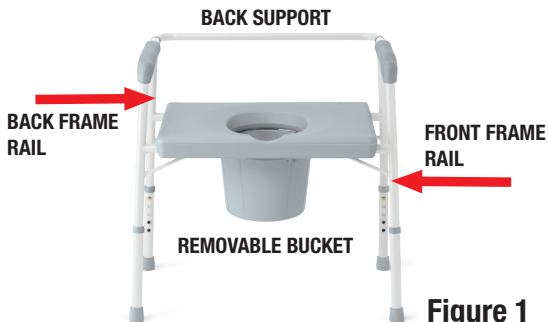
## MAINTENANCE

Periodic visual inspection of this commode is recommended to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE

## Parts Included:

1. Back support
2. Armrest
3. Platform seat
4. Front frame rail
5. Back frame rail
6. Commode pail with lid
7. Height adjustable legs
8. Leg tip

Please dispose of shipping and packaging materials according to local regulations.

## ASSEMBLY:

NOTE: DO NOT use excessive force when installing any parts.

1. Remove contents from box, making sure you have all

parts described here. Check all parts for damage before use. If any parts are missing or damaged, DO NOT attempt to use the commode.

**Important** If you do not understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise, injury may result.

2. If exposed to cold temperatures, allow contents to come to room temperature before assembly. This helps avoid damaging the clamps during assembly.
3. Align the clamps on the toilet seat platform so they rest on the front and back frame rails. (Figure 1)  
Push down on seat platform with both hands until all four clamps snap firmly into place.
4. From the front of the frame, slide the bucket (without lid) under the seat until you feel it “click” into place.
5. The commode is now ready for use.

## Adjust Commode Height

1. Before using the seat, make sure all legs are set to the same preferred height.
2. To do so, depress pushbutton and slide each leg to desired height by aligning with proper holes and letting the pushbutton snap into the correct hole to lock leg in place. (Figure 2)
3. Make sure legs are locked into place by pushing down and pulling up on each leg.

**Figure 2**

## Using as Bedside Commode

1. Set commode to desired height, making sure each leg is locked into place.
2. Be sure the commode pail is snapped into pail support bracket and properly positioned below commode opening.
3. When user is finished using commode, place pail lid over commode pail to contain odors.
4. Clean seat and pail with disinfectant after each use.

## **CAUTIONS:**

1. Inspect all parts for possible damage before assembling. If there is any damage - DO NOT USE. Contact dealer/ carrier for further instructions.
2. Make sure that the height adjusting pushbuttons fully extend through the same corresponding hole of each leg extension. This ensures that the leg extensions are securely locked in place at an even height.
3. Check fit and tightness of all nuts, bolts and knobs WEEKLY to ensure that the commode is stable and safe to use.
4. Make sure the rubber tips are in place on the leg extensions and not ripped, worn or missing. Replace any missing or damaged tips.
5. The rubber tips of all four leg extensions MUST touch the floor simultaneously at all times.
6. Users with limited physical strength should be supervised or assisted while using commode.
7. Use the armrest for assistance only. DO NOT use the armrests to support full body weight.
8. Always use caution when floor is wet. Ensure armrest handles are dry before use.
9. The platform seat must be securely attached to the frame prior to use. Failure to do so may result in injury or damage to the commode.
10. This product should not be used without instruction from a healthcare professional.
11. NEVER stand on commode

NOTE: To maintain appearance and promote good hygiene, clean periodically with mild soap and rinse with warm water. Let commode dry thoroughly after cleaning. Never use organic solvents, abrasive cleaners or scouring pads on any part of commode.

USE CAUTION. FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION.

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## LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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---

**GUARDIAN**<sup>®</sup>

Medline Industries, Inc.  
Three Lakes Drive  
Northfield, IL 60093 USA

---

Medline United States  
1-800-MEDLINE (633-5463)  
www.medline.com  
www.info@medline.com

Medline Canada  
1-800-268-2848  
www.medline.ca  
Canada@medline.com

Medline Mexico  
01-800-831-0898  
medlinemexico.com  
mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2020 Medline Industries, Inc. Guardian and Medline are registered trademarks of Medline Industries, Inc.

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