GUARDIAN®

Family-owned. Trusted by hospitals for over 50 years.
Entreprise familiale. Jugée fiable par les hôpitaux depuis plus de 50 ans.
Empresa familiar. Cuenta con la confianza de hospitales por más de 50 años.

BARIATRIC FOLDING COMMODE USER INSTRUCTIONS AND WARRANTY Weight capacity 650 lbs (294 kg)

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**LIFETIME
LIMITED
WARRANTY**

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GENERAL WARNINGS

To ensure the safe assembly and operation of the commode, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- DO NOT assemble and use this product without first reading and understanding this user guide, otherwise injury or damage may occur.
- DO NOT use if components are missing or are damaged, contact Medline at 1-800-MEDLINE.
- DO NOT exceed weight capacity of 650lbs (294 kg).
- DO NOT make any adjustments while using the product.
- DO NOT attempt to use the arms to support full body weight.
- DO NOT stand on the commode.

ASSEMBLY:

1. Before assembling seat and lid, allow toilet seat and lid to reach room temperature if exposed to cold. This will help prevent the seat and lid clamps from cracking while being assembled.
2. Align the toilet seat clamps onto the back frame rail. (Both clamps should sit outside the pail brackets). Push down on seat until clamps snap firmly into place over frame rail.
3. Repeat the same process for the lid.
4. When removing the seat or lid, always remove lid first and then the seat. To remove, grasp the back of the lid and/or seat and lift.
5. Install commode bucket by dropping into bucket frame. Lift bucket from frame to remove for emptying and cleaning.

Figure 1

LEG HEIGHT ADJUSTMENT

Before using the commode, make sure all legs are set to the same preferred height.

1. To adjust leg height, depress pushbutton and slide each leg to the desired height by aligning with proper holes and letting the pushbutton snap into the correct hole. (Figure 2)
2. Repeat until all legs are adjusted to the preferred height.
3. Make sure legs are locked into place by pushing down and pulling up on each leg.

Figure 2

CAUTIONS:

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product.

Prior to use, ensure the following:

1. Toilet seat must be in the down position before use. Failure to do so may result in possible injury or damage.
2. All push buttons are fully protruded and are properly engaged.
3. There are no loose, broken or missing components and ensure all parts are secure.
4. When using the backrest, ensure the commode remains balanced. Excessive twisting or turning while seated on the commode may cause a loss of balance and/or damage to the product.
5. Tips on the extension legs do not contain rips, cracks or are excessively worn. Immediately replace missing or damaged tips.
6. All four legs touch the floor simultaneously at all times.
7. User evenly distributes weight between the arm supports when transferring in and out of the commode.

MAINTENANCE

Periodic visual inspection of this commode is recommended to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE.

CLEANING

- Clean with regular household cleaning products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
- DO NOT use abrasive powders, cleaners or scouring pads.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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GUARDIAN[®]

Medline Industries, Inc.
Three Lakes Drive
Northfield, IL 60093 USA

Medline United States
1-800-MEDLINE (633-5463)
www.medline.com
www.info@medline.com

Medline Canada
1-800-268-2848
www.medline.ca
Canada@medline.com

Medline Mexico
01-800-831-0898
medlinemexico.com
mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2021 Medline Industries, Inc. Guardian and Medline are registered trademarks of Medline Industries, Inc.

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