MEDLINE

EVENCARE® G2™

BLOOD GLUCOSE MONITORING SYSTEM

WITH VOICE GUIDANCE

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USER'S GUIDE
/GUÍA DE USUARIO

Dear EVENCARE G2 Owner,

Thank you for choosing the EVENCARE G2 Blood Glucose Monitoring System to help you in monitoring your diabetes. We designed the EVENCARE G2 Blood Glucose Monitoring System to be accurate and easy to use. The added convenience of the voice feature can be used as a verbal aid for using the meter and hearing test results by visually impaired users. This manual contains all of the information needed to use and to maintain your new blood glucose meter. Please read it carefully before use.

If you need assistance, please call Medline's Technical Service Center at 1-866-274-2131 Monday through Friday between 8:00 a.m. and 5:00 p.m.(Central Time). Contact your Healthcare Provider at all other times.

Intended Use

The EVENCARE G2 Blood Glucose Monitoring System is intended for the quantitative measurement of glucose in fresh capillary whole blood samples drawn from the fingertips, forearm, or palm. Testing is done outside the body (In Vitro diagnostic use). The meter includes voice functionality to assist visually impaired users. It is intended for both lay use by people with diabetes and in a clinical setting by healthcare professionals, as an aid to monitoring levels in Diabetes Mellitus. It is not intended for the diagnosis or screening of diabetes or for neonatal use.

IMPORTANT:

- Alternate Site Testing (AST) with the EVENCARE G2 Blood Glucose Monitoring System can be used only during steady-state blood glucose conditions.
- Dehydration – Severe dehydration may lead to inaccurate blood glucose test results. If you suspect you are severely dehydrated, contact your healthcare professional immediately.
- Hematocrit range – A hematocrit range that is higher than 55% or lower than 30% can cause inaccurate blood glucose test results.

Table of Contents

Understanding Your New Blood Glucose System

The EVENCARE G2 Meter	8
Meter Display Screen	10
The EVENCARE G2 Test Strip	12
EVENCARE G2 Glucose Control Solution (Recommended)	13

Setting Up Your New System

The Set Up Screen	14
Setting the Clock	15
Setting the Alarms	18
Voice and Beeper Function	20
Language Selection	21

Voice Function Table **22**

Control Solution Testing

Performing a Control Solution Test	27
Control Solution Troubleshooting	31

Testing Your Blood Glucose

Preparing Your Lancing Device	32
Important Information on Alternate Site Testing (AST)	34
Performing Blood Glucose Test	36

Understanding Your Blood Glucose Test Results **40**

Viewing Your Meter's Memory **41**

Caring for the Meter

Changing the Batteries	44
------------------------	----

Cleaning and Disinfecting the Meter and Lancing Device	44
Storage and Precautions	48
Solving Problems	49
Product Warranty	52
Specifications	53

Understanding Your New Blood Glucose System

The EVENCARE G2 Blood Glucose Monitoring System contains the following items if it arrives as a meter only system:

- EVENCARE G2 Meter
- User's Guide
- Carrying Case
- Log Book

The EVENCARE G2 Meter

Test Strip Port

Insert test strip (see page 36) to turn on the meter.

Meter Display

Shows your blood glucose test results, memory values, averages, and other messages.

Enter () and buttons

Press and hold Enter () for 2 seconds to turn on or off the meter. Press Enter () to go into setup mode when not testing (see page 14).

Press  or  to enter control mode with test strip inserted (see page 28), during meter setting (see page 14), or to navigate the stored test values and averages (see page 41).

Battery Compartment
Holds two AAA batteries.

Data Port
This function is not yet available.

Meter Display Screen

BATTERY

Appears when batteries need to be changed (see page 44).

END

Will appear on the display screen after the last recorded test result, if less than 300 readings are stored.

MEM

Indicates the number in the meter display is a stored test value (see page 41).

ALARM

Indicates alarm has been set (see page 18).


CTL

This icon indicates a control solution test (see page 28) or the stored value is a control solution result (see page 43).

CABLE

Indicates the meter is in data transfer mode to PC*.

*This function is not available yet.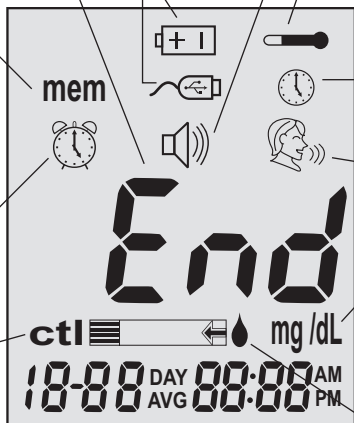

Displays the date, time, am/pm, or type of day averages (7, 14, or 30, see page 42).

BEEPER

Set sound volume (see page 20).

TEMPERATURE

Appears when it is either too hot or too cold to test (outside the ranges of 50°F-104°F).

CLOCK

Indicates the meter is ready to set up time and date (see page 15).

VOICE

Indicates the voice feature is on (see page 20).


UNITS

Unit of measurement for your blood glucose is mg/dL.

BLOOD DROP and ARROW

Indicates the meter is ready for testing (see page 36).

The EVENCARE G2 Blood Glucose Test Strip

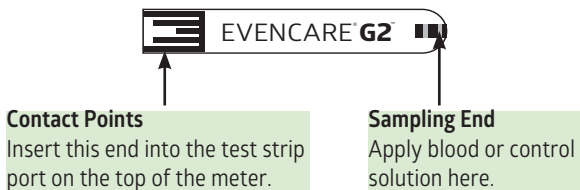The EVENCARE G2 Test Strips are glucose specific, biosensor-based test strips that require very little blood sample to quickly test glucose in fresh capillary whole blood. The test results are plasma-calibrated for easy comparison to lab results, however results may be 10% to 15% higher than results obtained from whole-blood referenced blood glucose assays. The test strip is also equipped with under-fill detection to alert you when there is not enough blood to perform a test, so you can be assured that every reading you get is an accurate and meaningful result.

IMPORTANT:

- Immediately close the vial cap of the test strip bottle tightly after each use. Keep the unused test strips in the original bottle.
- DO NOT leave any test strips outside the bottle while not in use.
- DO NOT reuse test strips. Test strips are for single use only.
- Carefully discard used test strips and lancets in proper waste containers.

EVENCARE G2 Glucose Control Solution (Recommended)


Call your local supplier or pharmacy to order control solution. If they do not stock it, please have your supplier or pharmacy contact Medline Customer Service to place an order.

EVENCARE G2 Glucose Control Solution is the standard glucose concentration solution required to perform a glucose control solution test (See Control Solution Testing). It is available at 2 levels. EVENCARE G2 LOW Glucose Control Solution helps you to validate the performance of your system at a low range blood glucose reading, while HIGH solution helps you to validate your system performance at a high range blood glucose reading. Run the control test to make sure the test strips and the meter are working together properly, to practice testing procedure, and after opening a new bottle of EVENCARE G2 Blood Glucose Test Strips.

Setting Up Your New System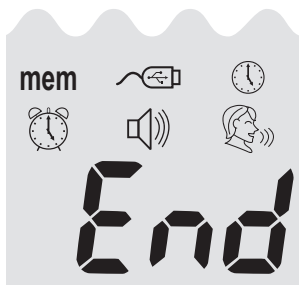

The Setup Screen

The Setup Screen of the EVENCARE G2 Meter lets you take advantage of the advanced functions of this meter. Simply use ▲ ▼ buttons to navigate to the desired function icon, and press the Enter (⏻) button to enter the function when the icon is flashing. Navigate to End on the display screen to exit the Setup Screen. The meter will also announce each function when the voice feature is turned on.

- mem Press Enter (⏻) to enter Memory Recall to retrieve stored results. Your meter can memorize up to 300 memories with date and time. (see page 41).
-  Press Enter (⏻) to enter PC Link Mode for data transfer onto a PC*. *This function is not available yet.
-  Press Enter (⏻) to enter Time and Date Setup (see page 15).
-  Press Enter (⏻) to enter Alarm Setup (see page 18). Your meter can have as many as 3 alarm settings.

 Press Enter () to enter Volume Setup (see page 20).

 Press Enter () to enter Language Selection (see page 21).

End..... Exit Setup Screen and return to testing.

Setting the Clock

You will need:

✓ Your EVENCARE G2 Meter

NOTE

If date and time on the EVENCARE G2 Meter has not been set, test and control results will not be stored in memory.

Step 1:

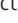
Press and hold Enter () to turn on the meter.

Step 2:

Press Enter () to go into setup screen, then press  or  until  start to flash. If the voice function is on, the meter will say, "Set date and time." Press Enter () to enter clock setup.

Step 3: Set the Year

The current year will flash. If the voice function is on, the meter will say, "Set year." Press  or  to select the correct year. Press Enter () to confirm your choice and advance to set the month.

Step 4: Set the Month

The current month will flash. If the voice function is on, the meter will say, "Set month." Press ▲ or ▼ to select the correct month. Press Enter (⏏) to confirm your choice and advance to set the date.

Step 5: Set the Date

The current date will flash. Press ▲ or ▼ to select the correct day. If the voice function is on, the meter will say, "Set day." Press Enter (⏏) to confirm your choice and advance to set time format.

Step 6: Set Time Format to 12-hour or 24-hour

The current time format will flash. Press ▲ or ▼ to select either 12-hour or 24-hour format. Press Enter (⏏) to confirm your choice and advance to set hour.

Step 7: Set the Hour

The current hour will flash. If the voice function is on, the meter will say, "Set hour." Press ▲ or ▼ to select the correct hour. Press Enter (⏏) to confirm your choice and advance to set minutes.

Step 8: Set Minute

The current minutes will flash. If the voice function is on, the meter will say, "Set minutes." Press ▲ or ▼ to select the correct minutes. Press Enter (⏏) to confirm your choice and finish setting the clock.

**NOTE:**

- Anytime during set up, inserting a EVENCARE G2 Test Strip will begin testing. Any changes made so far are stored.
- You may need to reset the time after changing the batteries.

After date and time have been set, the meter will announce the date and time if the voice feature is on. Begin testing by inserting an EVENCARE G2 Blood Glucose Test Strip (see page 32, Testing Your Blood Glucose), or press and hold Enter (⏏) to turn off the meter, or see the next section on how to set up the alarm feature on your meter.

Setting the Alarms

You can set up to three alarms on the EVENCARE G2 Meter. You must set the clock before you can set the alarms. When an alarm sounds, the meter will turn on and a melody will sound for 30 seconds. Pressing Enter (⏏) or inserting a test strip will silence the alarm.

You will need:

✓ Your EVENCARE G2 Meter

Step 1:

Press and hold Enter (⏏) to turn on the meter.

Step 2:

Press Enter (⏏) to enter setup screen.

Step 3:

Press ▲ or ▼ until  starts to flash. If the voice function is on, the meter will say, "Set testing alarm." Press Enter (⏏) to enter alarm setting.

Step 4:

The display screen will indicate alarm 1 status (the default is OFF). You can turn it on or off by pressing ▲ or ▼.

Step 5:

Press Enter (⏏) and the hour will flash. Press ▲ or ▼ to select the hour. Press Enter (⏏) to confirm your choice and advance to set minutes.

**Step 6:**

The current minutes will flash. Press ▲ or ▼ to select the correct minutes. Press Enter (⏏) to confirm your choice and advance to set alarm 2.

**Step 7:**

Set alarm 2 and 3 following steps 4 through 6. After you have finished setting up alarm 3, press Enter (⏏) to exit alarm setup.

NOTE:

- Alarm will not sound during testing.
- Alarm will follow the time format (12-hour or 24-hour) you choose when setting up the clock.
- Alarm settings will not be erased when changing batteries.

Begin testing by inserting a EVENCARE G2 Blood Glucose Test Strip (see page 32, Testing Your Blood Glucose), or press and hold Enter (⏏) to turn off the meter, or see the next section on how to set up the voice feature on your meter.

Voice and Beeper Function

Your meter has a voice function that can be used as a verbal aid for using the meter and hearing test results. The default setting for the voice function is on, and can be switched off to conserve battery power. When the voice function is off, the meter will beep to alert you to important messages.

You will need:

✓ Your EVENCARE G2 Meter

Setting Volume

Step 1:

Press and hold Enter () to turn on the meter.

Step 2:

Press Enter () to go into setup screen. Press  or  until  starts to flash. Press Enter () to enter volume setting.

Step 3:

The display screen will show the current sound volume (default is 2).

Press  or  to adjust the sound volume from 1 (lowest) to 5 (highest). A melody will sound according to the sound volume you have selected. Setting to 0 will turn off the voice function. The meter will instead beep to alert you to important messages. Once you have selected the desired volume, press Enter () again to exit volume setting.

Language Selection

Step 1:

Press and hold Enter (⏏) to turn on the meter.

Step 2:

Press Enter (⏏) to enter setup screen.

Step 3:

Press ▲ or ▼ until  starts to flash. The meter will announce "Language selection."

Step 4:

The default language (1 for English) is announced. Press (⏏) to change the language.

Press ▲ or ▼ to select another language. Press (⏏) when you have finished selecting the language to exit language selection. English and Spanish are the two language options available.

**NOTE:**

- Any time during set up you can insert an EVENCARE G2 Test Strip and begin testing. Any changes made so far are stored.
- Your choice of language and volume settings will not be erased when changing batteries.

Begin testing by inserting an EVENCARE G2 Blood Glucose Test Strip (see page 32, Testing Your Blood Glucose) or press and hold Enter (⏏) to turn off the meter.

Voice Function Table

The voice function can be used as an aid for you to hear the test results. The following table lists when the meter will speak and what the meter will say.

WHEN the meter will speak...	WHAT the meter will say...
Turned on.	Greeting tone will sound.
Prompt user to insert test strip.	"Please insert test strip."
Test strip is inserted and the meter is ready to test (with flashing arrow and blood drop).	"Blood glucose test. Please apply blood to the test strip."
Blood glucose test is completed and the result is shown on screen.	"Your blood glucose reading is (number) milligrams per deciliter."
Blood glucose test result is BELOW the measurement range of 20 mg/dL.	"Your blood glucose reading is below 20 mg/dL."
Blood glucose test result is ABOVE the measurement range of 600 mg/dL.	"Your blood glucose reading is above 600 mg/dL."

WHEN the meter will speak...	WHAT the meter will say...
Temperature is out of range (between 50°F-104°F).	"Temperature is out of range."
When the meter is turned off.	Melody will sound.

Test Strip is inserted and press ▲ or ▼ (CTL showing next to the test strip icon) .

"L1 control solution test. Please apply control solution onto the test strip."


"Blood glucose test. Please apply blood onto the test strip."

"L2 control solution test. Please apply control solution."

Setting Meter Functions via the Setup Screen

(see page 14, setting up your new system)

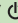Pressing Enter () to enter setup screen (after the meter has been turned on)

Memory Recall

Exit Function Mode

Language Selection

PC Link*

* This function is not available yet.

Set Date and Time

Set Testing Alarm

Set Sound Volume

WHEN the meter will speak...

WHAT the meter will say...

During date and time setup, setting up the year.

“Set year.”

During date and time setup, setting up the month.

“Set month.”

During date and time setup, setting up the date.

“Set date.”

During date and time setup, setting up the hour.

“Set hour.”

WHEN the meter will speak...	WHAT the meter will say...
During date and time setup, setting up the minutes.	"Set minutes."
After finishing setting up date and time.	"The date and time is (year) (month) (date) (hour) (minutes)."
Setting alarm 1, 2, or 3.	"Set alarm (number)."
When sending data to PC. *This function is not available yet.	"Sending data to PC."
Finish sending data to PC. *This function is not available yet.	"Data transmission complete."

Other warning messages (see page 49, Solving Problems)

WHEN the meter will speak...	WHAT the meter will say...
Meter date and time has not been set.	"Please set date and time"
Meter failure.	"Error 1 meter failure"
Used test strip.	"Error 3 used test strip"
Not enough blood or control solution.	"Error 4 not enough sample"

Control Solution Testing

The purpose of the control solution testing is to make sure the EVENCARE G2 Meter and the EVENCARE G2 Test Strips are working properly. You should perform control solution testing when:

- Using the meter for the first time.
- Using a new bottle of EVENCARE G2 Blood Glucose Test Strips.
- You left the test strip bottle cap open for awhile.
- You dropped the meter.
- You suspect your meter and test strips are not working properly.
- The blood glucose test results do not reflect how you feel.
- You want to practice the testing procedure.

IMPORTANT:


- Use only the EVENCARE G2 Glucose Control Solutions with the EVENCARE G2 Test Strips. Other brands of control solution will produce inaccurate results.
- Always check the expiration date. **DO NOT** use control solutions if they are expired.
- **DO NOT FREEZE.** Store the control solutions at room temperature.

You will need:

- ✓ EVENCARE G2 Control Solution (Low and High)
- ✓ Your EVENCARE G2 Meter
- ✓ A new EVENCARE G2 Blood Glucose Test Strip

Step 1:

Take out an EVENCARE G2 Test Strip from the test strip bottle and close the bottle immediately. Insert the test strip to turn on the meter.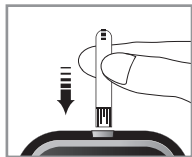


IMPORTANT:

- Check the expiration date printed on the test strip bottle. Do not use expired test strips.
- Use each test strip immediately after removing it from the bottle.
- Do not use wet or damaged test strips.
- Keep test strips away from direct sunlight and heat. Store the test strip bottle in a dry, cool place.
- Record the “date opened” on the bottle label. Discard the bottle and any remaining test strip after 6 months from date of opening.
- Test within the temperature range of 50°F to 104°F.
Your meter will not work outside these ranges.
- Insert the test strip properly.
- The flashing blood drop icon will appear once the test strip has been fully inserted.

Step 2:

Wait until the flashing blood drop and arrow icons appear on the meter display screen. The meter will also announce, “Please apply blood onto the test strip” if you have the voice feature turned on. Press ▼ one time to enter L1 (LOW) control solution testing.

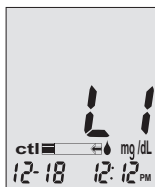Step 3:

ctl icon will appear next to the test strip icon and L1 will appear on the meter display screen. The meter will also announce “L1 control solution test, please apply control solution onto the test strip.” If you decide not to do a control solution test, press ▼ two times and ctl will disappear from the meter display.

Step 4:

Squeeze a drop of Low control solution onto a clean, dry, nonabsorbent surface. Do not apply control solution to the test strip directly from the control solution bottle. Replace the bottle cap on the control solution bottle immediately after use.

DO NOT touch the sampling end of the test strip with the bottle of control solution. See illustration of sampling end below.


Step 5:

Gently touch the tip of the test strip to the drop of control solution. The meter will beep indicating it has received the control solution. The screen will start to count down. After 6 seconds, the control solution test result will appear on the meter display screen.

Step 6:

Compare the reading on the screen to the Low range printed on the test strip bottle. If the reading falls outside the Low range printed on the test strip bottle, see Control Solution Troubleshooting, on page 31.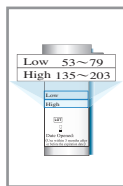

NOTE:

- Control solution test results will be stored in meter memory and indicated by ctl icon.
- Control solution test results will not be used for calculating averages.

Step 7:


Remove the used test strip. The meter will automatically turn off. Discard the used test strip. Follow steps 8 through 14 to perform the high control solution testing.

IMPORTANT:

Do not reuse test strips.


Step 8:

Take out an EVENCARE G2 Test Strip from the test strip bottle and close the bottle immediately. Insert the test strip to turn on the meter.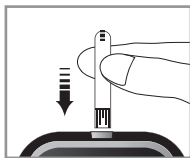

Step 9:

Wait until the flashing blood drop and arrow icons appear on the meter display screen. The meter will also announce, "Please apply blood onto the test strip" if you have the voice feature turned on. Press ▼ two times to enter L2 (HIGH) control solution testing.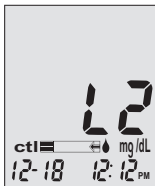

Step 10:

ctl icon will appear next to the test strip icon and L2 will appear on the meter display screen. The meter will also announce "L2 control solution test, please apply control solution onto the test strip." If you decide not to do a control solution test, press ▼ one time and ctl will disappear from the meter display.

Step 11:

Squeeze a drop of High control solution onto a clean, dry, non-absorbent surface. Do not apply control solution to the test strip directly from the control solution bottle. Replace the bottle cap on the control solution bottle immediately after use.


Step 12:

Gently touch the tip of the test strip to the drop of control solution. The meter will beep indicating it has received the control solution. The screen will start to count down. After 6 seconds, the control solution test result will appear on the meter display screen.

Step 13:

Compare the reading on the screen to the High range printed on the test strip bottle. If the reading falls outside the High range printed on the test strip bottle, see Control Solution Troubleshooting below.

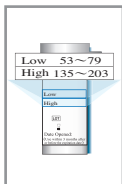**Step 14:**

Remove the used test strip. The meter will automatically turn off. Discard the used strip.

Control Solution Troubleshooting

If your control solution result is out of range (too high or too low), it may be caused by the following:

Possible Causes	What you can do...
<ul style="list-style-type: none"> Wrong brand of control solution being used. 	<ul style="list-style-type: none"> Make sure you are using EVENCARE G2 Control Solution.
<ul style="list-style-type: none"> Control solution not at room temperature. 	<ul style="list-style-type: none"> Make sure the testing environment is between 50°F to 104°F.
<ul style="list-style-type: none"> Expired or contaminated control solution, or damaged test strip. 	<ul style="list-style-type: none"> Check the expiration and open bottle date on both the control solution and the test strips. Repeat the test using a new test strip. If the result is still out of range, use a new bottle of control solution and retest.
<ul style="list-style-type: none"> Meter malfunction. 	<ul style="list-style-type: none"> If the problem persists, please call Medline's Technical Service Center at 1-866-274-2131 Monday through Friday between 8:00 a.m. and 5:00 p.m. (Central Time).

Testing Your Blood Glucose

You will need:

- ✓ Your EVENCARE G2 Meter
- ✓ A new EVENCARE G2 Blood Glucose Test Strip
- ✓ Lancing device with a clean, unused lancet (or safety lancets)
- ✓ Clear cap for Alternate Site Testing (AST) on palm and forearm


Before you begin, make sure:

- To set up your meter properly and run a control test. See Setting Up Your New System on page 14, and Control Solution Testing on page 27 for details.
- Wash your hands and the testing site thoroughly with soap and warm water, and dry well.
- You are testing in an area between the temperature ranges of 50°F-104°F. Your meter will not test outside of this range and will display thermometer icon with verbal indication if the voice feature is turned on. Move the meter into an area that is between 50°F-104°F, and let it sit for 10 to 15 minutes before testing again.

Preparing Your Lancing Device (for Home Testing)



Step 1:

Unscrew the adjustable cap of the lancing device and insert the lancet by pushing down firmly until it is fully seated.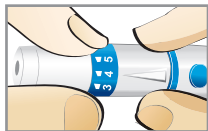

Step 2:

Twist the protective cover off the lancet. Screw the adjustable cap clockwise back onto the lancing device.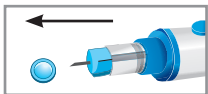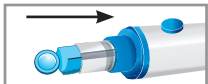


NOTE:

- For fingertip testing, use the opaque adjustable cap.
- For alternate site testing on palm and forearm, use the clear adjustable cap.

Step 3:

Adjust the depth setting on the adjustable cap for the puncture depth level (1 is the lightest and 5 is the deepest).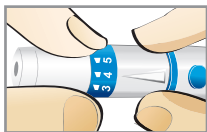

To select the best depth:


1-2 for soft skin

3 for average skin

4-5 for thick or calloused skin

Step 4:

With one hand holding the adjustable cap, pull back the sliding barrel back with your other hand until you hear a click sound. The lancing device is now ready for blood sampling.

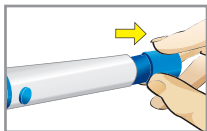**IMPORTANT:**


Use a new sterile lancet every time you test to avoid cross contamination. If alcohol wipes are used to cleanse the fingers, make sure the area is dry before the blood sample is obtained.

Important Information on Alternate Site Testing (AST)

The EVENCARE G2 Blood Glucose Monitoring System can test for blood glucose from areas other than your fingertip such as palm and forearm (alternate site testing, or AST).

Alternate site testing can be less painful than fingertip testing, but because of the physiological differences between your fingertip and palm and forearm¹, alternate site test results may be significantly different than results from fingertip testing under certain conditions.

You should consult with your doctor or healthcare professional before using alternate site testing.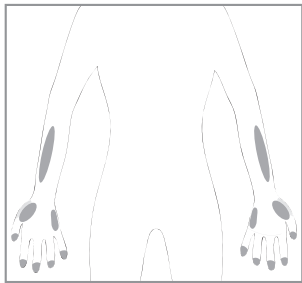

DO AST ONLY in the following intervals:

- In a pre-meal or fasting state (more than 2 hours since the last meal).
- 2 hours or more after taking insulin.
- 2 hours or more after exercise.

Alternate Site Testing SHOULD NOT be used when:

- You are Hypoglycemic unaware (not able to tell if you have low blood sugar).
- Within 2 hours of a meal, exercise, or medication.
- You will be operating machinery or driving a car.
- During times of stress.
- You are sick.
- You think your blood glucose is low.
- Your AST results do not match the way you feel.
- You are testing for hyperglycemia.
- Your routine glucose results are often fluctuating.

NOTE:

- Rubbing forearm until warm before lancing will help minimize the difference with fingertip test results. The palm (at the base of the thumb) does not require rubbing before testing.
- If results from alternate site testing do not agree with how you feel, use fingertip testing instead.


Reference:

1. Jungheim K., Koschinsky T.: "Risky delay of hypoglycemia detection by glucose monitoring at the arm." *Diabetes Care* 2001;24(7):1303-04.

Performing a Blood Glucose Test

Step 1:

Take out an EVENCARE G2 Test Strip from the test strip bottle and close the bottle immediately. Insert the test strip to turn on the meter.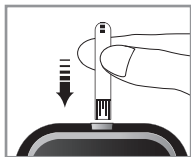


IMPORTANT:

- Check the expiration date printed on the test strip bottle and its package. Do not use expired test strips.
- Use each test strip immediately after removing it from the bottle.
- Do not use wet or damaged test strips.
- Keep away from direct sunlight and heat. Store the test strip bottle in a dry, cool place.
- Record the “date opened” on the bottle label. Discard the bottle and any remaining test strip after 6 months from date of opening.
- The flashing blood drop icon will appear once the test strip has been fully inserted.

Step 2:

Wait for a flashing blood drop and an arrow icon to appear on the meter display screen. The meter will also announce, “Please apply blood onto the test strip” if you have the voice feature turned on.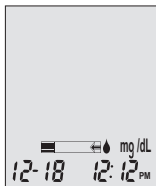


Step 3:

Obtain blood sample.

For Fingertip Testing:

Hold the lancing device (use the white adjustable cap or safety lancet) against the side of your fingertip and press the release button to create a puncture.

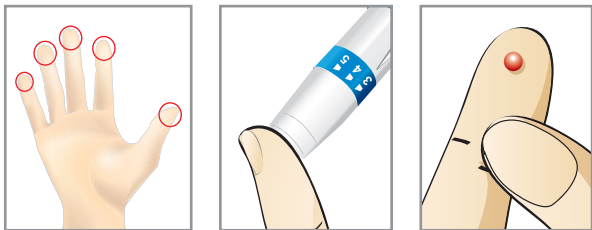
**TIP:**

- Gently massage your hand and finger toward the puncture site to form a drop of blood. Do not “milk” or squeeze around the puncture site.
- Lance the side of your fingertip to avoid soreness. To avoid calluses, choose a different lancing site each time.


For Testing on Palm and Forearm:

Hold the lancing device (use the clear adjustable cap) against the test site (base of palm or forearm). Avoid a test site that has veins, moles, excessive hair, or bone.

Press the lancing device firmly against the test site and press the release button, do not lift up the lancing device.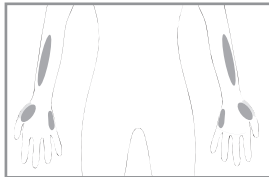

For forearm test sites, apply and release pressure (“pumping”) for a few seconds while keeping the lancing device in constant contact with the skin. Palm testing (at the base of the thumb) does not require pumping. When the blood is approximately (●) size, lift the lancing device straight up without smearing the blood.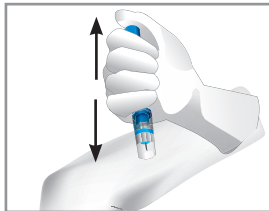


NOTE:

- Rubbing forearm until warm before lancing will help minimize the difference with fingertip test results. The palm (at the base of the thumb) does not require rubbing before testing.
- If results from alternate site testing do not agree with how you feel, use fingertip testing instead.

Step 4:

Gently bring the test strip and touch the drop of blood at a slight angle. The test strip acts like a straw to pull the blood in. Keep the test strip in the blood drop until the meter beeps to indicate the test strip has enough blood to test.

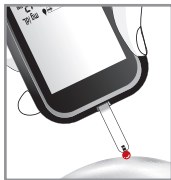**Step 5:**

The screen will start to count down. After 6 seconds, your blood glucose result will appear on the meter display screen and will be announced verbally if you have the voice feature turned on. Your test result will be automatically stored into the meter memory.

**CAUTION:**

- If you see "HI" or "LO" displayed, your blood glucose level may be above 600 mg/dL or below 20 mg/dL. Test again using fingertip testing, DO NOT test on the forearm. If you still receive the same result, call your physician or healthcare professional immediately.

Step 6:

Properly discard the used test strip. The meter will turn off automatically.

Step 7:

Remove the used lancet from your lancing device according to instructions and discard into a proper waste container.

IMPORTANT:

- Used lancets and strips are biohazardous materials and can transmit bloodborne diseases. Dispose of them according to local government regulation to avoid injury or contamination.

Understanding Your Blood Glucose Test Results

Blood glucose values will vary depending on food intake, medication, health, stress, and exercise. The ideal range for adults with and without diabetes should be¹:

- 70 to 100 mg/dL before meals, and
- 70 to 140 mg/dL after meals

It is important to consult with your physician or healthcare professional to determine an appropriate target range for you.

What to do if you get a high or low reading

If the Meter displays results that are HI or LO, or you get a result that is more than 250 mg/dL or below 50 mg/dL AND you feel ill:

- Treat your diabetes according to the instruction from your doctor and/or consult your healthcare provider.

If the Meter displays results that are “HI” or “LO,” or you get a result that is more than 250 mg/dL or below 50 mg/dL AND you DO NOT feel ill:

- Test your meter with a control solution, refer to page 27, Control Solution Testing.
- Test again using fingertip.

If you still get a high or low reading, contact your healthcare professional.

NOTE:

- Inaccurate results may occur in severely hypotensive individuals or patients in shock.
- Inaccurate low results may occur for individuals experiencing a hyperglycemic-hyperosmolar state, with or without ketosis.
- Critically ill patients should not be tested with blood glucose meters.

Reference:

1. American Diabetes Association. Standards of Medical Care in Diabetes—2008. Diabetes Care. 2008;31:S12–S54.

Viewing Your Meter's Memory

Your EVENCARE G2 Meter automatically stores up to 300 test and control results with date and time. The meter also provides 7, 14, and 30 day averaging to help track your blood glucose trend. You must set up the date and time on your meter properly to use the memory and day averaging function, see *Setting the Clock* on page 15. The meter will not record test or control results if the date and time are not set.

Viewing Stored Readings and 7, 14, 30 Day Averages in Meter Memory

You will need:

✓ Your EVENCARE G2 Meter

Step 1:

Press and hold Enter (⏏) to turn on the meter.

Step 2:

Press Enter (⏏) to enter setup screen.


Step 3:

Press ▲ or ▼ until **MEM** starts to flash. If the voice function is on, the meter will say "memory recall." Press Enter (⏏) to enter.

MEM will appear at the upper left corner of the display screen together with your most recent testing result. Press ▲ to see the next result, and the next, and so on. Holding ▲ will flash through the results.


END will appear on the display screen after the last recorded test result if less than 300 readings are stored. You can press ▼ to go back viewing from the oldest to the most recent readings.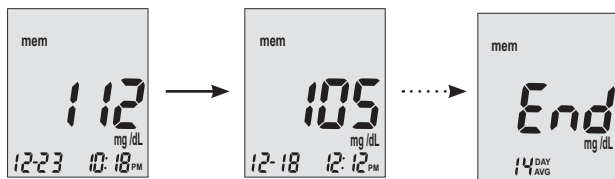

Step 4.

When you reach the most recent result, continue to press ▼ to start viewing day averages. The meter will display the 7 day average, press ▼ again to see 14 day average, and press ▼ again to see 30 day average.

The number of results included in the averages is shown right next to the ^{DAY}AVG icon. Control solution results are not included in the day average calculations.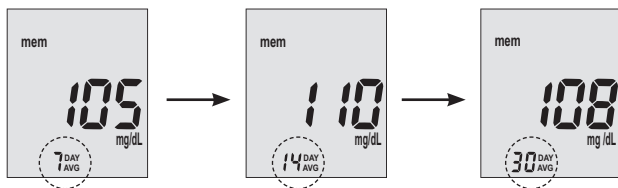


NOTE:

- Control test results are flagged by **ctl** on the bottom left corner of the display screen in addition to **MEM**.
- When the meter memory is blank, the display screen will show **nil**.
- When there are no day average results available, the display screen will show 4 dashes.
- If the meter memory is full, adding a new test result will cause the oldest result to be deleted.
- Meter memory test results and day averages will not be erased when changing batteries.
- You can begin testing anytime by inserting a test strip.

Caring for the Meter

Caring for your EVENCARE G2 Meter is easy. Follow these simple guidelines to keep your meter working properly.

Changing the Batteries

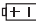Batteries need to be changed when the  icon appears on the meter display.


Materials you will need:

- ✓ Two Alkaline AAA Batteries
- ✓ Your EVENCARE G2 Meter

Step 1.

Press firmly on the battery cover and slide in the direction of the arrow.

Step 2.

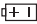Insert new batteries . Slide the battery cover back into the meter firmly.

NOTE:

- Meter will prompt you to reconfirm the date and time when turned on. See page 15 for Setting the Clock.
- The test results, alarm settings, volume setting, and language choice stay in memory when the batteries are changed.

Cleaning and Disinfecting your EVENCARE G2 Meter

Cleaning and disinfecting your meter and lancing device is very important in the prevention of infectious disease. Cleaning is the removal of dust and dirt from the meter and lancing device surface so no dust or dirt gets inside. Cleaning also allows for subsequent disinfection to ensure germs and disease causing agents are destroyed on the meter and lancing device surface.

Your EVENCARE G2 Meter and lancing device are validated to withstand a cleaning and disinfection cycle of ten times per day for an average period of three years. The following products are validated for disinfecting the EVENCARE G2 meter and lancing device:

- Dispatch® Hospital Cleaner Disinfectant Towels with Bleach (EPA Registration Number: 56392-8)
- Medline Micro-Kill+™ Disinfecting, Deodorizing, Cleaning Wipes with Alcohol (EPA Registration Number: 59894-10)
- Clorox Healthcare® Bleach Germicidal and Disinfectant Wipes (EPA Registration Number: 67619-12)
- Medline Micro-Kill™ Bleach Germicidal Bleach Wipes (EPA Registration Number: 69687-1)


These disinfecting wipes are available through major retailers online or call 1-866-274-2131 to purchase.

Other EPA Registered wipes may be used for disinfecting the EVENCARE G2 system, however, these wipes have not been validated and could affect the performance of your meter. If you use an EPA Registered wipe not listed above and you notice any of the below signs after cleaning and disinfecting your meter, stop using the device and call Medline Technical Service at 1-866-274-2131.

- Control solution out of range.
- Clouding meter LCD display.
- Corrosion or erosion of plastic housing or buttons.
- Cracking of plastic housing.
- Malfunction of any meter button.

Cleaning and Disinfecting Your Meter and Lancing Device

1. Wash hands with soap and water and dry thoroughly.
2. Inspect for blood, debris, dust, or lint anywhere on the meter or lancing device.
3. To clean the meter, use a moist (not wet) lint-free cloth dampened with a mild detergent. Wipe all external areas of the meter or lancing device including both front and back surfaces until visibly clean. Avoid wetting the meter test strip port.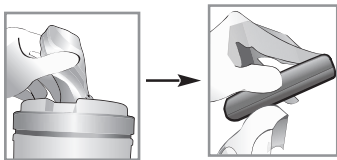


NOTE:

- Do not get water inside the meter or immerse the meter or lancing device in water or any other liquids.
- Do not use glass or household cleaners on the meter or the lancing device.
- Avoid using ammonia to clean the meter or lancing device.
- Avoid wetting the test strip port of the meter.

4. To disinfect your meter, clean the meter with one of the validated disinfecting wipes listed below. Other EPA registered wipes may be used for disinfecting the EVENCARE G2 system, however these other wipes have not been validated and could affect the performance of your meter.
 - Dispatch® Hospital Cleaner Disinfectant Towels with Bleach (EPA Registration Number: 56392-8)
 - Medline Micro-Kill+™ Disinfecting, Deodorizing, Cleaning Wipes with Alcohol (EPA Registration Number: 59894-10)
 - Clorox Healthcare® Bleach Germicidal and Disinfectant Wipes (EPA Registration Number: 67619-12)
 - Medline Micro-Kill™ Bleach Germicidal Bleach Wipes (EPA Registration Number: 69687-1)

Wipe all external areas of the meter or lancing device including both front and back surfaces until visibly clean. Avoid wetting the meter test strip port. Allow the surface of the meter or lancing device to remain wet at room temperature for the contact time listed on the wipe's directions for use.

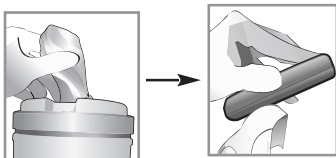5. Wipe meter dry or allow to air dry.
6. Wash hands with soap and water and dry thoroughly.

NOTE:

- If the meter or lancing device is being operated by a second person who is providing testing assistance to the user, the meter or lancing device should be disinfected prior to use by the second person.
- Keep disinfecting wipes out of reach of children and stored according to their instructions.
- Do not flush disinfecting wipes down toilet.

IMPORTANT:

If you notice any of the below signs after cleaning and disinfecting your meter or lancing device; stop using the device and call Medline Technical Service immediately at 1-866-274-2131:

- Control solution out of range.
- Clouding meter LCD display.
- Corrosion or erosion of plastic housing or buttons.
- Cracking of plastic housing.
- Malfunction of any meter button.

NOTE:

- These disinfectants were validated separately; therefore only one disinfectant should be used on the device for the life of the device, as the effect of using more than one disinfectant interchangeably has not been evaluated.

Storage and Precautions

- Handle the meter with care - severe shock, such as dropping the meter, could damage the electronics.
- The meter and test strips are designed to be used within the temperature ranges between 50°F to 104°F.
- Avoid leaving the meter in extremely hot or cold places, such as near a heat source or in an extremely hot or cold car.
- Do not store or use the meter or test strips in high humidity levels, such as bathroom or kitchen.
- Always close bottle cap immediately after removing a test strip and make sure it's closed tightly.
- Do not take the meter apart as doing so will void the warranty. If there are technical problems or questions, please call Medline's Technical Service Center at 1-866-274-2131 Monday through Friday between 8:00 a.m. and 5:00 p.m.(Central Time). Contact your healthcare provider at all other times.

Solving Problems

This section details the significant display screen messages and error codes you will encounter when using your EVENCARE G2 meter and EVENCARE G2 test strips.

What You See	What the meter says	What it Means	What Should You Do
	"Error 1 Meter Failure"	The meter's electronics or test strip is damaged.	<ul style="list-style-type: none"> Remove the batteries and turn on the meter again. Remove the test strip and insert a new test strip again. If the problem persists, please call 1-866-274-2131 Monday through Friday, 8:00 a.m. - 5:00 p.m.(CST). Contact your healthcare provider at all other times.
	"Error 3 used test strip"	Used or damaged test strip.	<ul style="list-style-type: none"> Repeat the test with a new test strip. Remember to add sample only when you see the prompt on the display screen or the meter verbally prompts you to add sample.
	"Error 4 not enough sample"	Not enough sample on the test strip to start.	<ul style="list-style-type: none"> Remove the test strip and repeat the test with a new test strip. See Testing Your Blood Glucose on page 32.

What You See	What the meter says	What it Means	What Should You Do
	“Low Battery”	Batteries are low.	Change the batteries according to page 44, Changing the Batteries.
	“Temperature is out of range”	Temperature out of range.	Move the meter into an area that is between 50°F-104°F, and allow 10 to 15 minutes for it to reach the new temperature.
	“Your blood glucose reading is above 600 mg/dL.”	Test result is higher than 600 mg/dL.	Wash and dry your hands and repeat the test on a different fingertip with a new test strip. If the result is still “HI,” contact your physician or a healthcare professional immediately.
	“Your blood glucose reading is below 20 mg/dL.”	Test result is lower than 20 mg/dL.	Wash and dry your hands and repeat the test on your fingertip with a new test strip. If the result is still “LO,” contact your physician or a healthcare professional immediately.

What You See	What the meter says	What it Means	What Should You Do
	Not Applicable	There are no results in the memory.	<ul style="list-style-type: none">• Check to see if the date and time on your meter are set up. See Setting Up Your New System on page 14.• Start testing your blood glucose, see Testing Your Blood Glucose on page 32.

Product Warranty

The EVENCARE G2 Blood Glucose Meter is warranted to be free of defects in workmanship and materials for a period of five (5) years from the date of purchase. Medline's liability for warranty claims is limited to repair or replacement, and in no event shall Medline be liable for indirect or consequential damages, or for any loss arising from misuse, improper use, abuse, neglect, unauthorized repair or modification.

This warranty is void and of no force and effect in the event of product misuse, improper use, abuse, neglect, unauthorized repair or modification. This warranty specifically excludes EVENCARE G2 Test Strips and EVENCARE G2 Control Solution.

THIS WARRANTY IS THE SOLE AND EXCLUSIVE WARRANTY TO THE EXCLUSION OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, ALL OF WHICH ARE WAIVED AND DISCLAIMED.

All warranty claims must be directed to Medline's Technical Service Center at 1-866-274-2131. This warranty is extended only to the original purchaser of the meter.

Specifications

Test Strips:	EVENCARE G2 Blood Glucose Test Strips
Test Range:	20 - 600 mg/dL
Calibration:	Plasma
Test Time:	6 Seconds
Sample Size:	Larger than 1uL
Hematocrit Range:	30-55%
Display Type:	Large LCD screen
Memory:	300 blood test results with date and time
Result Averaging:	7, 14, and 30 days averaging
Dimension:	93L x 58W x 20.5H (mm)
Weight:	79g
Batteries:	2 Alkaline AAA Batteries
Battery Life:	1000 tests of continuous use or 1 year
Automatic Power-off:	After 2 minutes of nonuse
Operating Temperature:	50°F to 104°F
Relative Humidity:	Less than 85%
Storage Condition:	Meter at 39°F - 122°F Test Strips at 39°F - 86°F

For additional information, refer to the EVENCARE G2 Test Strip insert.

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www.medline.com ©2016 Medline and EVENCARE are registered trademarks and G2 is a trademark of Medline Industries, Inc. Dispatch and Clorox Healthcare are registered trademarks of The Clorox Company Corp. Micro-Kill is a trademark of Medline Industries, Inc. Manufactured in Taiwan for Medline Industries Inc, Mundelein, IL 60060 **1-800-MEDLINE** RH16APB U.S. Patent # 7,655,128