



Instructions for Use and Cleaning – NON134000(CS)

Please see the steps listed below for usage of item NON134000 and NON134000CS – Individually Boxed Blue Pill Crusher:

1. Unscrew the blue cap and place pill(s) inside the crusher
 - a. Medline does not recommend crushing more than three small pills or two large pills at the same time
2. Screw the blue cap onto the clear base and fully tighten
 - a. Ensure that the cap is threaded correctly on the base before tightening
3. Continue to loosen and tighten the cap until the pill(s) are finely ground

Please note that this product is for single-patient use. To prevent cross-contamination, the cap and base should be cleaned prior to each use. Medline suggests rinsing out with water then using an isopropyl alcohol prep pad and letting air-dry in order to clean this product.

David
Geldermann

Digitally signed by David Geldermann
DN: cn=David Geldermann, o=Medline
Industries, Inc., ou=Primary Care Quality,
email=dgeldermann@medline.com, c=US
Date: 2016.06.14 11:50:19 -05'00'

Signature

June 14, 2016

Date

David Geldermann
Quality Assurance Specialist
Primary Care Division