

5 PHYSICIAN PRESCRIPTION AND DETAILED WRITTEN ORDER

Phys. Name _____
 Address _____
 Suite _____
 City _____ State _____ Zip _____
 Phone (____) _____
 Fax (____) _____
 Specialty _____

I prescribe a Vacuum Erection Device (CPT/HCPCS Code L7900). It is my expert opinion that a vacuum device is medically necessary to facilitate management of his dysfunction. My patient has been diagnosed with Organic Impotence (Code 607.84) as a result of the following condition(s):

- | | |
|---|---|
| <input type="checkbox"/> 185 Carcinoma of the Prostate | <input type="checkbox"/> 401.9 Hypertension |
| <input type="checkbox"/> 188.9 Carcinoma of the Bladder | <input type="checkbox"/> 952.9 Spinal Cord Injury |
| <input type="checkbox"/> 443.9 Peripheral Vascular Disease | <input type="checkbox"/> 154.0 Colorectal Cancer |
| <input type="checkbox"/> 600.00 BPH w/o Obstruction | <input type="checkbox"/> 257.2 Hypogonadism |
| <input type="checkbox"/> 600.01 BPH w/ Obstruction | <input type="checkbox"/> 414.00 Coronary Artery Disease |
| <input type="checkbox"/> 250.00 Non-Insulin Dependent Diabetes Mellitus | |
| <input type="checkbox"/> 250.01 Insulin Dependent Diabetes Mellitus | |
| <input type="checkbox"/> Other _____ | |

Doctor Signature _____ Date _____
 NPI# _____

THE COMPLETE VACURECT™ RX MODEL 1001

System Includes:

- RX Vacuum Unit
- Deluxe Travel Pouch
- 10 Different Sizes of Tension Systems (1 (TS) lasts up to three years!)
- Service Oil
- Personal Lubricant - 4oz.
- Simple User Instructions

This product carries a 30 day Satisfaction Guarantee with an EXCHANGE option in addition to a 5 year FREE Replacement Warranty on the Vacuum unit against defects.

SAFE, SIMPLE, COMPACT, EFFECTIVE & FDA REGISTERED

Vacurect



VACUUM THERAPY THE PRACTICAL SOLUTION

Although other treatments for ED (e.g. Viagra®, Injections and Penile implants) are widely used today, Vacuum Therapy is gaining increasing popularity by patients and their physicians because of the very positive associated benefits. In fact, Vacuum Therapy should be used as a daily exercise to prevent atrophy and to promote penile health. Evidence from leading urologists promotes the idea that this treatment improves penile health by restoring adequate blood flow and revitalizing vascular tissues.

THE VACURECT™ THE ULTIMATE IN VACUUM THERAPY

The Vacurect™ is setting a new standard in vacuum therapy, and has already restored thousands of relationships. The Vacurect™ is compact, efficient, and simple to operate. It uses a natural pumping action that does not detract from natural intimacy. The Vacurect™ uses a unique double-sealing Tension System that ensures maximum efficiency. Even better, the Vacurect™ comes with 10 Tension Systems of different sizes that last up to three years (no costly Tension Systems to buy every 3 months!)

Approved by Medicare & Most Insurance



HOW THE VACURECT™ WORKS



STEP 1
Attach the tension system to the base of the cylinder by pressing firmly.



STEP 2
Apply a dime size amount of personal lubricant on the index finger and apply to the glans head of the penis. Apply the excess lubricant to the opening of the tension system.



STEP 3
Gently rest the tip of the penis in the opening of the tension system.



STEP 4
Move the sleeve up and down the cylinder to create the vacuum; this will draw the penis through the tension system into the cylinder.

STEP 5
After creating the erection, remove the cylinder from the tension system, leaving the tension system in place at the base of the penis.



COMPETITOR'S VACUUM ERECTION DEVICES

VACURECT™ ONE PIECE DESIGN

Other Vacuum Pumps typically are bulky, cumbersome, hard to incorporate during foreplay and not sexually appealing. These qualities proved to be significant drawbacks to the advancement of vacuum therapy among patients.

The Vacurect™ is the first and only system that allows you to enjoy the benefits of Vacuum Therapy without the drawbacks. The Vacurect™ is one-piece, easy to use, very compact, and always reliable.

VIEW OUR INSTRUCTIONAL VIDEO ON OUR WEBSITE WWW.BONRO.COM

Covered by Medicare & Most Medical Insurance



"For the last five years, my husband and I have been using an early model vacuum system...it was awkward to include in our love making and the rings or bands were uncomfortable for my husband. Now the Vacurect™ has unbelievably enhanced our intimate moments together." Clifton, NJ