


Incontinence care

Application guide for briefs

After the soiled product has been removed and the skin cleansed, assessed and protected, you are ready to apply the new brief, per facility policy.

Lying down:

- 1 Completely unfold the new brief lengthwise, including the side panels.
- 2 Hold brief at each end and gently pull, activating the leg gathers to create a "cupped" shape for optimal fit and performance.
- 3 Roll patient to one side and spread out the brief, including side panels, underneath patient's buttocks with tape tabs facing the head of the bed.
- 4 Roll the patient back over.
- 5 Pull the front of the product up into the perineal area. Center it front to back, making sure the product is symmetrically positioned on the body.

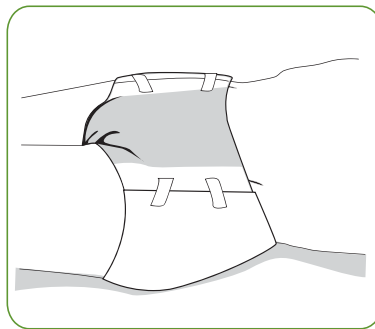

Standing:

- 1 Completely unfold the new brief lengthwise, including the side panels.
- 2 Hold brief at each end and gently pull, activating the leg gathers to create a "cupped" shape for optimal fit and performance.
- 3 Place the product between the thighs from the front with tape tabs to the rear.



Remaining application steps:

- 1 Spread the front side panels against the hips toward the back of the body.
- 2 Then overlap the rear side panels on top of these toward the front of the body. Use the hook and loop tabs to secure the brief. The tape tabs always go on the front of the body.

The bottom two tabs should be angled slightly upward. The top two tabs should point straight across, or be angled slightly downward.
- 3 Press down on the tabs to secure them to the outside.
- 4 Smooth out all wrinkles and folds and check the fit of the product. Make any adjustments as necessary.

Remember, the closure tabs can be unfastened and fastened again, if needed.
- 5 Gently pull the skin of each inner thigh downward and away from the perineal area. This allows the leg cuffs inside the brief to move into place.