

Medifil® II



Medifil® II Collagen Particles are bovine collagen fibers bioengineered for advanced wound care applications using our proprietary Kollagen™ technology.

Available Size	HCPCS* Code
1 gram vial	A6010
Reorder Number	MF2001



*All products are reimbursable under Medicare in the USA.

Medifil® II Collagen Particles

Indications:

Medifil® II Collagen Particles are indicated for the management of burns, sores, blisters, scrapes and ulcers

Properties:

- Super-absorbent
- Removes exudate
- Allows gaseous exchange
- Creates an environment which may be conducive to the normal healing process
- Shelf life of five years
- Cost effective

Application:

1. Debride and clean the wound. Leave the wound moist to facilitate the action of Medifil II®.
2. Apply Medifil II® particles up to 1/4 inch depth.
3. Apply a dressing if needed.
4. Wash with saline solution when particles become saturated.

Precautions:

- Do not pack the wound tightly; allow for expansion of the particles
- Wounds may appear larger during the first few days of treatment due to the reduction of edema
- Treatment of the underlying condition (venous or arterial flow, pressure, etc.) should proceed concurrently with the use of Medifil II®
- Medifil II® is insoluble collagen and should never be used as an injectable soluble collagen
- All open and unused portions must be discarded
- **Do not use if sensitive to materials of bovine origin**