



Safety Information

Children with challenging behavioral problems or severe involuntary movement may not be suitable for the P Pod due to such excessive body movement posing the risk of severe imbalance during use. A careful risk assessment with a healthcare professional is highly recommended.

Measurements for P Pod



Always keep the tape measure straight across the body.

Do not wrap the tape measure around the body.



Hip Width

Measure from outside of the hips to the outside of the hips at the widest point.



Trunk Width

Measure across the chest from armpit to armpit at the widest point.



Seat Depth

Position the leg at a right angle. Measure from the buttock to the back of the knee.



Axilla Height

Position the leg at a right angle. Measure from the rear of the buttock to the armpit.



Shoulder Height / Head Height

Position the leg at a right angle. Measure from the bottom of the rear of the buttock to the top of the shoulder for shoulder height; to the top of the head for head height.



Knee-to-Heel Height

Bend the hip and knee at a right angle. Measure from the back of the knee to the bottom of the heel.



MODEL NUMBER	APP-1000XS	APP-2000S	APP-3000M	APP-4000L	APP-5000XL
Product Size	Extra Small	Small	Medium	Large	Extra Large
Upholstery	MP/PB/SG	MP/PB/SG	MP/PB/SG	MP/PB/SG/WB/JP/SR	MP/PB/SG/WB/JP/SR
Hip Width	10"	10.25"	12.25"	14.25"	16.5"
Trunk Width	8.75"	10"	10.5"	12.5"	13.5"
Seat Depth	9"	11.5"	13"	13.75"	17"
Axilla Height	10.25"	11.5"	12.5"	15"	16.5"
Shoulder Height	14.5"	16.5"	17.25"	20"	21.75"
Head Height	19.75"	22"	24.5"	26"	28.75"