

## Instruction for Use: Carefix CarePocket® (art. no. 0912)

### Indication:

Medical device for retention of urine leg bag.

### Features:

CarePocket® provides a pocket for retention of a urine leg bag. It has a slit at the bottom for the valve of urine bags with bottom tap opening. The sleeve can contain urine leg bag of volumes up to 750 ml/24 oz. Suitable for use on both thigh and calf. With outer seams.



### Precautions:

It is recommended to empty the urine bag when it is half full. If you have any questions regarding use and/or correct fitting, or if any unexpected discomfort or skin irritation occurs, please consult your healthcare professional.

### Instruction for use:



1. Pull on the leg bag holder (ensure the color size code is at the top).



2. Place the leg bag in the pocket and pull the tap through the lower hole.



3. The CarePocket® is in place.

### Composition:

80% Polyester, 20% Elastane.



### Care Handling:



140°F

Best durability achieved by washing at 60°C/140°F in washing bag followed by gentle tumble-drying. Use of chlorine based bleach or chlorine based disinfection will decrease durability.

**Please keep this instruction for further reference.**



MAKING HEALTHCARE MORE HUMAN

## Instruction for Use: Carefix CarePocket<sup>®</sup> (art. no. 0912)

---

### Customer Service:

E-mail: [customerservice@tytex.com](mailto:customerservice@tytex.com)

International: +45 9660 4200; +49 2262 762914

Americas: +1-855-569-6131



Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485  
Made in Slovakia by: TYTEX A/S · Industrivej 21 · DK-7430 Ikast · [www.tytex.com](http://www.tytex.com)

