

Nasco's Life/form® Testicular Exam Simulator

2 Pages

13.



**First Aid Arm
LF01005U
Instruction Manual**




Life/form. Products by Nasco

First Aid Arm LF01005

4 Pages

14.

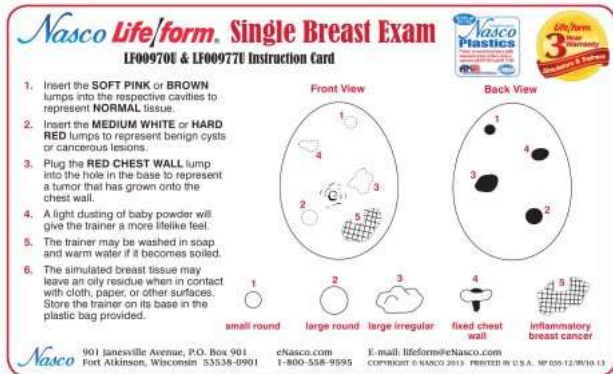

**Nasco Life/form. Single Breast Exam
LF00970U & LF00977U Instruction Card**

1. Insert the **SOFT PINK** or **BROWN** lumps into the respective cavities to represent **NORMAL** tissue.
2. Insert the **MEDIUM WHITE** or **HARD RED** lumps to represent benign cysts or cancerous lesions.
3. Plug the **RED CHEST WALL** lump into the hole in the base to represent a tumor that has grown onto the chest wall.
4. A light dusting of baby powder will give the trainer a more lifelike feel.
5. The trainer may be washed in soap and warm water if it becomes soiled.
6. The simulated breast tissue may leave an oily residue when in contact with cloth, paper, or other surfaces. Store the trainer on its base in the plastic bag provided.

Front View **Back View**

1 2 3 4 5

small round large round large irregular fixed chest wall inflammatory breast cancer

Nasco 901 Janesville Avenue, P.O. Box 901 Fort Atkinson, Wisconsin 53538-0901 eNasco.com 1-800-558-9595 E-mail: lifeform@Nasco.com COPYRIGHT © NASCO 2013 PRINTED IN U.S.A. NP 005-12.0910.13

LF00970

2 Pages

15.