

**PHILIPS**

**RESPIRONICS**

*DreamWear*


Under the nose  
nasal mask

# Sizing guide

## Instructions:


- 1 Print out this Adobe PDF File at 100% on your printer. Use the print accuracy check at right to ensure printing at the proper size.
- 2 Fold the bottom part of this page on the dashed line.
- 3 Cut along the dotted line around the fitting gauge.
- 4 Place the fitting gauge on your nose as shown in the picture below.
- 5 Determine the appropriate size by which area your nose reaches.  
(S – Small, M – Medium or L - Large)

Philips Respiroics always recommends consulting a physician or registered Sleep Therapist in fitting a mask to ensure a proper sizing and fit.

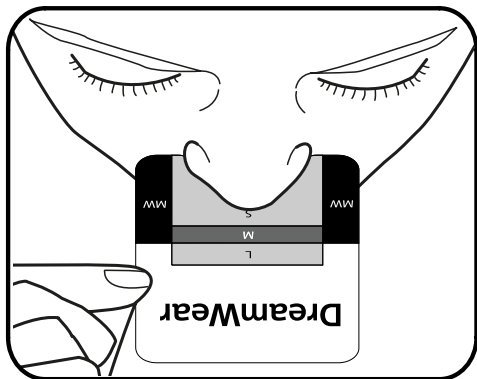


**FOLD** along dashed line

## Print accuracy check

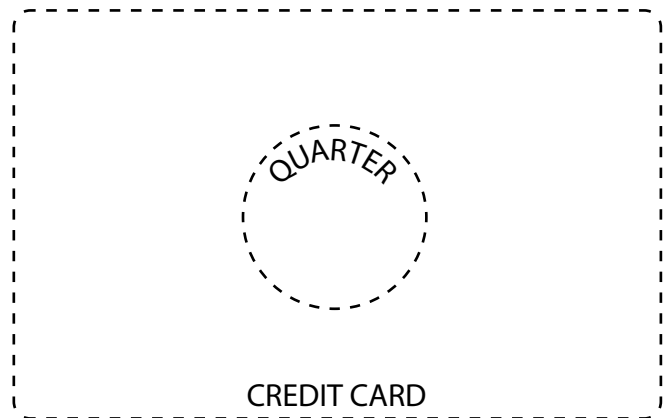

## Fitting gauge

**CUT** along dotted line

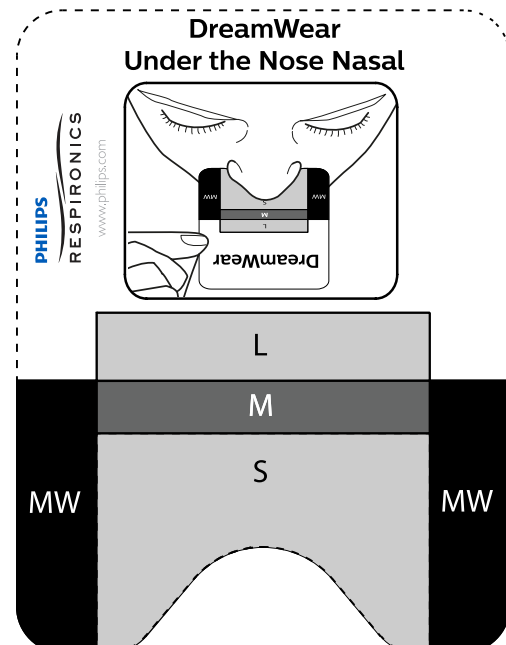

[Click here to view detailed product descriptions and prices.](#)

**Call 800-397-5899 to order today!**